<p><i>Note: This portion of the document should not appear on produced Labels or IFUs.</i></p> <p>Stryker Instruments (269) 323-7700 (800) 253-3210</p>	<p><b>Dimensions:</b> 8.5 inch (width) x 11 inch Folded to 8.5 inch (width) x 5.5 inch</p>	<p><b>Color/Material/Finish:</b> Black Graphics on White Background 20# Bond or Equivalent</p>	<p><b>Label Stock:</b> N/A</p>	
			<p><b>Description/Type:</b> Instructions For Use</p>	
			<p><b>Part Number:</b> 0206-520-011</p>	<p><b>Rev.</b> K</p>

Rx ONLY

CE 0197

STERILE R

EN

## For Use With

**WARNING:** Use only Stryker-approved equipment, unless otherwise specified. DO NOT modify any equipment.

The 180.0 g Cement Cartridge is intended for use with the Automatic High Vacuum Foot Pump (REF 0206-500-000), a Stryker cement mixing system, and a Stryker cement gun.

DESCRIPTION	REF
MixEvac® 3 Bone Cement Mixer	0206-015-000
Advanced Cement Mixing (ACM®) System	0306-XXX-000 series
Revolution® Cement Mixing System	0606-553-000
Dual Speed Cement Injection Gun	0206-600-000
Late Injection Cement Gun	0306-600-000
Percutaneous Cement Gun	0306-650-000
Friction Cement Gun	0606-100-000
Revolution Cement Gun	0606-600-000
RMA Revolution Cement Gun	0606-600-000S1

## Safety Directives

**WARNINGS:**

- Before using this equipment, or any component compatible with this equipment, read and understand the instructions for use. Pay particular attention to safety information. Become familiar with the equipment before use. Failure to comply may result in patient and/or healthcare staff injury.
- Upon initial receipt and before use, inspect the package for damage and confirm the integrity of the sterile barrier. DO NOT use any equipment if damage is apparent or the sterile barrier has been compromised. Failure to comply may result in patient and/or healthcare staff injury.
- DO NOT use the equipment after the expiration date printed on the package. The equipment may not be safe or effective after the expiration date.
- Upon initial receipt and before use, inspect each component for damage. DO NOT use any equipment if damage is apparent. Failure to comply may result in patient and/or healthcare staff injury.

ENGLISH (EN)	DANSK (DA)
ESPAÑOL (ES)	SUOMI (FI)
DEUTSCH (DE)	PORTUGUÊS (PT)
FRANÇAIS (FR)	POLSKI (PL)
ITALIANO (IT)	日本語 (JA)
NEDERLANDS (NL)	中文 (ZH)
SVENSKA (SV)	한국어 (KO)

- DO NOT reuse, reprocess, or repackage a device that is intended for single use only. A single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing. Design features may make cleaning difficult. Reuse may create a contamination risk and compromise structural integrity resulting in operational failure. Critical product information may be lost during repackaging. Failure to comply may lead to infection or cross-infection and result in patient and/or healthcare staff injury.
- ALWAYS follow the current local recommendations and/or regulations governing environmental protection and the risks associated with recycling or disposing of the equipment at the end of its useful life.
- Cartridges cannot withstand the excessive pressure that is created when doughy cement is extruded. DO NOT attempt to extrude the cement if excessive force is necessary. In this situation, remove the application tip and hand pack the cement. Excessive force can cause the cartridge to break. Pieces of the broken cartridge can harm patient and staff.

- Place the piston in the bottom of the cartridge.
- Place the funnel on the cartridge.
- Carefully pour the cement into the cartridge.

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