| | | | | |
|--|--|--|--|------------------------------------|
| <p>NOTE: This portion of the document should not appear on produced Labels or IFUs.</p> <p>Stryker Instruments (269) 323-7700 (800) 253-3210</p> | <p><u>Dimensions:</u></p> <p>8.5 inch (width) x 11 inch Booklet</p> | <p><u>Color/Material/Finish:</u></p> <p>Black Graphics on White Background 20# Bond or Equivalent</p> | <p><u>Label Stock:</u></p> <p>N/A</p> | |
| | <p><u>Print Location:</u></p> <p>SPR</p> | <p><u>Suppliers/Services:</u></p> <p>Crimson Life Sciences</p> | <p><u>Description/Type:</u></p> <p>Instructions For Use</p> | |
| | | | <p><u>Part Number:</u></p> <p>0306-553-702</p> | <p><u>Rev.</u></p> <p>P</p> |

Stryker ACM® (Advanced Cement Mixing) System

REF 0306-XXX-XXX Series

Instructions For Use

R_x ONLY CE 0197

EN

ACM allows you to mix cement with a constant high vacuum throughout the process. Air bubbles are removed for the virtual elimination of porosity. Cement is transferred directly from the mixing bowl to the application cartridge while under vacuum.

User/Patient Safety


WARNINGS:

- Read and understand the instructions. If you have questions, contact your Stryker sales representative or Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.
- Before use in the clinical environment, practice using the system with acrylic bone cement. Mixing time and proper application can be technique dependent.
- Upon initial receipt and before use, inspect the package for damage and confirm the integrity of the sterile barrier. DO NOT use any equipment if damage is apparent or the sterile barrier has been compromised.
- DO NOT reuse, reprocess, or repack a single use device. A single use device is intended for a single use only. A single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing. Design features may make cleaning difficult. Reuse may create a serious risk of contamination and may compromise the structural integrity of the single use device resulting in operational failure. Critical product information may be lost if the single use device is repackaged. Failure to comply may lead to infection or cross infection and result in patient and/or healthcare staff injury.
- Use only Stryker approved accessories.


CAUTION: Mix enough cement to allow for losses in the application tip and cement penetration, especially in total hip procedures.

Prepare and Organize Equipment

The ACM System is used in conjunction with Stryker Dual Speed Cement Injection Gun and High Vacuum Pump or Automatic High Vacuum Pump.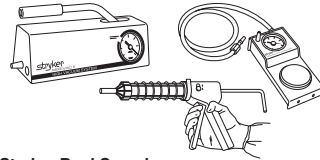

Stryker Dual Speed Injection Gun: Turn the ram to the RELEASE position, then pull all the way back. Set aside until ready to use.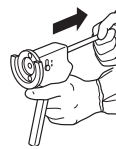

Application Tips and Accessories: Select appropriate tips and accessories. Remove them from their packages using proper sterile technique. Set aside until ready to use.

Cement Media: Remove media components from their outer packages using proper sterile technique. Set aside until ready to use.


Vacuum Pump: Place the vacuum pump outside sterile field. If using the Stryker Portable High Vacuum Pump, it may be sterilized and placed on the sterile field.

Nitrogen Source:


WARNING: Reduce regulated pressure on nitrogen source to zero before connecting to Stryker Automatic High Vacuum Pump.

Connect nitrogen source to vacuum pump.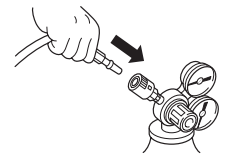

Set pressure to 70-100 psi.


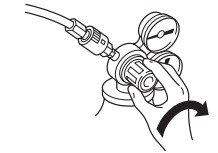
WARNING: Do not operate pump outside the recommended pressure range.

Instructions


NOTE: Mixing times are approximate and may vary with room temperature. Do not place the cement mixing components where they will be warmed to above 70 °F.

1. Remove bowl assembly from blister using proper sterile technique.
2. Attach vacuum tube to vacuum pump.
3. Remove the lid from the bowl by turning the lid counterclockwise. Set the lid aside taking care not to bend the paddles.
4. Pour bone cement powder into the bowl followed by liquid monomer.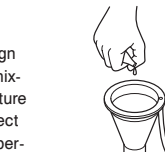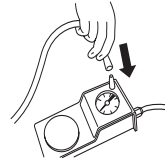

WARNING: DO NOT add foreign substances to the cement's mixture. The alteration of the mixture with such substances can affect the cement's mechanical properties and setting time during use.

5. Return the lid to the bowl; turn clockwise to secure. Depress footswitch on the Automatic High Vacuum Pump or manually activate the Portable High Vacuum Pump. Evacuate the bowl until the gauge reads 20-22 inHg.
6. Mix back and forth until the powder and liquid are blended. Tap unit on the work surface to dislodge powder from sides of bowl and into the mixture.

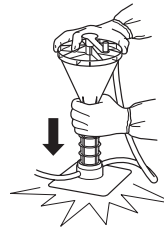7. Once the mixture is blended, rotate the paddles clockwise to further remove air from the mixture.
8. When mixing is complete, firmly depress the transfer rod.

9. Rotate the paddles clockwise until the cement is completely transferred into the cartridge.


10. Peel vent tab (VACUUM RELEASE) from lid to release vacuum from bowl.


WARNING: Never release the vacuum by removing the vacuum hose from the bowl or pump.

11. Grasp the cartridge and remove the bowl and cartridge from base with a counterclockwise turn. Keep the cartridge upright.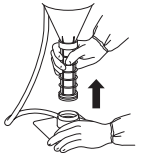


12. With other hand, separate cartridge from bowl with a counterclockwise turn.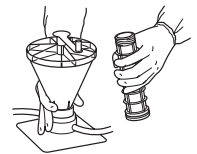


13. Set bowl into base and set aside. These components may be discarded.

14. Place desired application tip on the cartridge.

NOTE: Align the tab with the arrow on the cartridge then turn the tip so that the tab slides over the ramp and latches to secure the tip.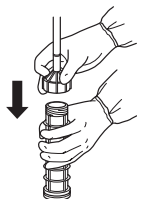


15. Snap the cartridge into the Dual Speed Injection Gun.

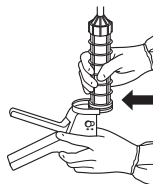16. Hold the gun and cartridge vertically and fill the tip. Turn the ram to the INJECT position. The FILL setting on the side of the gun will yield an extrusion rate of 9 mL per stroke. The PRESSURIZE setting will extrude 3 mL per stroke with greater mechanical advantage.

WARNING: The cartridge cannot withstand the excessive pressure that is created when doughy cement is extruded. DO NOT attempt to extrude the cement if excessive force is required. In this situation, remove the application tip and hand pack the cement. Excessive force can cause the cartridge to break. Pieces of the broken cartridge can harm the patient and staff.

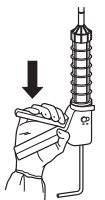ENGLISH (EN) NEDERLANDS (NL) NORSK (NO)
 ESPAÑOL (ES) SVENSKA (SV) POLSKI (PL)
 DEUTSCH (DE) DANSK (DA) 日本語 (JA)
 FRANÇAIS (FR) SUOMI (FI) 中文 (ZH)
 ITALIANO (IT) PORTUGUÊS (PT) 한국어 (KO)

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stryker[®]

Signature History Report

| Version | Signature | Role | Event Date | Vote |
|---------|---------------------------|-------------------------------|-------------------------|---------------------|
| P | Capouch, Don (Contractor) | CA_Owner | 2014-10-30 16:16:10 EDT | Continue |
| P | Pearson, Dawn | Editor | 2015-03-12 14:54:39 EDT | Continue |
| P | Gomaa Sara | CA_Owner | 2015-03-12 22:26:05 EDT | Continue |
| P | Shen, Jean | CA_Regulatory Affairs | 2015-03-18 16:03:33 EDT | Approve |
| P | Curtis, Megan | Curtis, Megan | 2015-03-20 14:45:08 EDT | Approve |
| P | William.Sykes@stryker.com | CA_Product Safety | 2015-03-26 12:34:55 EDT | Approve |
| P | Gomaa Sara | CA_Owner | 2015-03-26 13:35:05 EDT | Withdraw |
| P | Gomaa Sara | CA_Owner | 2015-03-26 13:55:34 EDT | Return to Editor |
| P | Drilich, Emily | Editor | 2015-04-09 09:28:16 EDT | Continue |
| P | Gomaa Sara | CA_Owner | 2015-04-09 10:00:41 EDT | Continue |
| P | Curtis, Megan | Curtis, Megan | 2015-04-09 10:25:17 EDT | Approve |
| P | Shen, Jean | CA_Regulatory Affairs | 2015-04-09 16:05:36 EDT | Approve |
| P | William.Sykes@stryker.com | CA_Product Safety | 2015-04-10 09:43:59 EDT | Approve |
| P | Santana Gonzalez, Michael | CA_Product Design Engineer | 2015-04-10 15:01:39 EDT | Approve |
| P | Reyes Santiago, Luis | CA_Quality Engineer | 2015-04-13 09:12:43 EDT | Approve |
| P | Snell, Tami (Contractor) | Document Control | 2015-04-13 15:14:12 EDT | - |
| P | Drilich, Emily | Editor | 2015-06-03 13:34:15 EDT | Continue |
| P | Gomaa Sara | CA_Owner | 2015-07-31 10:35:42 EDT | Approve |

NOTE: IF THIS REPORT IS BLANK, PLEASE SEE WINDCHILL ECN FOR SIGNATURES.

Related Information

| | |
|---------------------------------|----------------------------------|
| CR Number | CR0204309 |
| CR Class | DMR-Commercially Released |
| Originating Entity | INST: Kalamazoo |
| Origin | Other |
| Initial CR Approval Date | 10/27/2014 07:39:39 |
| CN Number | CN91649 |
| CA Number | CA93224 |
| NPD Disposition Plan | |