<p>NOTE: This portion of the document should not appear on produced Labels or IFUs.</p> <p>Stryker Instruments (269) 323-7700 (800) 253-3210</p>	<p><u>Dimensions:</u> 5.5 inch (width) x 8.5 inch Booklet</p>	<p><u>Color/Material/Finish:</u> Color Graphics on White Background 20# Bond or Equivalent</p>	<p><u>Label Stock:</u> N/A</p>	
	<p><u>Print Location:</u> Print Center</p>	<p><u>Suppliers/Services:</u> TransPerfect MDS</p>	<p><u>Description/Type:</u> Instructions For Use</p>	
			<p><u>Part Number:</u> 0702-034-700</p>	<p><u>Rev.</u> H</p>


stryker®

Neptune® 2 Waste Management System Fluid Suction HEPA Filter

REF 0702-034-000

Instructions For Use

R_x ONLY

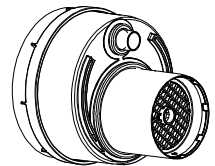ENGLISH (EN)
ESPAÑOL (ES)
DEUTSCH (DE)
FRANÇAIS (FR)
ITALIANO (IT)
NEDERLANDS (NL)
SVENSKA (SV)
DANSK (DA)
SUOMI (FI)
PORTUGUÊS (PT)
NORSK (NO)
POLSKI (PL)
ΕΛΛΗΝΙΚΑ (EL)
日本語 (JA)
中文 (ZH)
한국어 (KO)

Indications For Use

The Neptune 2 Waste Management System is intended to be used in the operating room, pathology, surgical centers, and doctor's offices to collect and dispose of surgical fluid waste as well as collect smoke generated from electrocautery or laser devices.

Contraindications

The Neptune 2 Waste Management System is **contraindicated against:**

- Connection directly to **chest tubes**.
- Connection to **closed wound drainage systems**.

Contact Information

NOTE: For additional information, especially safety information, or in-service training, contact your Stryker sales representative or call the Stryker Neptune Service Center at 1-800-550-7836.

For Use With

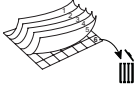
The Stryker Fluid Suction HEPA Filter REF 0702-034-000 (filter) is for use with the Neptune 2 Waste Management System, specifically the rover.

Description

The filter provides high efficiency particulate air (HEPA) filtration of the air evacuated from the fluid collection canister(s) of the rover before the air is vented.

Symbol Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	GENERAL WARNING SIGN
	CONSULT INSTRUCTIONS FOR USE
	REPLACE THIS FILTER EVERY 6 MONTHS OR AS INDICATED BY THE ROVER.
	DO NOT MAKE CONTACT WITH THE FILTER MEDIA.

User/Patient Safety

WARNINGS:

- Before using any system component, or any component compatible with this system, read and understand the instructions. Pay particular attention to WARNING information. Become familiar with the system components prior to use. Failure to comply may result in patient and/or healthcare staff injury.
- Only trained and experienced healthcare professionals should use this equipment.
- Upon initial receipt, inspect each component for damage. DO NOT use any equipment if damage is apparent. Failure to comply may result in patient and/or healthcare staff injury.

- Use only Stryker-approved system components and accessories, unless otherwise specified. Failure to comply may result in patient and/or healthcare staff injury.
- DO NOT disassemble, modify, service, or repair any system component or accessory, unless otherwise specified. Call Stryker Neptune Customer Service.
- BLOODBORNE PATHOGEN HAZARD
 - ALWAYS wear personal protective equipment (PPE) when operating or handling this equipment.
 - ALWAYS leave tubing attached to the manifold and close unused ports during disposal.
 - The used filter is biohazard waste. ALWAYS follow local regulations regarding proper handling and disposal of biohazard waste.

Failure to comply may cause infection and result in healthcare staff injury.

- DO NOT make contact with the filter media (Figure 1). The media filters the air evacuated from fluid collection canister(s) of the rover before the air is vented. Failure to comply may cause media damage and result in patient and/or healthcare staff injury.

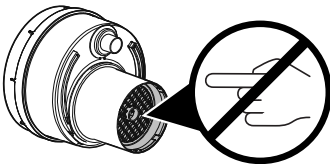
Figure 1 – To Prevent Filter Media Damage

Instructions

NOTE: See the instructions for use supplied with your rover for control panel and error message information. Because of rover configuration differences, some instructions and illustrations contained in this filter manual may not be exactly like those found in your rover instructions for use. However, the procedures to replace the filter and reset the timer are not affected.

To Replace the Filter

NOTES:

- Replace the filter every six months or as indicated on the control panel display of the rover. Failure to comply will result in the reduction of fluid suction or an increase in odor.
- The filter life is 500 hours. The rover calculates filter usage time automatically. ALWAYS reset the filter timer after the filter is replaced. See *To Reset the Filter Timer* procedure.

1. Make sure the rover power switch is OFF.
2. Remove the door from the filter compartment (Figure 2).

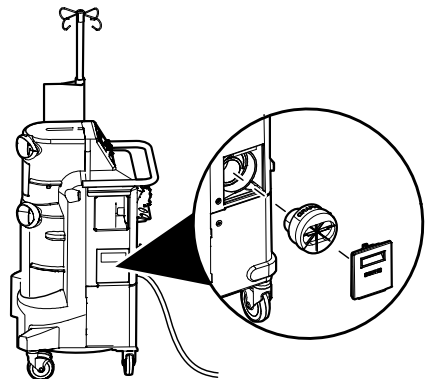
Figure 2 – To Remove and Install Filter

To Replace the Filter (continued)

3. Remove and properly dispose of the used filter.
4. Mark the replacement date on the label of the new filter.
5. Install the new filter into the filter compartment and replace the door (Figure 2).
6. Make sure the power cord is connected between the rover and facility power.
7. Push the rover power switch to the ON position and acknowledge the HIGH SUCTION DEVICE WARNING.
8. Press the SUCTION ON/OFF button and turn the SUCTION LIMIT SETTING dial of either canister to the maximum suction level. Make sure the maximum suction level value on the CONTROL PANEL DISPLAY is obtained (Figure 3).

NOTE: Make sure there are no manifolds installed in the rover.

9. If the suction level is NOT reached, reposition the filter. If the suction level is still NOT reached, contact the Stryker Neptune Service Center.

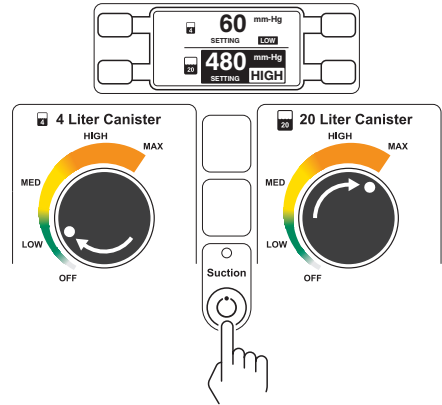
Figure 3 – To Apply Suction

To Reset the Filter Timer

1. Make sure the power cord is connected between the rover and facility power.
2. Make sure the power switch is in the ON position and the HIGH SUCTION DEVICE WARNING has been acknowledged.
3. From the control panel, push and hold the push button until the SYSTEM SETUP screen appears (Figure 4).

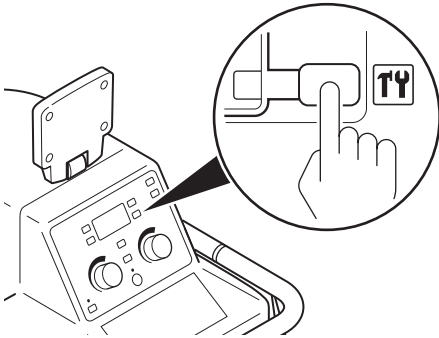
Figure 4 – To Access System Setup

4. From the SYSTEM SETUP screen, push the buttons next to the arrow icons to highlight the SUCTION FILTER LIFE. Push the button next to OK to make selection (Figure 5).

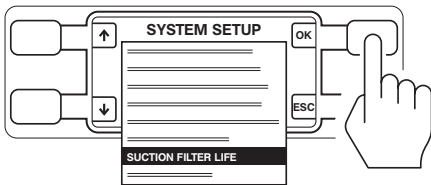
Figure 5 – To Select Suction Filter Option

5. The SUCTION FILTER LIFE screen will appear. Press the display button (RESETS TIMER) to change the current value of the filter life to 500 hours (Figure 6).

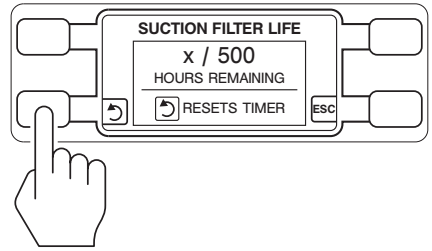
Figure 6 – To Reset Timer Value

6. Press the display button (OK) to accept the change. Press the display button (ESC) to cancel the change (Figure 7).

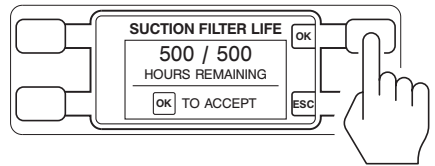
Figure 7 – To Accept Timer Value

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stryker[®]

Signature History Report

Version	Signature	Role	Event Date	Vote
H	Yavari, Fazel	CA_Owner	2015-01-26 12:49:12 EST	Continue
H	Drilich, Emily	Editor	2015-02-06 15:46:57 EST	Continue
H	Yavari, Fazel	CA_Owner	2015-02-09 17:35:21 EST	Continue
H	Yavari, Fazel	CA_Product Design Engineer	2015-02-09 17:37:04 EST	Approve
H	Nollar, Andrew	Nollar, Andrew	2015-02-10 07:48:09 EST	Approve
H	Ekola, David	CA_Quality Engineer	2015-02-10 08:13:31 EST	Approve
H	Osborn, Nate	CA_Product Safety	2015-02-12 15:24:42 EST	Approve
H	Wiersema, Kim	CA_Regulatory Affairs	2015-02-19 12:41:02 EST	Rework
H	Yavari, Fazel	CA_Owner	2015-02-19 13:24:29 EST	Return to Editor
H	Drilich, Emily	Editor	2015-02-23 14:53:17 EST	Continue
H	Yavari, Fazel	CA_Owner	2015-02-23 15:34:00 EST	Continue
H	Yavari, Fazel	CA_Product Design Engineer	2015-02-23 15:34:23 EST	Approve
H	Wiersema, Kim	CA_Regulatory Affairs	2015-02-23 16:18:40 EST	Approve
H	Ekola, David	CA_Quality Engineer	2015-02-24 08:46:49 EST	Approve
H	Nollar, Andrew	Nollar, Andrew	2015-02-25 09:24:37 EST	Approve
H	Roberto.Pasos@stryker.com	CA_Product Safety	2015-02-27 09:24:32 EST	Approve
H	Campbell Desiree	Document Control	2015-02-27 09:31:49 EST	-
H	Drilich, Emily	Editor	2015-03-16 13:18:15 EDT	Continue
H	Yavari, Fazel	CA_Owner	2016-04-14 17:05:58 EDT	Rework to New Rev Level

Signature History Report

Version	Signature	Role	Event Date	Vote
H	Pearson, Dawn	Editor	2016-04-29 12:35:41 EDT	Continue
H	Keeler, Stephanie	CA_Owner	2016-05-17 10:03:13 EDT	Continue
H	Wiersema, Kim	CA_Regulatory Affairs	2016-05-17 10:07:55 EDT	Approve
H	Nollar, Andrew	CA_Marketing Manager	2016-05-17 10:29:32 EDT	Approve
H	Ekola, David	CA_Quality Engineer	2016-05-17 13:24:22 EDT	Approve
H	Sahu, Sameer	CA_Product Design Engineer	2016-05-19 10:11:50 EDT	Approve
H	donna.brown@stryker.com	CA_Product Safety	2016-05-25 10:18:24 EDT	Approve
H	Snell, Tami	Document Control	2016-05-25 11:08:06 EDT	-
H	Pearson, Dawn	Editor	2016-05-26 17:28:08 EDT	Continue
H	Keeler, Stephanie	CA_Owner	2017-01-27 16:12:36 EST	Approve

NOTE: IF THIS REPORT IS BLANK, PLEASE SEE WINDCHILL ECN FOR SIGNATURES.

Related Information

CR Number	CR0220549
CR Class	DMR-Commercially Released
Originating Entity	INST: Kalamazoo
Origin	Product Improvement
Initial CR Approval Date	11/06/2014 16:18:45
CN Number	CN146732
CA Number	CA108287
NPD Disposition Plan	