1000400677K
2016/06

WCR: NONE

Produced for:
Stryker Corporation
5900 Optical Court
San Jose, CA 95138 USA
1-800-624-4422

U.S. Patents: www.stryker.com/patents

Manufactured for Stryker Endoscopy
by **TEKNA**, Inc.


Rx ONLY

StrykerVision Mount

REF 240-095-200

In this Kit:

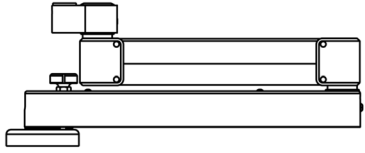NOTE: Drawings not to scale.

1x **Articulating Arm**

1x **Adapter Plate**

Hardware

- 4x 4mm x 16mm screw
- 1x 10-32 x 3/8" Flat phillips head
- 1x Keyed washer
- 4x 1/4"-20 x 3 Hex bolt
- 4x 1/4" Washer
- 4x 1/4" Lock washer
- 1x 3/32" Hex Wrench
- 1x 5/32" Hex Wrench
- 1x 7/32" Hex Wrench

NOTE: Cables removed for clarity.

IMPORTANT INFORMATION: File in your maintenance records

SAFETY PRECAUTIONS:

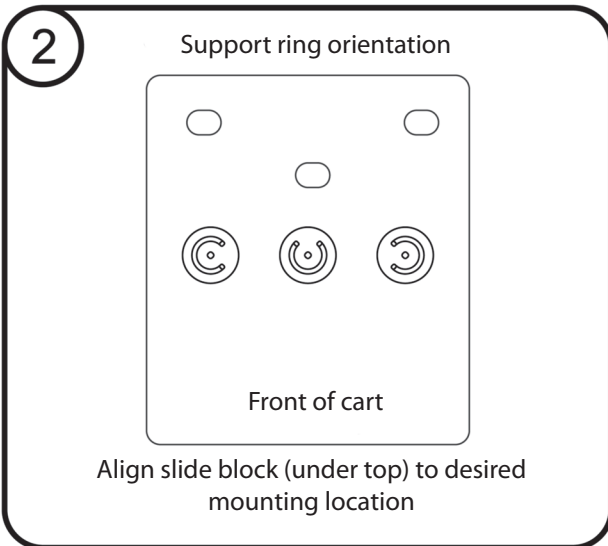
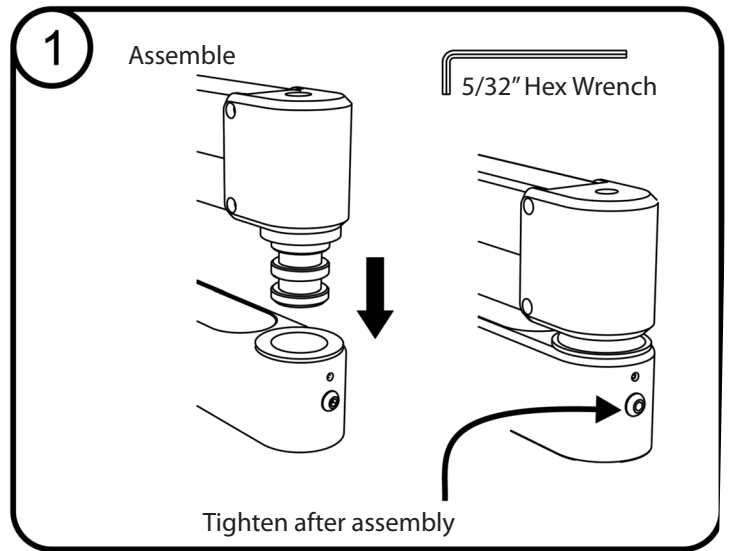
- Assembly is a two person operation.
- Improper installation may result in possible injury and/or damage to equipment.
- Maximum load is 33 lb (14.9 kg).

MAINTENANCE:



- Periodically check all hardware and re-tighten fasteners as required.
- Wipe surfaces with soap and water or disinfectant.
- Dry with non-abrasive cloth or paper towel.

NOTE: This cart is designed for mounting a single arm only.

Assembly

Fasten in this order

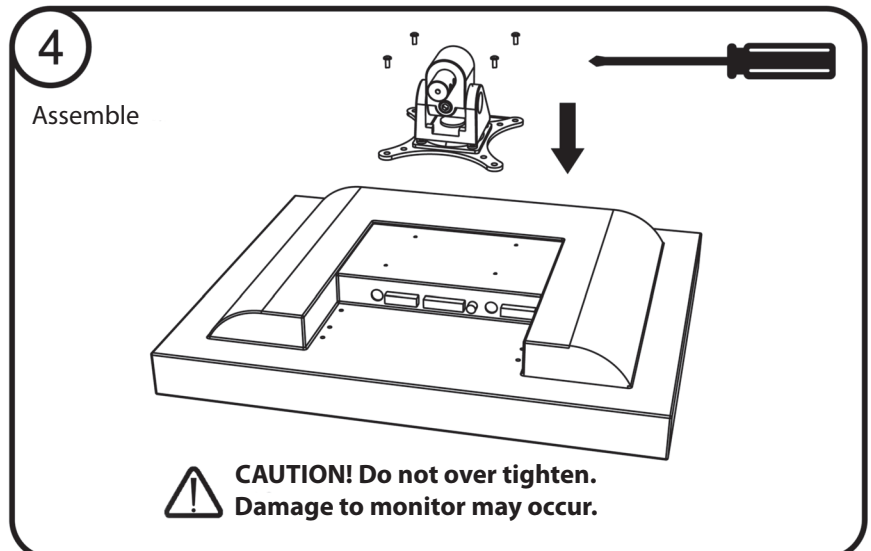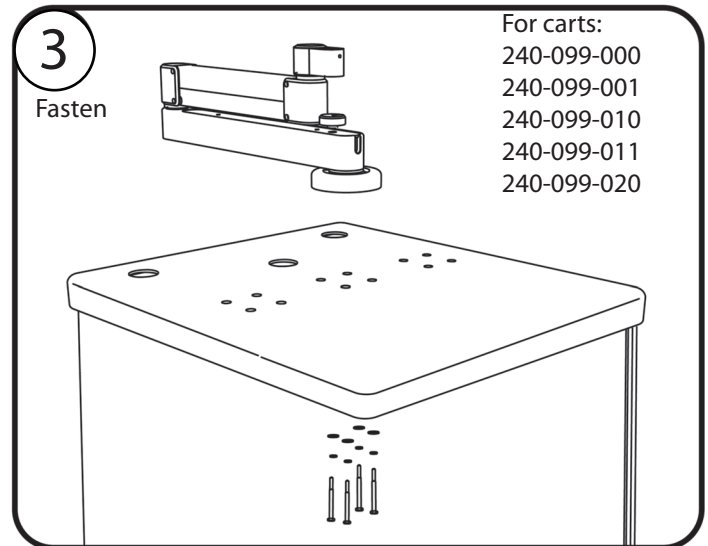5
Mount

3/32" Hex Wrench

After insertion, tighten tensioning screw until desired rotation resistance is obtained.

Phillips Head Screwdriver

Tighten Screw with keyed washer to retain tilter mechanism.

Operation

6
Rotational Lock

Rotate lock knob clockwise to lock, counterclockwise to unlock.

7 Monitor Hold Down

Warning! Monitor must be latched during transportation. Failure to properly secure monitor may result in equipment damage or injury to personnel.

Adjustments

- Should there be a need for adjustment, tighten or loosen any of the 7 adjustment points that are shown.
- Take care not to over tighten any of the adjustments.
- Over tightening will make moving the arm and monitor difficult and may cause premature wear of components.
- Periodically check all adjustments to insure proper operation.

WARNING!

- Failure to properly adjust arm could result in equipment damage or injury.

CAUTION!

- Do not over tighten tensioning screws. Damage to bushing may occur.
- Periodically ensure that the tensioning screw, Philips head screw and washer are suitably tightened and flush as directed in step 5.
- Ensure all adjustment points are operating at required tightness to prevent inadvertent movement.

Adjustment Points

Monitor pivot tension adjustment

Monitor tilt tension adjustment

Horizontal Arm Rotation Adjustment.
NOTE: Horizontal Arm cover plate must be removed to access the adjustment screw.

Upper arm pivot tension adjustment

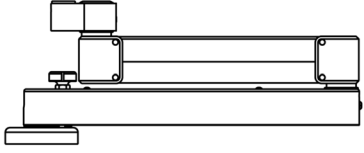
Upper arm articulation tension adjustments

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