Flat Panel Roll-Stand

Operating and Maintenance Manual

REF 240-099-109

240-099-110

English

Français

Deutsch

Italiano

Português

Español

Nederlands

Dansk

Soumi

Norsk

Svenska

Polski

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Română

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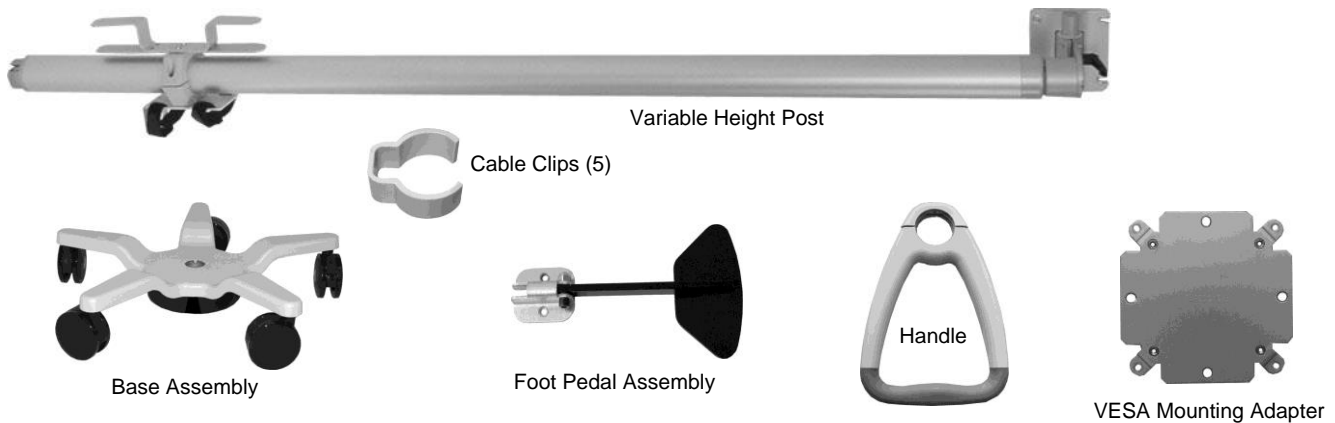
Flat Panel Roll Stand	EN-5
Socle roulant pour écran plat	FR-13
Flachbildschirm-Rollenständer	DE-21
Supporto con ruote per schermo piatto	IT-29
Suporte rolante para monitores de ecrã plano	PT-37
Soporte rodante para pantalla plana.....	ES-45
Rolstandaard voor flatpanel-scherm	NL-53
Rullestativ til fladskærm.....	NO-61
Näyttöpaneelin rullateline	SV-69
Rullestativ for flatskjermer.....	PL-77
Rullstativ för platt monitor	SV-85
Stojak na kółkach do monitora z płaskim ekranem.....	PL-93
Κυλιόμενη βάση για επίπεδες οθόνες	EL-101
Suport cu rotile pentru monitorul cu ecran plat	RO-109
平板显示器滚动支架.....	ZH-117
フラットパネル用ロールスタンド	JA-125
평면 패널 롤 스탠드	KO-133

Parts Reference

The following parts and hardware are included with this installation kit (see photos for parts, hardware not shown):

Item #	Description	Qty
1	Base Assembly (includes casters)	1
2	Post, Variable Height (with factory-mounted accessories)	1
3	Foot Pedal Assembly	1
4	1/4-20 x 3/4" Socket Head Cap Screw (SHCS)	2
5	Lock Washer, 1/4"	2
6	Handle	1
7	#10-32 x 9/16" Pan Head Machine Screw (PHMS)	2
8	VESA Mounting Adapter	1
9	M4 x 8mm PHMS	8
10	Cable Clips	5
11	3/16" Hex Key, T-Handle	1
12	9/64" Hex Key	1

Tools Required: Phillips screwdriver (not provided), 3/16" T-handle and 9/64" hex keys (provided).

CAUTION:

DO NOT MOVE ROLL STAND WHILE DISPLAY IS IN HIGHEST VERTICAL POSITION. LOWER DISPLAY BEFORE MOVING ROLL STAND.

DO NOT DISASSEMBLE ANY COMPONENT OF THE ROLL STAND ASSEMBLY.

DO NOT MOUNT A LOAD ON THE ROLL STAND THAT EXCEEDS 33 LB [14.9 KG].

TOTAL MASS WITH MAX. LOAD: 95 LB [43 KG].

NEVER APPLY FORCE AT TOP – ALWAYS PUSH NEAR MIDDLE.

Intended Use/Indications

The Flat Panel Roll Stand is intended to support a secondary monitor that is used in operating rooms. It is also intended to be used as a peripheral observation station that is used for viewing images.

The device is intended for the general surgical population.

Contraindications

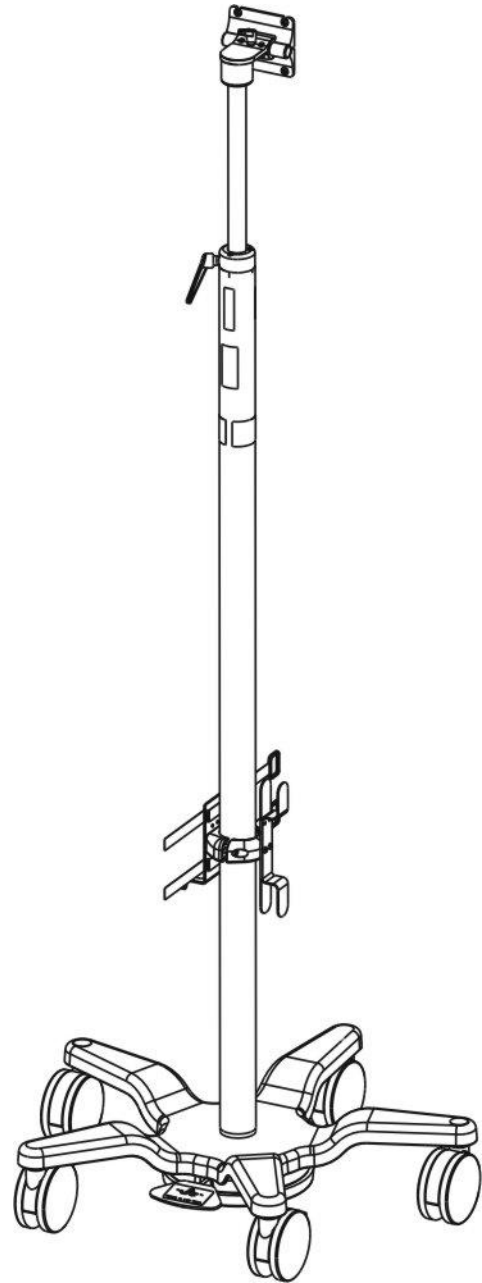
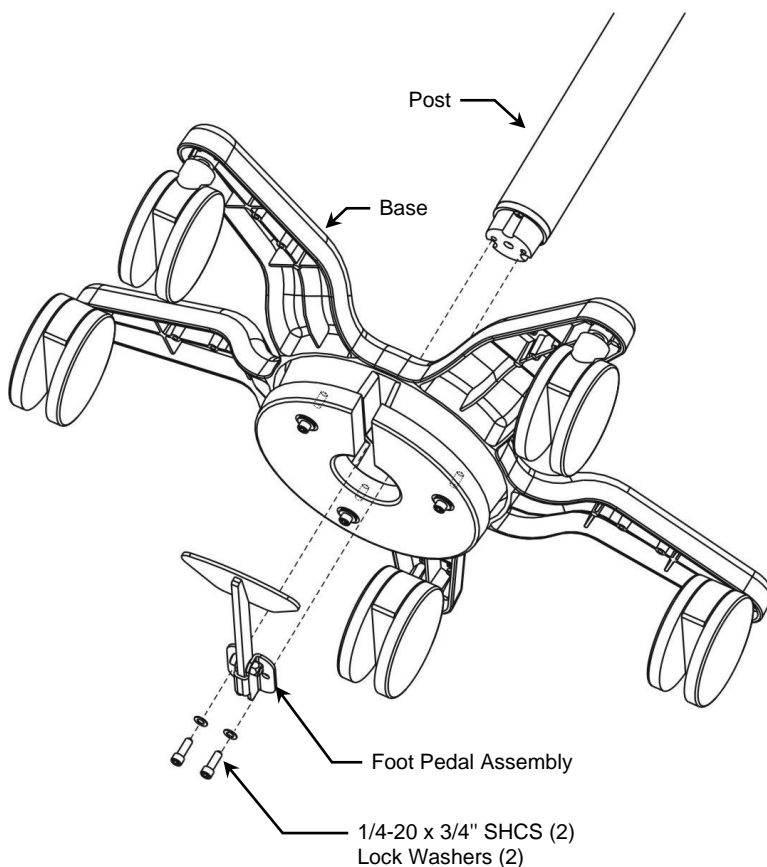
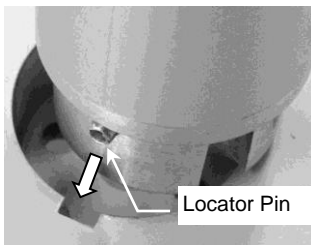
There are no known contraindications.

User Qualifications

The Carts and Stands devices should be operated only by qualified personnel such as surgeons and clinic support staff.

Assembling the Roll Stand


1. Using the 3/16" T-handle hex key provided, remove two (2) 1/4-20 x 3/4" SHCS and lock washers from end of Post.
2. Insert bottom end of Post in Base, ensuring locator pin is seated in slot provided (photo below left). Lay Roll Stand on its side for attachment of Foot Pedal Assembly.
3. Align the two (2) mounting holes in the Foot Pedal Assembly with the two (2) threaded mounting holes in the bottom of the Post. Using the 3/16" T-handle hex key, secure the Pedal Assembly to the Post with two (2) 1/4-20 x 3/4" SHCS. Set Roll Stand upright for attachment of Handle and mounting of flat panel display.

Attaching the Roll Stand Handle

Installation Note: A label is provided to ensure that the Handle is positioned at the correct height, directly above the Foot Pedal (photo below).

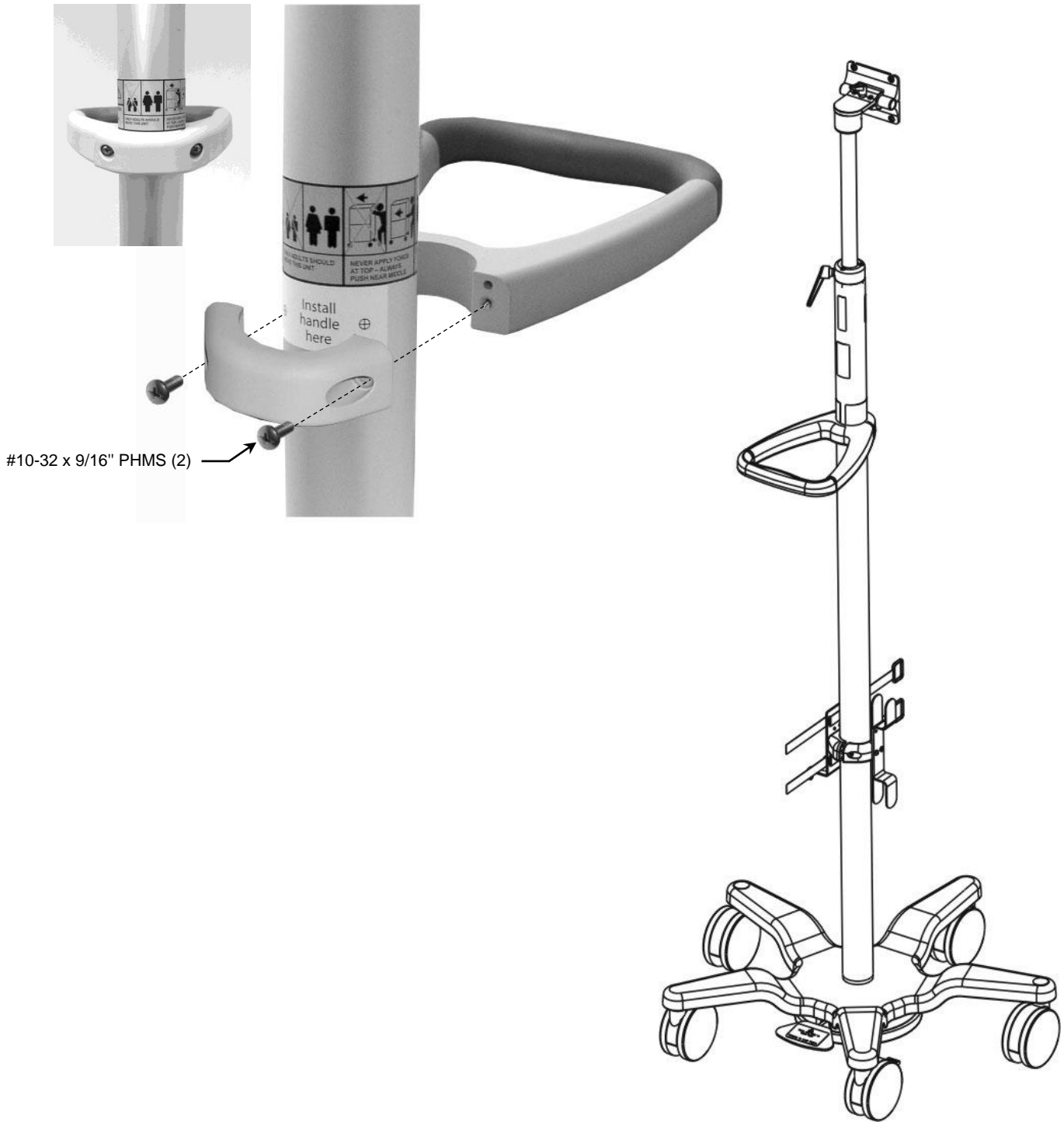1. Position Handle over bottom of label as shown below and fasten to Post with two (2) #10-32 x 9/16" PHMS. Alternately tighten screws until Handle does not rotate on Post.


Mounting the Flat Panel on the Roll Stand



WARNING: This Roll Stand is intended for use with flat panel displays that have mounting holes provided by the manufacturer and that have been evaluated for mounting. Do not modify the original enclosure of any display to facilitate mounting onto the Roll Stand. Do not mount a load on the roll stand that exceeds 33 lb [14.9 kg].

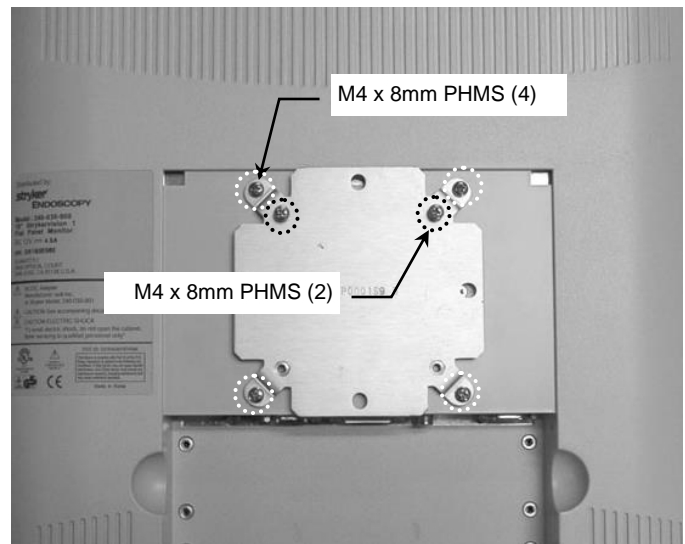1. Locate the 100 x 100mm mounting pattern on the rear of the flat panel (below left). Fasten the VESA Mounting Adapter to corresponding mounting holes with four (4) M4 x 8mm PHMS (below right).
2. Thread two (2) M4 x 8mm PHMS into the upper two (2) holes of the 75 x 75mm mounting pattern on the Mounting Adapter (below right). Leave 5 mm of thread exposed to allow screws to fit into slots on the VESA Mounting Plate.

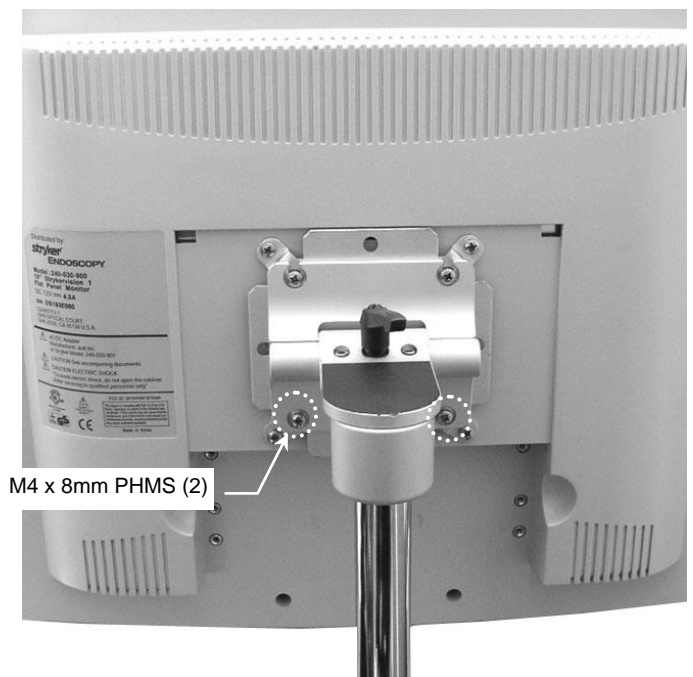
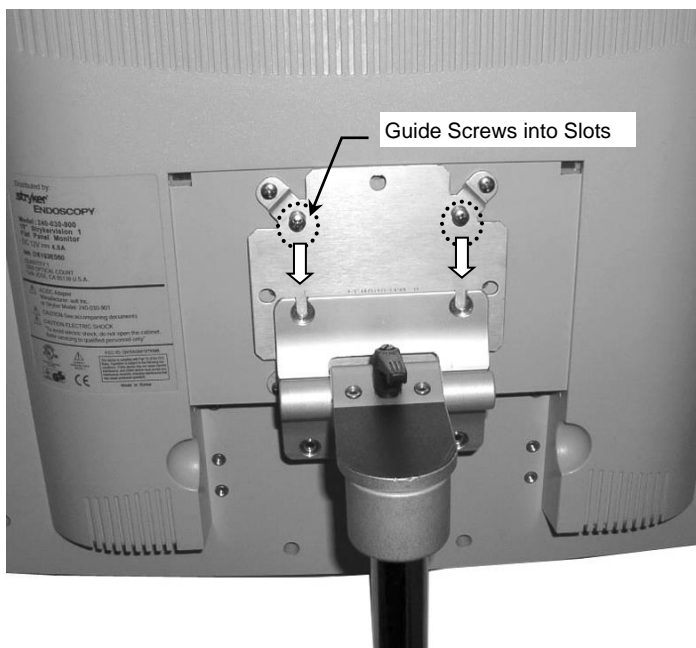
3. Guide two (2) upper screws into slots in the Mounting Plate at top of Roll Stand (below left). Do not fully tighten these screws until Step 4.
4. Thread two (2) M4 x 8mm PHMS into the lower two (2) mounting holes and tighten all four (4) mounting screws.



Mounting Power Supply on Roll Stand Post

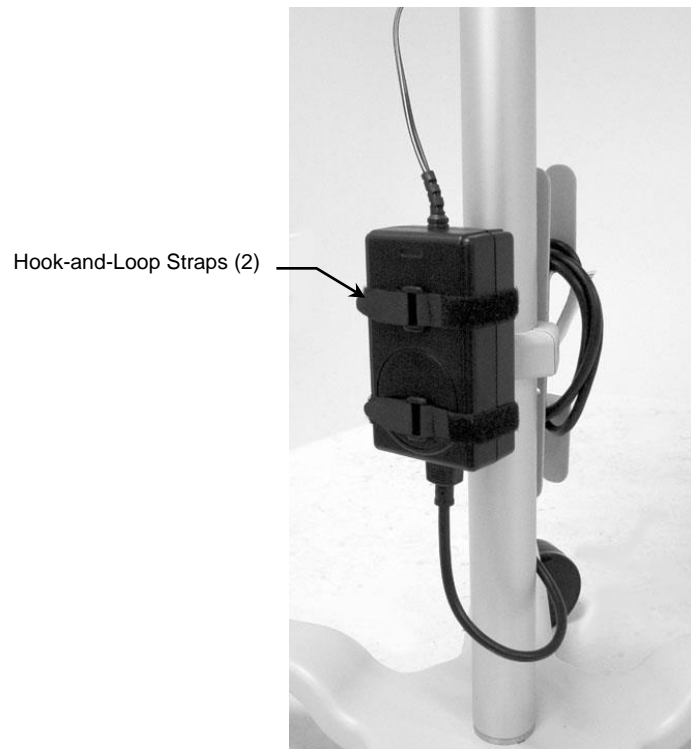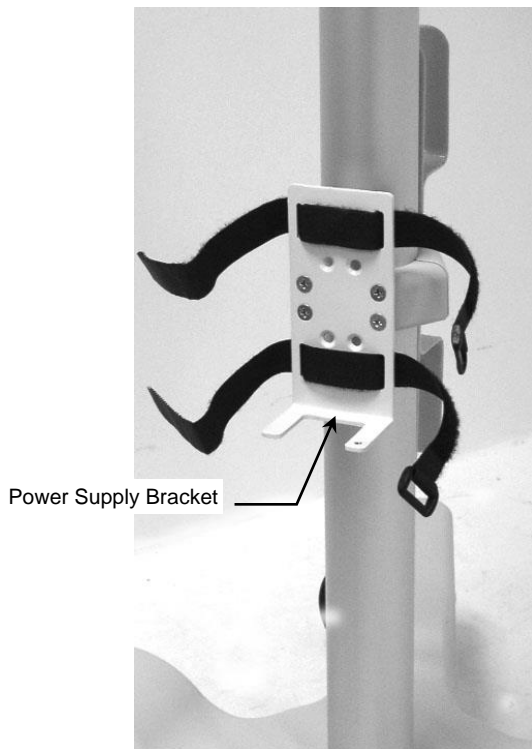1. Place power supply in Power Supply Bracket. Secure power supply to Bracket with two (2) hook-and-loop straps.

Cable Management

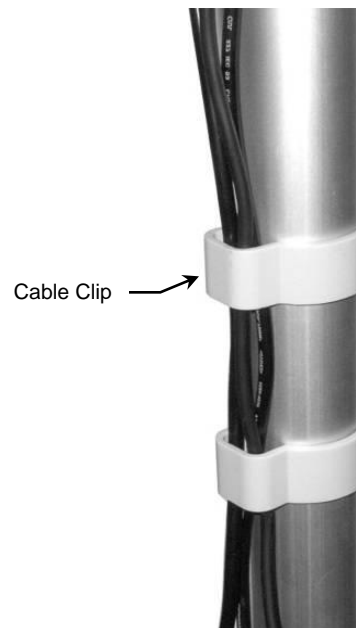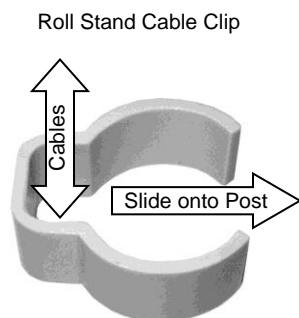Five (5) Cable Clips are provided for managing cables along Roll Stand Post. Place loose cables inside Clip and slide Clip onto Post as shown below.

Making Roll Stand Adjustments

 **CAUTION:** DO NOT MOVE ROLL STAND WHILE DISPLAY IS IN HIGHEST VERTICAL POSITION. LOWER DISPLAY BEFORE MOVING ROLL STAND.

Height Adjustment

CAUTION: The Roll Stand Post operates by pneumatic pressure. The Post will travel upward when the Foot Pedal is pressed. Ensure that nothing will obstruct upward travel before pressing the Foot Pedal.

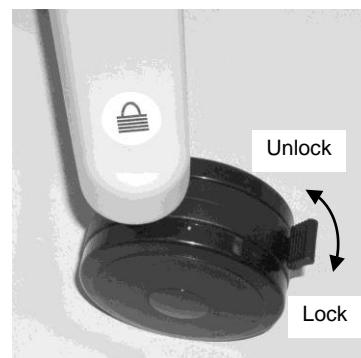
1. Press and hold the Foot Pedal to release the Post for positioning.
2. Move the mounted instrument to the desired height and release the Foot Pedal to lock the Post in position.

Locking the Casters

There are two (2) locking casters on the Roll Stand. Locking casters are identified by labels on the top surface of two (2) legs of the Roll Stand Base.

Press lever downward to lock a caster. Lift lever to unlock.

Swiveling the Flat Panel

Loosen the Swivel Adjustment Lever. Swivel flat panel to desired position. Tighten Swivel Adjustment Lever to lock flat panel in position.

Tilting the Flat Panel and Adjusting Tilt Tension

Tilting the Flat Panel

Grasp flat panel and tilt to desired position.

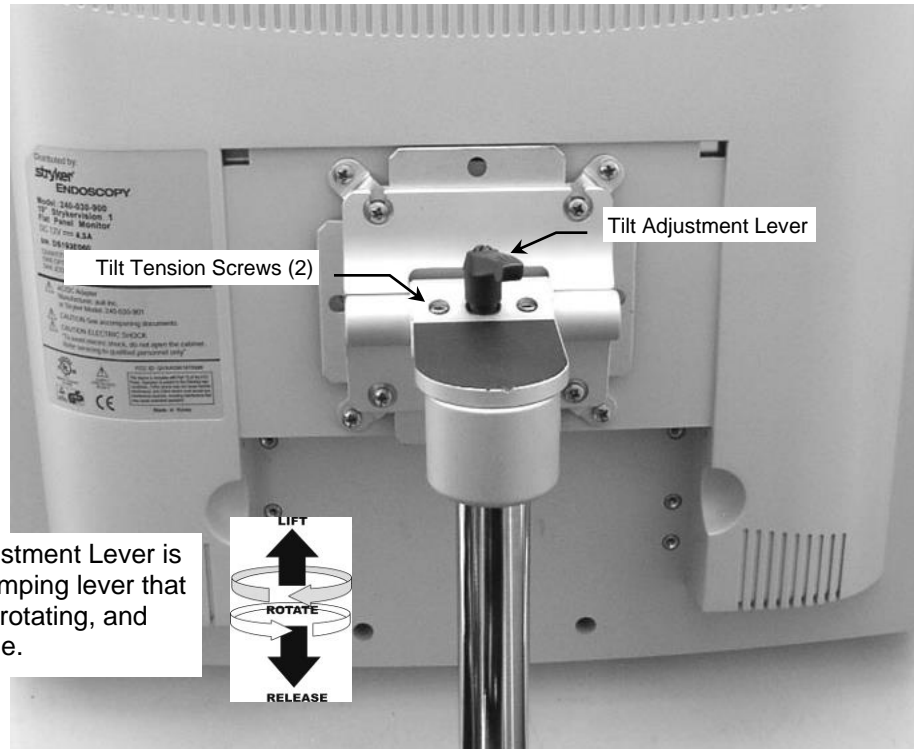
Note: If tilt position cannot be maintained, see *Increasing Tilt Tension* section. If instrument is difficult to tilt, see *Decreasing Tilt Tension* section.

Decreasing Tilt Tension

Using the 9/64" Hex Wrench, decrease tension by turning *both* tilt tension screws the same number of turns counter clockwise.

Increasing Tilt Tension

Using the 9/64" Hex Wrench, increase tension by turning *both* tilt tension screws the same number of turns clockwise.

Cleaning and Maintenance

Cleaning

- Wipe surfaces with soap and water or disinfectant.
- Dry with non-abrasive cloth or paper towel.
- To ensure ease of use, keep casters free of dirt, oil, and debris.

Maintenance

- Periodically check all hardware and re-tighten fasteners as required.
- Keep debris from wheel areas.
- Check roll stand for loose, damaged and worn components and service as required.

Expected Service Life

The device's service life is largely determined by wear, cleaning methods, and any damage resulting from use. To extend the time between device servicing, always follow the care and handling instructions in this user manual.

Inspect the Roll Stand prior to each use. If a condition relating to functionality or appearance (such as but not limited to the following) is observed or suspected, discontinue use immediately and contact your Stryker representative.

- Significant physical damage to the roll stand or casters (including but not limited to) damaged coatings, corrosion, abnormal noises, sharp edges, or damage preventing a secure mounting interface
- Inability to adjust the monitor tilt or to securely hold the desired tilt
- Inability to adjust the height of roll stand or securely hold the desired height
- Inability to freely move or apply the brake
- Looseness in the pole attached to the base
- Malfunctioning foot pedal

Disposal

In accordance with European Directive 2012/19/EU on Waste Electrical and Electronic Equipment (WEEE) as amended, product should be collected separately for recycling. Do not dispose of as unsorted municipal waste. Contact local distributor for disposal information. Ensure infected equipment is cleaned per the Cleaning and Maintenance section of this IFU prior to recycling.

Adverse Event Reporting

Any serious incident that has occurred in relation to this device should be reported to Stryker and, in the European Union, to the competent authority of the Member State in which the affected person resides.

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