POWEReam® Attachments

REF 4405-XXX-XXX Series

Instructions For Use

R_x ONLY

ENGLISH (EN)	SUOMI (FI)
ESPAÑOL (ES)	PORTUGUÊS (PT)
DEUTSCH (DE)	NORSK (NO)
FRANÇAIS (FR)	POLSKI (PL)
ITALIANO (IT)	ΕΛΛΗΝΙΚΑ (EL)
NEDERLANDS (NL)	日本語 (JA)
SVENSKA (SV)	中文 (ZH)
DANSK (DA)	한국어 (KO)

Introduction

This *Instructions For Use* manual is the most comprehensive source of information for the safe and effective use of your product. This manual may be used by in-service trainers, physicians, nurses, surgical technologists, and biomedical equipment technicians. Keep and consult this reference manual during the life of the product.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

For additional information, especially safety information, or in-service training, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

NOTE: The user and/or patient should report any serious product-related incident to both the manufacturer and the Competent Authority of the European Member State where the user and/or patient is established.

Trademarks not the property of Stryker Corporation are the property of their respective owners.

Indications For Use

See the instructions for use supplied with the handpiece.

Contraindications

See the instructions for use supplied with the handpiece.

For Use With

This section describes system components that must be used with the equipment described in this manual to create a safe and effective system.

DESCRIPTION	REF
Cordless Driver 4 Handpiece	4405-000-000

User/Patient Safety

WARNINGS:

- Before using any system component, or any component compatible with this system, read and understand the instructions. Pay particular attention to WARNING information. Become familiar with the system components prior to use.
- Only trained and experienced healthcare professionals should use this equipment.
- The healthcare professional performing any procedure is responsible for determining the appropriateness of this equipment and the specific technique used for each patient. Stryker, as a manufacturer, does not recommend surgical procedure or technique.
- Upon initial receipt and before each use, clean and sterilize the equipment as indicated. For processing instructions, see the care instructions manual supplied with the handpiece.
- Upon initial receipt and before each use, operate the equipment and inspect each component for damage. DO NOT use any equipment if damage is apparent. For inspection criteria, see the care instructions manual supplied with the handpiece.

- A wobbling attachment and/or accessory may cause bone or tissue damage or inaccurate wire or pin placement. If wobbling occurs, take corrective action as indicated in the *Troubleshooting* section.
- ALWAYS operate the equipment within the specified environmental condition values. See the *Specifications* section.

Accessories

WARNINGS:

- Use only Stryker-approved system components and accessories, unless otherwise specified.
- DO NOT modify any system component or accessory, unless otherwise specified.
- DO NOT reuse, reprocess, or repack a device that is intended for single use only. Failure to comply may lead to infection or cross-infection and result in patient and/or healthcare staff injury.

NOTE: For a complete list of accessories, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

Keyed Chuck


Description

POWEReam 1/4 inch Keyed Chuck (REF 4405-231-000)

Handles wires, pins, and drill bits up to 1/4 inch (6.4 mm) diameter. Utilizes low speed and high torque.

Length: 109 mm [4.30 inch]; Diameter: 34 mm [1.34 inch]; Mass: 0.35 kg [0.76 lb]

Features

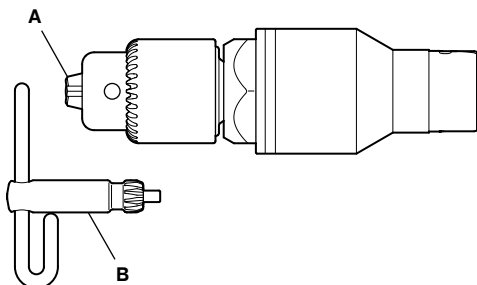A	Jaws
B	Chuck Key

Instructions

To Install an Accessory

1. See the instructions for use supplied with the handpiece to securely install the attachment into the handpiece.
2. Use the chuck key to open the jaws.
3. Insert the accessory through the center of the jaws.

WARNING: ALWAYS use the chuck key to manually close the jaws. Operating the handpiece to automatically close the jaws may result in injury to the user.

4. Use the chuck key to close the jaws and secure the accessory.
5. Visually inspect the accessory to make sure it is centered in the jaws.
6. Gently twist and tug the accessory to make sure it is securely installed in the chuck.

To Remove an Accessory

1. Use the chuck key to open the jaws.
2. Remove the accessory.


Reamer Attachments

Description

POWERream AO® (REF 4405-210-000)

Accepts Synthes tools, such as drill bits, automatic screwdrivers, and flexible reamers, with an appropriate snap-lock shank. Utilizes low speed and high torque.


Length: 115 mm [4.51 inch]; Diameter: 34 mm [1.34 inch]; Mass: 0.35 kg [0.79 lb]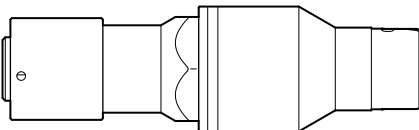

POWERream Hudson® (REF 4405-213-000)

Accepts tools, such as drill bits, automatic screwdrivers, and flexible reamers, with a Hudson fitting. Utilizes low speed and high torque.


Length: 108 mm [4.25 inch]; Diameter: 34 mm [1.34 inch]; Mass: 0.32 kg [0.69 lb]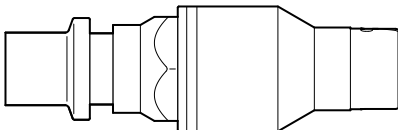

POWERream Hudson Modified Trinkle (REF 4405-235-000)

Accepts tools, such as drill bits, automatic screwdrivers, and flexible reamers, with a Hudson or modified Trinkle fitting. Utilizes low speed and high torque.


Length: 108 mm [4.23 inch]; Diameter: 34 mm [1.34 inch]; Mass: 0.30 kg [0.65 lb]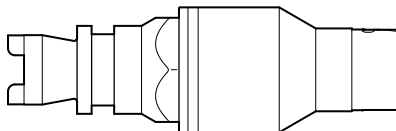

POWERream Trinkle (REF 4405-260-000)


Accepts tools, such as drill bits, automatic screwdrivers, and flexible reamers, with a standard Trinkle fitting. Utilizes low speed and high torque.

Length: 112 mm [4.42 inch]; Diameter: 34 mm [1.34 inch]; Mass: 0.30 kg [0.65 lb]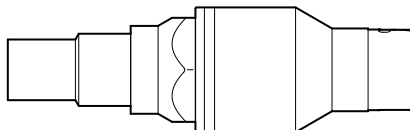

Features

NOTE: The POWEReam Hudson (REF 4405-213-000) is shown for reference.

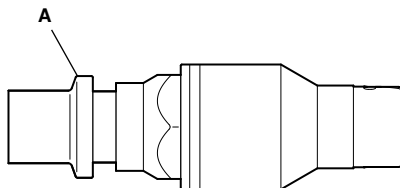A Retaining Collar

Instructions

To Install an Accessory

1. See the instructions for use supplied with the handpiece to securely install the attachment into the handpiece.
2. Slide back the retaining collar.
3. Insert the accessory.
4. Release the retaining collar.
5. Gently twist and tug the accessory to make sure it is securely installed in the attachment.

To Remove an Accessory

1. Slide back the retaining collar.
2. Remove the accessory.

Care Instructions

For processing instructions and disposal/recycle information, see the care instructions manual supplied with the handpiece.

Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	Lock
	General warning sign

Troubleshooting

WARNING: DO NOT disassemble or service this equipment, unless otherwise specified.

NOTE: For service, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

PROBLEM	CAUSE	ACTION
The handpiece operates but the accessory does not move.	The attachment is not fully installed in the handpiece.	Remove and install the attachment. Make sure the attachment is securely locked in the handpiece.
The attachment will not fit or cannot be secured in the handpiece.	The attachment and/or the distal end of the handpiece contains debris.	Clean the attachment and/or handpiece. See the care instructions manual supplied with the equipment.
	The attachment and/or handpiece is damaged.	Return the attachment and/or handpiece to Stryker for repair.
The attachment and/or cutting accessory is wobbling in the handpiece.	The accessory is the wrong size for the attachment.	Install a different accessory or attachment as required.
	The accessory extends too far from the distal end of the attachment or is not properly centered in the attachment.	Remove and properly install the accessory.
	The accessory is damaged.	Inspect the accessory for damage and replace the accessory as required.
	The attachment and/or handpiece is damaged.	Return the attachment and/or handpiece to Stryker for repair.

Specifications

WARNING: ALWAYS follow the recommended duty cycle to prevent the equipment from overheating. See the instructions for use supplied with the handpiece.

CAUTION: ALWAYS store the equipment within the specified environmental condition values throughout its useful life.

NOTE: When in use, the attachments described in this manual are applied parts. See the instructions for use supplied with the handpiece for information specific to applied parts.

Mode of Operation:	Non-continuous Operation
Duty Cycle:	For duty cycle information, see the instructions for use supplied with the handpiece.

Environmental Conditions:	Operation	Storage and Transportation
Temperature Limitation:		
Humidity Limitation:		
Atmospheric Pressure Limitation:		

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