stryker®

Cordless Driver 4/Sabo®2 Insert Tray

REF 4405-452-010


Cordless Driver 4/Sabo2 1HP Insert Tray

REF 4405-453-010

Instructions For Use

R_x ONLY

CE

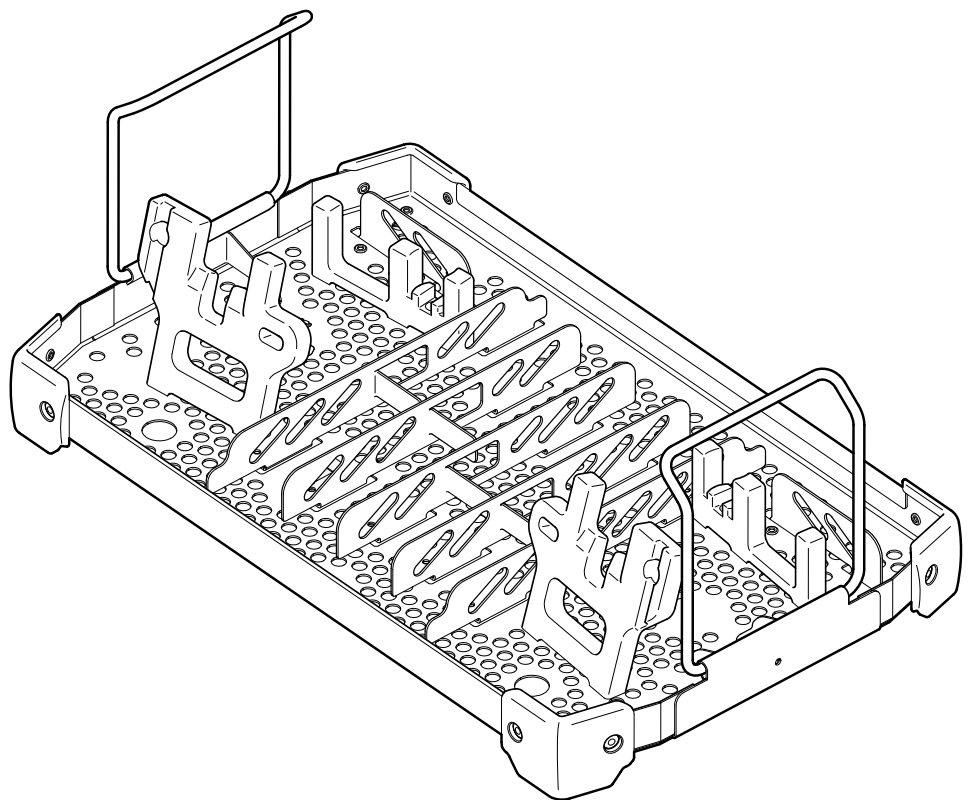ENGLISH (EN)
ESPAÑOL (ES)
DEUTSCH (DE)
FRANÇAIS (FR)
ITALIANO (IT)
NEDERLANDS (NL)
SVENSKA (SV)
DANSK (DA)
SUOMI (FI)
PORTUGUÊS (PT)
NORSK (NO)
POLSKI (PL)
ΕΛΛΗΝΙΚΑ (EL)
TÜRKÇE (TR)
РУССКИЙ (RU)
日本語 (JA)
中文 (ZH)
한국어 (KO)

Introduction

This instructions for use manual contains information intended to ensure the safe, effective and compliant use of your product. Keep and consult this reference manual during the life of the product.

This manual is intended for in-service trainers, biomedical equipment technicians, and central supply/sterile processing technicians.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

For additional information, including safety information, in-service training, or current literature, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

NOTE: The user and/or patient should report any serious product-related incident to both the manufacturer and the Competent Authority of the European Member State where the user and/or patient is established.

Indications For Use

The Stryker Sterilization Case is designed to contain an insert tray and protect reusable medical devices during steam sterilization, storage and transportation. The Sterilization Case must be wrapped in a validated sterilization wrap prior to sterilization to maintain sterility. The Stryker Insert Tray is designed to provide a convenient holder for reusable medical devices during automated washing and steam sterilization. This Insert Tray may also be placed in a rigid container that does not require a validated wrap.

Contraindications

None known.

For Use With

WARNING: Use only Stryker-approved equipment, unless otherwise specified. DO NOT modify any equipment without the authorization of the manufacturer.

Each insert tray is intended for use with a specific sterilization case or sterilization container model. Cordless Driver 4/Sabo2 Insert Tray REF 4405-452-010 is intended for use with the following:

DESCRIPTION		REF
3/4 Size x 8" (200 mm) Case		7102-450-040
3/4 Size x 8" (200 mm) Perforated Bottom Container	Lid	7102-550-030 (Aesculap Model JK789)
	Base	7102-550-020 (Aesculap Model JN744)
3/4 Size x 8" (200 mm) Solid Bottom Container	Lid	7102-550-030 (Aesculap Model JK789)
	Base	7102-650-020 (Aesculap Model JK744)

Cordless Driver 4/Sabo2 1HP Insert Tray REF 4405-453-010 is intended for use with the following:

DESCRIPTION		REF
1/2 Size x 6" (150 mm) Case		7102-453-040
1/2 Size x 6" (150 mm) Perforated Bottom Container	Lid	7102-553-030 (Aesculap Model JK389)
	Base	7102-553-020 (Aesculap Model JN342)
1/2 Size x 6" (150 mm) Solid Bottom Container	Lid	7102-553-030 (Aesculap Model JK389)
	Base	7102-653-020 (Aesculap Model JK342)

Insert Trays REF 4405-452-010 and REF 4405-453-010 are intended for use with the following Stryker reusable medical devices:

DESCRIPTION	REF	INSERT TRAY REF	
System 8 Cordless Driver Handpiece	4505-000-000	4405-452-010 4405-453-010	
Cordless Driver 4 Handpiece	4405-000-000		
Cordless Driver 3 Handpiece	4300-000-000		
System 8 Sabo Sagittal Saw	4508-000-000		
Sabo2 Sagittal Saw	4408-000-000		
Sabo Sagittal Saw	4300-034-000		
Cordless Driver Attachments	4505-XXX-XXX series 4500-XXX-XXX series 4100-XXX-XXX series		
POWERream® Attachments	4405-XXX-XXX series		
System 8 Reciprocating Saw	8206-000-000		4405-452-010 only
System 8 Sagittal Saw	8208-000-000		
System 8 Precision® Saw	8209-000-000		
System 7 Reciprocating Saw	7206-000-000		
System 7 Sagittal Saw	7208-000-000		
System 7 V-Notch Sagittal Saw	7208-008-000		
System 7 Stryker Precision Oscillating Tip Saw	7209-000-000		
System 7 Stryker Precision Saw	7209-009-000		

WARNING: DO NOT use the insert trays to process the following right angle drive attachments:

DESCRIPTION	REF
Right Angle Drive – AO Large Reamer	4100-310-000
Right Angle Drive – Modified Trinkle Reamer	4100-335-000
Right Angle Drive – Radiolucent	4100-355-000

Definitions


The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	General warning sign

Features


NOTE: The Cordless Driver 4/Sabo2 Insert Tray (REF 4405-452-010) and 3/4 Size x 8" (200 mm) Case (REF 7102-450-040) are shown for reference.

Insert Tray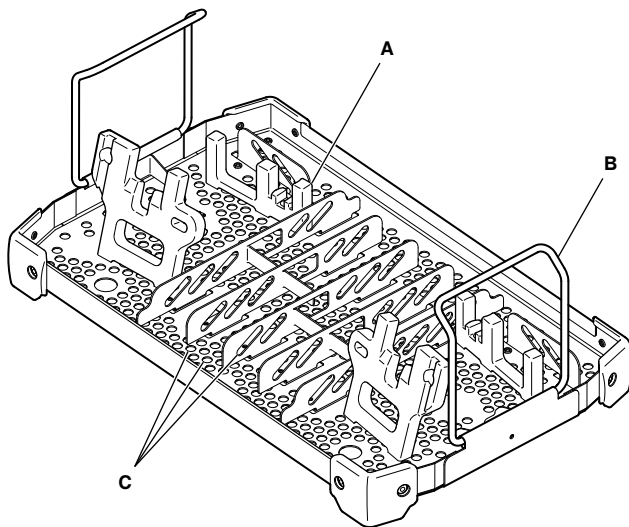

A	Bracket
B	Handle
C	Perforations (Holes)


Sterilization Case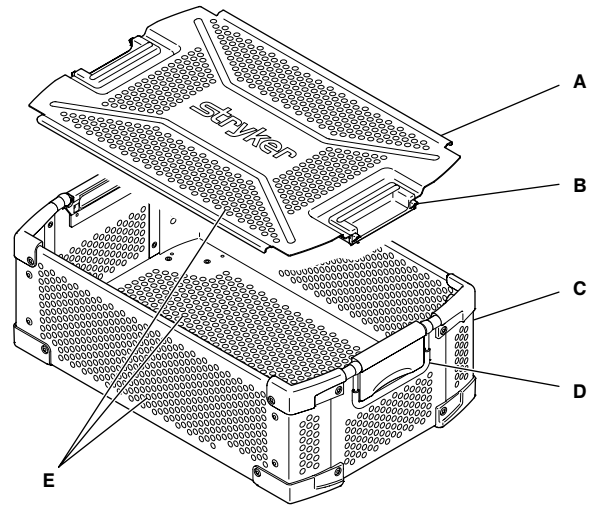

A	Lid
B	Latch
C	Base
D	Handle
E	Perforations (Holes)

Sterilization Container

NOTE: Insert trays may also be used with specific, perforated bottom or solid bottom sterilization containers. See the *For Use With* section for appropriate part numbers.

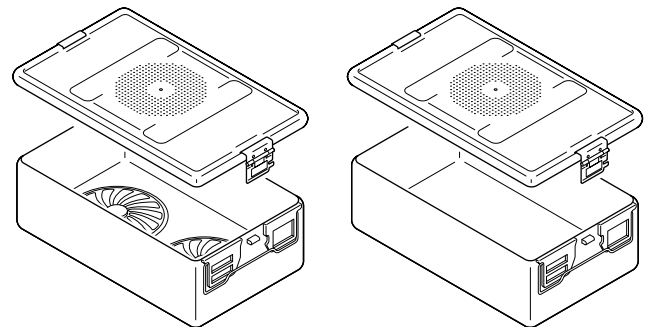Perforated Bottom

Solid Bottom

Safety Directives

WARNINGS:

- Before using this equipment, or any component compatible with this equipment, read and understand the instructions for use. Pay particular attention to safety information. Become familiar with the equipment before use.
 - Only individuals trained and experienced in the processing of reusable medical devices should process this equipment.
 - DO NOT reuse, reprocess, or repackage a device that is intended for single use only.
 - A single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing.
 - Design features may make cleaning difficult.
 - Reuse may create a contamination risk and compromise structural integrity resulting in operational failure or fragmentation during use.
 - Critical product information may be lost during repackaging.
- Failure to comply may lead to infection or cross infection and result in patient and/or healthcare staff injury.

Care Instructions

For general processing instructions, see the care instructions manual supplied with your device.

Maximum Quantity of Devices

WARNING: DO NOT exceed the specified maximum quantity of devices in an insert tray. The maximum quantity of devices is based on Stryker validation of this equipment. Exceeding the maximum quantity of devices may prevent proper sterilization of the equipment and/or result in excessive weight. Always follow the current local recommendations and/or regulations governing weights and weight limits.

Cordless Driver 4/Sabo2 Insert Tray (REF 4405-452-010)

DEVICE TYPE	MAXIMUM QUANTITY
Handpiece	2
Attachment	12
Accessory (Chuck Key)	2
Total of All Devices	16

Cordless Driver 4/Sabo2 1HP Insert Tray (REF 4405-453-010)

DEVICE TYPE	MAXIMUM QUANTITY
Handpiece	1
Attachment	8
Accessory (Chuck Key)	2
Total of All Devices	11

Recommended Placement of Devices


WARNINGS:

- ALWAYS make sure the equipment is clean and dry before sterilization.
- ALWAYS make sure handpieces are oriented with the distal end down.
- Large attachments may not fit in the smaller insert tray compartments. ALWAYS load attachments into tray compartments that are large enough to allow each attachment to lie in a flat, horizontal position.


NOTES:

- See the *Maximum Quantity of Devices* section.
- The illustrated placement of devices is a recommendation only. Other placement combinations are acceptable.

Cordless Driver 4/Sabo2 Insert Tray (REF 4405-452-010)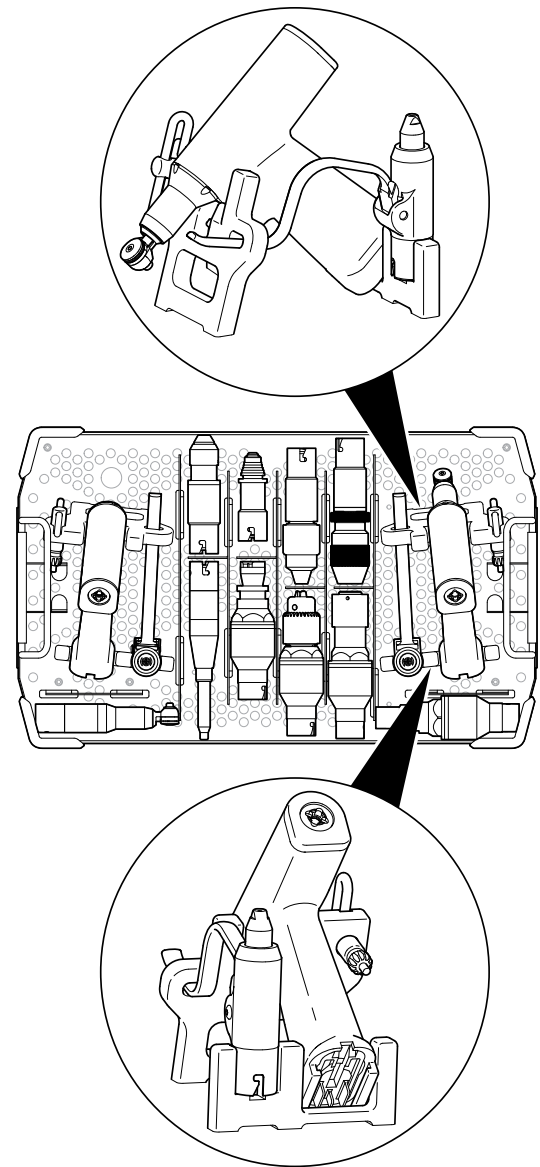


NOTE: For CD4/Sabo2 Insert Tray REF 4405-452-010 only, brackets may be used to hold fully assembled, sterile handpieces in the operating room prior to surgery.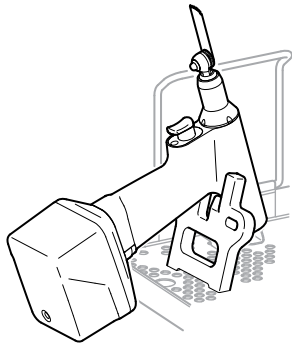

Specifications

Model:	Cordless Driver 4/Sabo2 Insert Tray (REF 4405-452-010)	Cordless Driver 4/Sabo2 1HP Insert Tray (REF 4405-453-010)
Dimensions:	159 mm [6.3 inch] height 406 mm [16.0 inch] length 251 mm [9.9 inch] width	113 mm [4.4 inch] height 247 mm [9.7 inch] length 251 mm [9.9 inch] width
Mass:	1.30 kg [2.87 lb]	0.96 kg [2.12 lb]
Material:	Stainless Steel Polypropylene Polyaryletheretherketone (PEEK)	

Cordless Driver 4/Sabo2 1HP Insert Tray (REF 4405-453-010)

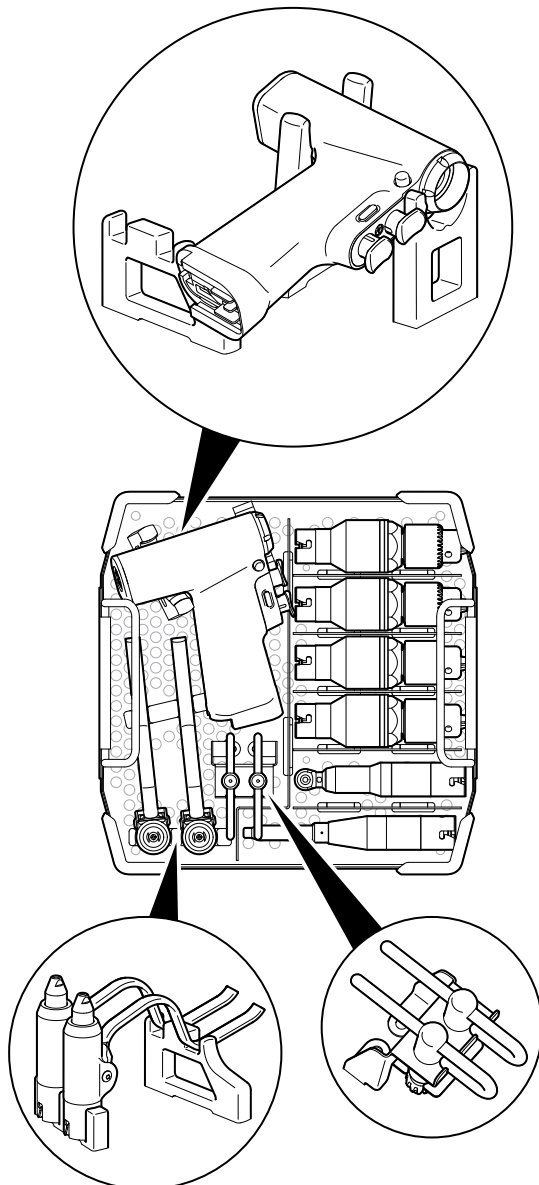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