<p><i>Note: This portion of the document should not appear on produced Labels or IFUs.</i></p> <p>Stryker Instruments (269) 323-7700 (800) 253-3210</p>	<p>Dimensions: 8.5 inch (width) x 11 inch Booklet</p>	<p>Color/Material/Finish: Black Graphics on White Background 20# Bond or Equivalent</p>	<p>Label Stock: N/A</p>	
			<p>Description/Type: Instructions For Use</p>	
			<p>Part Number: 6102-450-701</p>	<p>Rev. F</p>

stryker®


System 6 Sterilization Cases

REF 6102-450-000
6102-452-000
6102-454-000

Instructions For Use

R_x ONLY

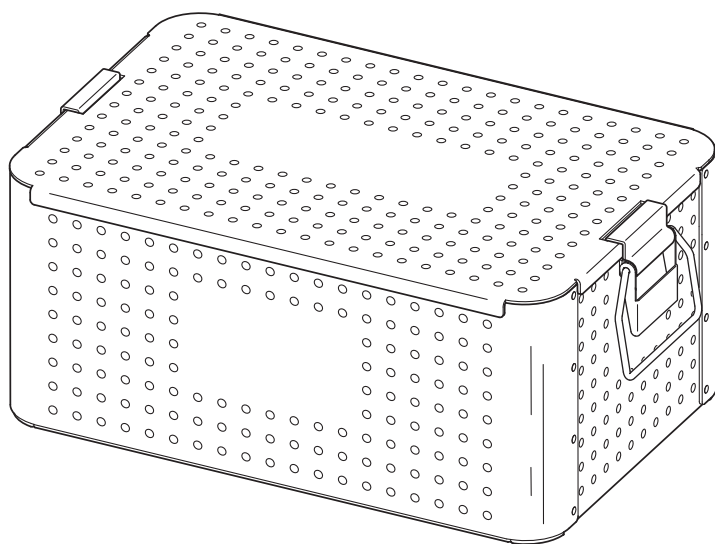CE

ENGLISH (EN)
ESPAÑOL (ES)
DEUTSCH (DE)
FRANÇAIS (FR)
ITALIANO (IT)
NEDERLANDS (NL)
SVENSKA (SV)
DANSK (DA)
SUOMI (FI)
PORTUGUÊS (PT)
NORSK (NO)
POLSKI (PL)
ΕΛΛΗΝΙΚΑ (EL)
日本語 (JA)
中文 (ZH)
한국어 (KO)

Introduction

This instructions for use manual contains information intended to ensure the safe, effective and compliant use of your product. Keep and consult this reference manual during the life of the product.

This manual is intended for in-service trainers, biomedical equipment technicians, and central supply/sterile processing technicians.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

For additional information, including safety information, in-service training, or current literature, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

Indications For Use

The Stryker Sterilization Case is designed to contain an insert tray and protect reusable medical devices during steam sterilization, storage and transportation. The Sterilization Case must be wrapped in a validated sterilization wrap prior to sterilization to maintain sterility. The Stryker Insert Tray is designed to provide a convenient holder for reusable medical devices during automated washing and steam sterilization. This Insert Tray may also be placed in a rigid container that does not require a validated wrap.

Contraindications

None known.

Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	General warning sign

For Use With

The tray is intended for use with Stryker System 6 handpieces and attachments. The tray will accommodate a variety of handpiece and attachment combinations.

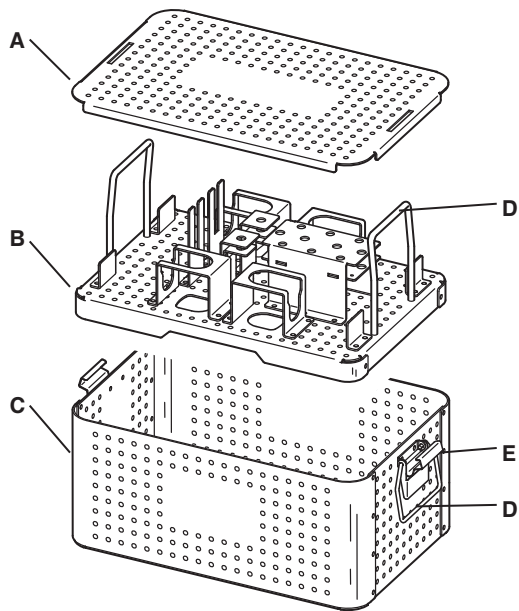
Safety Directives

WARNINGS:

- Upon initial receipt and before each use, inspect each component for damage. DO NOT use any equipment if damage is apparent.
- Before using this equipment, or any component compatible with this equipment, read and understand the instructions for use. Pay particular attention to safety information. Become familiar with the equipment before use.
- Only individuals trained and experienced in the processing of reusable medical devices should process this equipment.
- DO NOT reuse, reprocess, or repack a device that is intended for single use only.
 - A single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing.
 - Design features may make cleaning difficult.
 - Reuse may create a contamination risk and compromise structural integrity resulting in operational failure or fragmentation during use.
 - Critical product information may be lost during repackaging. Failure to comply may lead to infection or cross infection and result in patient and/or healthcare staff injury.

Features


A	Lid
B	Insert Tray
C	Base
D	Handles (four)
E	Latches (two)

NOTE: Sterilization Case REF 6102-450-000 is shown for reference.

To Remove the Lid from the Base

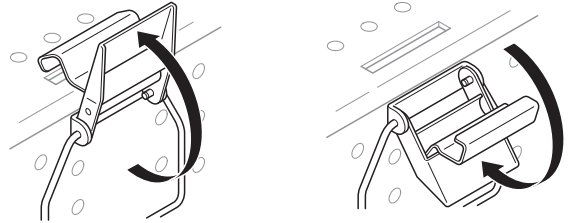
1. Flip the latches up and away from the tray base.

2. Remove the lid from the base.

To Secure the Lid to the Base

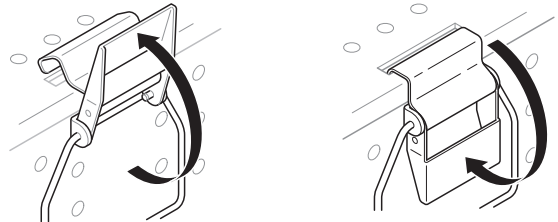1. Place the insert tray inside of the base.
2. Place the lid on the base.
3. Flip the latches up to engage the lid slot.

4. Press the latches down and toward the base body to make sure the latches lock the lid securely to the base.

Maximum Quantity of Devices

WARNING: DO NOT exceed the specified maximum quantity of devices in an insert tray. The maximum quantity of devices is based on Stryker validation of this equipment. Exceeding the maximum quantity of devices may prevent proper sterilization of the equipment and/or result in excessive weight. Always follow the current local recommendations and/or regulations governing weights and weight limits.

Sterilization Case, Large (Four Handpiece) (REF 6102-450-000)

DEVICE TYPE	MAXIMUM QUANTITY
Handpieces	4
Attachments	12
Total of All Devices	16

Mini Sterilization Case (Two Handpiece) (REF 6102-452-000)

DEVICE TYPE	MAXIMUM QUANTITY
Handpieces	2
Attachments	12
Total of All Devices	14

Sterilization Case (Three Handpiece) (REF 6102-454-000)

DEVICE TYPE	MAXIMUM QUANTITY
Handpieces	3
Attachments	10
Total of All Devices	13

Recommended Placement of Devices

WARNINGS:

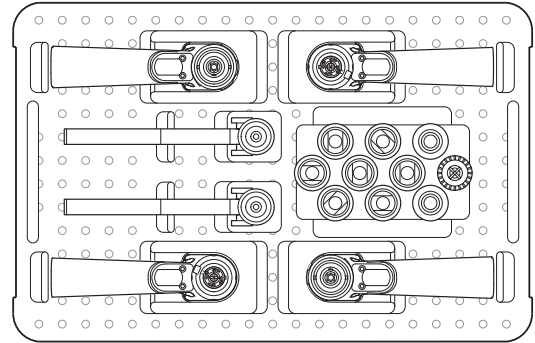
- ALWAYS make sure the equipment is clean and dry before sterilization.
- ALWAYS make sure handpiece and attachment lumens remain at a vertical angle to allow water to sufficiently drain.

NOTES:

- See the *Maximum Quantity of Devices* section.
- The illustrated placement of devices is a recommendation only. Other placement combinations are acceptable.


Sterilization Case, Large (Four Handpiece) (REF 6102-450-000)

Place the handpieces and attachments into the insert tray as shown.


Mini Sterilization Case (Two Handpiece) (REF 6102-452-000)

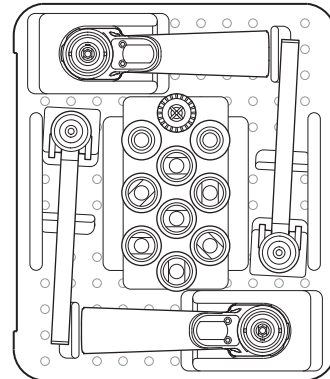Place the handpieces and attachments into the insert tray as shown.

Sterilization Case (Three Handpiece) (REF 6102-454-000)

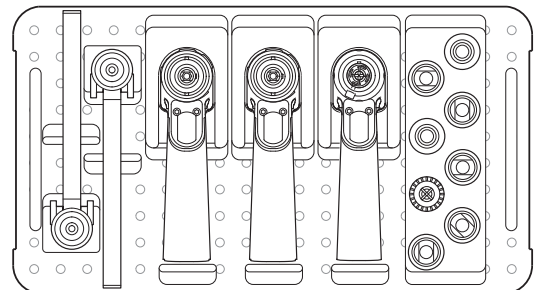Place the handpieces and attachments into the insert tray as shown.

Care Instructions

For general processing instructions, see the care instructions manual supplied with your device.

Specifications

Model: System 6 Sterilization Case, Large (Four-Handpiece) (REF 6102-450-000)

Dimensions: 19.3 cm [7.6 inch] height
28.7 cm [11.3 inch] width
43.7 cm [17.2 inch] length

Mass: 6.8 kg [15 lb]

Material: Case: Aluminum
Handles, Latches, Rivets: Stainless Steel

Model: System 6 Mini Sterilization Case (Two-Handpiece) (REF 6102-452-000)

Dimensions: 19.1 cm [7.5 inch] height
24.9 cm [9.8 inch] width
28.7 cm [11.3 inch] length

Mass: 4.5 kg [10 lb]

Material: Case: Aluminum
Handles, Latches, Rivets: Stainless Steel

Model: System 6 Sterilization Case (Three-Handpiece) (REF 6102-454-000)

Dimensions: 18.8 cm [7.4 inch] height
21.6 cm [8.5 inch] width
38.6 cm [15.2 inch] length

Mass: 7.3 kg [16 lb]

Material: Case: Aluminum
Handles, Latches, Rivets: Stainless Steel

NOTE: Specifications are approximate and may vary from unit to unit.

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