stryker®

System 6™ Non-sterile Battery

REF 6126-110-000

Instructions For Use

R_x ONLY

CE

ENGLISH (EN)

ESPAÑOL (ES)

DEUTSCH (DE)

FRANÇAIS (FR)

ITALIANO (IT)

NEDERLANDS (NL)

SVENSKA (SV)

DANSK (DA)

SUOMI (FI)

PORTUGUÊS (PT)

NORSK (NO)

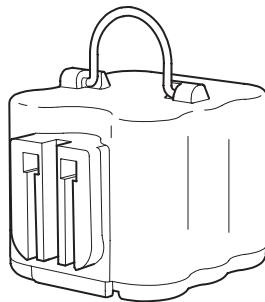
POLSKI (PL)

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Introduction

This *Instructions For Use* manual is the most comprehensive source of information for the safe and effective use of your product. This manual may be used by in-service trainers, physicians, nurses, surgical technologists, and biomedical equipment technicians. Keep and consult this reference manual during the life of the product.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

If additional information or in-service training is required, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

NOTE: The user and/or patient should report any serious product-related incident to both the manufacturer and the Competent Authority of the European Member State where the user and/or patient is established.

Indications For Use

The Stryker Non-sterile Battery (REF 6126-110-000) is used to power Stryker System 6 handpieces. DO NOT sterilize these batteries. After a Non-sterile Battery is charged, it is placed in a sterilized Aseptic Housing using a Transfer Shield. The housing containing the battery is then installed into a Stryker handpiece. See the *For Use With* section.

Charge these battery packs on the System 6 Battery Charger (REF 6110-120-000) or the Universal Battery Charger (REF 7110-120-000). The charger must be configured with the System 6 Charger Module (REF 6110-625-000) only.

Contraindications

None known.

User/Patient Safety

WARNINGS:

- Only trained and experienced healthcare professionals should use this equipment.
- Before using any system component or any component compatible with this system, read and understand the instructions. Pay particular attention to WARNING information. Become familiar with the system components prior to use.
- Upon initial receipt and before each use, inspect each component for damage. Damage may include, but is not limited to, bent contacts and cracks in the housing. DO NOT charge or use if damage is apparent.
- DO NOT use this equipment in areas in which flammable anesthetics or flammable agents are mixed with air, oxygen or nitrous oxide.
- DO NOT modify any system component or accessory.
- DO NOT sterilize.

For Use With

Stryker Non-sterile Batteries are intended for use with the following Stryker Aseptic Battery Kits:

Non-sterile Battery	Aseptic Battery Kit*
REF 6126-110-000	REF 6126-000-000

*Includes Non-sterile Battery (REF 6126-110-000), Transfer Shield (REF 6126-130-000), and Aseptic Housing (REF 6126-120-000).

Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	Refer to instruction manual/booklet
	General warning sign
DO NOT STERILIZE	Do not sterilize

Instructions

CAUTION: ALWAYS remove the non-sterile battery from the aseptic housing before charging the battery.

NOTES:

- Recharge batteries before each surgery, even if they have not been used, to ensure the maximum running time.
- The charge cycle takes approximately 5 to 90 minutes.
- The expected run time for each battery is 2.5 to 15 minutes depending on demand procedure utilization.
- Only use batteries that have been charged within the last 48 hours.
- Store batteries on the charger where they will be maintained fully charged. DO NOT store batteries in handpieces where they will discharge. Discharge will occur even though the handpiece is not running.
- Charge the batteries at normal room temperature, approximately 64 to 75 °F (18 to 24 °C).
- Maintenance documentation for this equipment is available upon request to Stryker-authorized service personnel only.
- To properly install the battery into a handpiece, read and understand the appropriate Stryker handpiece instructions for use.
- To properly charge the battery, read and understand the instructions for use provided with the battery charger.
- To install an aseptic battery into an aseptic housing (REF 6126-120-000), see the System 6 Aseptic Battery Kit (REF 6126-000-000) instructions for use.

Cleaning

 **WARNING:** DO NOT immerse or sterilize this battery.

1. Remove the non-sterile battery from the aseptic housing.
2. Inspect the battery housing for cracks or damage. DO NOT use the battery if damage is apparent.

CAUTION: DO NOT rinse, spray, or wipe the battery with a saturated cloth.

3. Wipe the non-sterile battery with a clean, dry cloth.

Storage and Handling

CAUTION: ALWAYS store the equipment within the specified environmental condition values throughout its useful life. See the Specifications section.

To ensure the longevity, performance and safety of this equipment, use of the original packaging material is recommended when storing or transporting this equipment.

Disposal/Recycle

 **WARNING:** ALWAYS follow the current local recommendations and/or regulations governing environmental protection and the risks associated with recycling or disposing of the equipment at the end of its useful life.

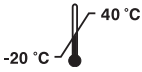
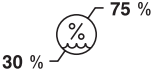
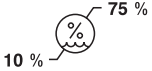
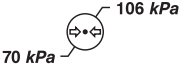
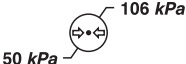
Ni-MH

To comply with European Community Batteries Directive 2006/66/EC, this device has been designed for safe removal of the batteries by the end user using the instructions provided herein. Infected units and batteries should be decontaminated before they are sent for recycling.

Specifications

NOTE: Specifications are approximate and may vary between devices.

Model:	System 6 Non-sterile Battery (REF 6126-110-000)
Dimensions:	53 mm [2.1 inch] 53 mm [2.1 inch] 66 mm [2.6 inch]
Mass:	0.36 kg [0.79 lb]
Electrical:	2A charge 9.6 V --- (Direct Current) output
Mode of Operation:	See the recommended duty cycle information supplied with each handpiece.
Ingress Protection:	IPX0 Ordinary Equipment

Environmental Conditions:	Operation	Storage and Transportation
Temperature Limitation:		
Humidity Limitation:		
Atmospheric Pressure Limitation:		

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