


stryker®

# Electric System 6™ Handpiece Cord

REF 6292-004-000

Instructions For Use

R<sub>x</sub> ONLY  
CE

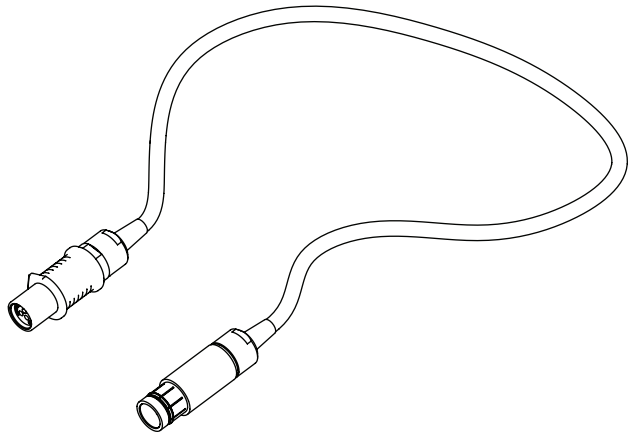

ENGLISH (EN)

## Introduction

This *Instructions For Use* manual is the most comprehensive source of information for the safe and effective use of your product. This manual may be used by in-service trainers, physicians, nurses, surgical technologists, and biomedical equipment technicians. Keep and consult this reference manual during the life of the product.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

For additional information, especially safety information, or in-service training, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

**NOTE:** The user and/or patient should report any serious product-related incident to both the manufacturer and the Competent Authority of the European Member State where the user and/or patient is established.

## Indications For Use

See the instructions for use supplied with the Stryker Electric System 6 handpiece.

## Contraindications

None known.

## For Use With

The Stryker Electric System 6 Heavy Duty Handpiece Cord is used with the Consolidated Operating Room Equipment (CORE™) System consoles and a variety of Stryker Electric System 6 handpieces. The cord provides the electrical connection between the handpiece and the console.

## User/Patient Safety



### WARNINGS:

- Before using any system component or any component compatible with this system, read and understand the instructions. Pay particular attention to WARNING information. Become familiar with the system components prior to use.
- Only trained and experienced healthcare professionals should use this equipment.

- Upon initial receipt and before each use, operate the equipment and inspect each component for damage. DO NOT use any component if damage is apparent. See the care instructions manual supplied with the handpiece.
- Upon initial receipt and before each use, clean and sterilize the equipment as indicated. See the care instructions manual supplied with the handpiece.
- DO NOT modify any system component or accessory.

## Definitions

The symbols located on the equipment and/or labeling are defined in this section and/or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	General warning sign



## Instructions

**CAUTION:** The handpiece cord has push/pull type connectors. DO NOT apply a twisting motion to insert or remove the cord's connector. Failure to comply may result in product damage.

## To Install Cord

1. Align the marks on the black connector with the marks on the handpiece.
2. Insert the cord's black connector into the handpiece until the connector latch engages. An audible click should be heard. Gently tug on the cord to ensure that the cord is secure.
3. Connect the other end of the handpiece cord to the console.

## To Remove Cord

1. Press the cord latch and pull the handpiece cord out of the handpiece.
2. Grasp the ribbed portion of the cord connector and pull the cord out of the console.

## Care Instructions

For processing instructions and disposal/recycle information, see the care instructions manual supplied with the handpiece.

## Specifications

<b>Model:</b>	Electric System 6 Handpiece Cord (REF 6292-004-000)
<b>Dimensions:</b>	12 ft [3.6 m] length
<b>Environmental Conditions:</b>	See the instructions for use supplied with the handpiece.

ES/DE/FR/IT/NL 6292-001-710  
JA/ZH/KO 6292-001-720  
SV/DA/FI/PT/NO 6292-001-730  
PL/EL 6292-001-750



**Stryker Instruments**  
1941 Stryker Way  
Portage, Michigan  
(USA) 49002  
1-269-323-7700  
1-800-253-3210

**stryker**<sup>®</sup>