



Heavy Duty Rotary Handpiece Attachments

REF 6203-XXX-XXX Series
7203-XXX-XXX Series

Instructions For Use

R_x ONLY



Contents

Introduction	3	Instructions	11
Indications For Use	3	To Install an Accessory.	11
Contraindications	3	To Remove an Accessory	11
For Use With	3	Locking Keyless Chuck	11
Safety Directives	4	Drill and Reamer Attachments	12
Accessories	5	Description	12
Definitions	5	AO Small Drill	12
Wire/Pin Collets	6	Hudson	12
Description	6	Hudson/Modified Trinkle	12
Features	8	Modified Trinkle	12
Instructions	8	Trinkle	13
To Insert a Wire or Pin	8	Triathlon	13
To Remove a Wire or Pin.	9	AO Large Reamer	13
Chucks with Key	9	DHS/DCS Quick Lock	13
Description	9	Features	14
Features	9	Instructions	14
Instructions	10	To Install an Accessory.	14
To Install an Accessory.	10	To Insert a Pin with the Triathlon Attachment	14
To Remove an Accessory	10	To Remove an Accessory	14
Keyless Chucks	10	Care Instructions	14
Description	10	Troubleshooting	15
Features	10	Specifications	16

Introduction

This instructions for use manual contains information intended to ensure the safe, effective and compliant use of your product. This manual is intended for in-service trainers, physicians, nurses, surgical technologists, and biomedical equipment technicians. Keep and consult this reference manual during the life of the product.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

For additional information, including safety information, in-service training, or current literature, contact your Stryker sales representative or call Stryker customer service at 1-269-323-7700 or 1-800-253-3210. Outside the US, contact your nearest Stryker subsidiary.

Trademarks not the property of Stryker Corporation are the property of their respective owners.

NOTE: The user and/or patient should report any serious product-related incident to both the manufacturer and the Competent Authority of the European Member State where the user and/or patient is established.

Indications For Use

See the instructions for use supplied with the handpiece.

Contraindications

See the instructions for use supplied with the handpiece.

For Use With

Stryker Heavy Duty Rotary Handpiece Attachments are intended for use with the following Stryker Heavy Duty Rotary Handpieces:

DESCRIPTION	REF
System 8 Dual Trigger Rotary Handpiece	8205-000-000
System 8 Single Trigger Rotary Handpiece	8203-000-000
System 7 Dual Trigger Rotary Handpiece	7205-000-000
System 7 Single Trigger Rotary Handpiece	7203-000-000
Electric System 6™ Dual Trigger Rotary Handpiece	6295-000-000
Electric System 6 Single Trigger Rotary Handpiece	6293-000-000
System 6 Dual Trigger Rotary Handpiece	6205-000-000
System 6 Single Trigger Rotary Handpiece	6203-000-000

Safety Directives



WARNINGS:

- Before using this equipment, or any component compatible with this equipment, read and understand the instructions for use. Pay particular attention to safety information. Become familiar with the equipment before use.
- Only healthcare professionals trained and experienced in the use of this medical device should operate this equipment.
- The healthcare professional performing any procedure is responsible for determining the appropriateness of this equipment and the specific technique used for each patient. Stryker, as a manufacturer, does not recommend surgical procedure or technique.
- Upon initial receipt and before each use, clean and sterilize the equipment as indicated. See the care instructions manual supplied with the handpiece.
- Upon initial receipt and before each use, operate the equipment and inspect each component for damage. DO NOT use any equipment if damage is apparent. See the care instructions manual supplied with the handpiece.
- ALWAYS slide the function switch to the safe mode position when the handpiece is idle, before installing an attachment or accessory, or when passing the handpiece to another person.
- DO NOT attempt to install or remove an attachment or cutting accessory while the handpiece is operating.
- DO NOT operate the handpiece in the drill mode position when a reamer attachment and/or accessory is installed in the handpiece.
- Before operating the handpiece, ALWAYS gently tug the attachment and accessory to make sure the attachment is securely locked in the handpiece and the accessory is securely locked in the attachment.
- ALWAYS operate the equipment within the specified environmental condition values. See the instructions for use supplied with the handpiece.
- ALWAYS follow the recommended duty cycle to prevent the equipment from overheating. See the instructions for use supplied with the handpiece.
- DO NOT grasp or touch any rotating component while the handpiece is operating.
- See the specific attachment sections of this manual and the instructions for use and care instructions supplied with the handpiece for additional safety information.

Accessories



WARNINGS:

- Use only Stryker-approved equipment, unless otherwise specified.
- DO NOT modify any equipment without the authorization of the manufacturer.
- DO NOT reuse, reprocess, or repackage a device that is intended for single use only.
 - A single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing.
 - Design features may make cleaning difficult.
 - Reuse may create a contamination risk and compromise structural integrity resulting in operational failure.
 - Critical product information may be lost during repackaging.

Failure to comply may lead to infection or cross infection and result in patient and/or healthcare staff injury.


NOTE: For a complete list of accessories, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

The following Stryker-approved accessories are sold separately:

DESCRIPTION	REF
Universal Chuck Key	4100-131-132

Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
▼	Alignment Mark
S	Small Diameter
M	Medium Diameter
L	Large Diameter
Single Trigger	Single Trigger — This attachment is intended for use with single trigger rotary handpieces only.
	General warning sign

Wire/Pin Collets

NOTE: For System 8 collets, see the instructions for use supplied with the collet.

Description



WARNING: DO NOT use a single trigger collet with a dual trigger rotary handpiece. Installing a single trigger collet onto a dual trigger rotary handpiece creates a pinch point between the clamping lever and the forward trigger of the handpiece.

Wire collet and pin collet attachments are used to install wire and pins. Select the appropriate collet based on the handpiece type and the wire or pin diameter.


All collets have a pre-adjust knob with small (S) and large (L) diameter settings. Newer collet models also have a medium (M) diameter setting. Set the pre-adjust knob to the proper diameter range.

MODEL REF	DIMENSIONS	HANDPIECE TYPE	WIRE/PIN DIAMETER (S)	WIRE/PIN DIAMETER (M)	WIRE/PIN DIAMETER (L)
6203-026-000	Length: 107.1 mm; Height: 162.8 mm; Diameter: 27.6 mm; Mass: 222.3 g	Single and Dual Trigger	0.7 to 1.27 mm	N/A	1.3 to 1.8 mm
6203-036-000	Length: 107.2 mm; Height: 161.9 mm; Diameter: 27.6 mm; Mass: 218.5 g	Single Trigger	[0.028 to 0.050 inch]		[0.051 to 0.071 inch]
7203-026-000	Length: 112.2 mm; Height: 155.4 mm; Diameter: 27.4 mm; Mass: 203.0 g	Single and Dual Trigger	0.7 to 1.4 mm	1.1 to 1.6 mm	1.6 to 2.0 mm
7203-036-000	Length: 112.2 mm; Height: 155.4 mm; Diameter: 27.4 mm; Mass: 203.0 g	Single Trigger	[0.028 to 0.054 inch]	[0.045 to 0.062 inch]	[0.062 to 0.079 inch]

MODEL REF	DIMENSIONS	HANDPIECE TYPE	WIRE/PIN DIAMETER (S)	WIRE/PIN DIAMETER (M)	WIRE/PIN DIAMETER (L)
6203-126-000	Length: 107.1 mm; Height: 162.6 mm; Diameter: 27.6 mm; Mass: 220.8 g	Single and Dual Trigger	2.0 to 2.6 mm [0.079 to 0.102 inch]	N/A	2.7 to 3.2 mm [0.106 to 0.126 inch]
6203-136-000	Length: 107.1 mm; Height: 164.1 mm; Diameter: 27.5 mm; Mass: 218.8 g	Single Trigger			
7203-126-000	Length: 112.2 mm; Height: 155.4 mm; Diameter: 27.4 mm; Mass: 203.0 g	Single and Dual Trigger	2.0 to 2.6 mm [0.079 to 0.102 inch]	2.4 to 2.8 mm [0.094 to 0.110 inch]	2.8 to 3.2 mm [0.110 to 0.126 inch]
7203-136-000	Length: 112.2 mm; Height: 155.4 mm; Diameter: 27.4 mm; Mass: 203.0 g	Single Trigger			
6203-226-000	Length: 107.2 mm; Height: 161.8 mm; Diameter: 27.5 mm; Mass: 220.5 g	Single and Dual Trigger	3.0 to 3.6 mm [0.118 to 0.142 inch]	N/A	3.7 to 4.2 mm [0.146 to 0.165 inch]
6203-236-000	Length: 107.1 mm; Height: 162.4 mm; Diameter: 27.6 mm; Mass: 218.5 g	Single Trigger			
7203-226-000	Length: 112.2 mm; Height: 155.4 mm; Diameter: 27.4 mm; Mass: 203.0 g	Single and Dual Trigger	3.0 to 3.4 mm [0.118 to 0.134 inch]	3.2 to 3.8 mm [0.126 to 0.150 inch]	3.8 to 4.2 mm [0.150 to 0.165 inch]
7203-236-000	Length: 112.2 mm; Height: 155.4 mm; Diameter: 27.4 mm; Mass: 203.0 g	Single Trigger			

Features

NOTE: Wire/Pin Collet (REF 6203-036-000) is shown for reference.

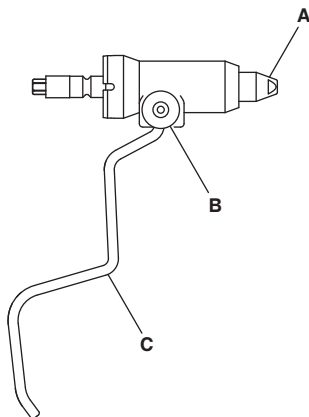

A	Nose
B	Pre-adjust Knob
C	Clamping Lever

Instructions

To Insert a Wire or Pin

1. Pull and rotate the pre-adjust knob to align the symbols for the proper wire or pin diameter range. Make sure the knob retracts and locks in place after alignment.

NOTE: If the pre-adjust knob is difficult to rotate, actuating the clamping lever slightly will help.

2. Insert the wire or pin into the nose of the collet, or through the back of the handpiece, until the desired length extends from the nose. Make sure the wire or pin is inserted far enough so the collet holds the wire or pin before squeezing the clamping lever.
3. Rotate the shift collar on the handpiece to the drill mode position.
4. Slide the function switch on the handpiece to the forward mode position.

5. Squeeze the clamping lever to grasp the wire or pin.
6. Hold the wire or pin against the bone and depress the trigger on the handpiece to drive the wire or pin into the bone.
7. If necessary, perform the following steps to obtain additional wire or pin length:
 - 7.1. Release the clamping lever.
 - 7.2. Pull the handpiece back to allow more wire or pin length.
 - 7.3. Squeeze the clamping lever.
 - 7.4. Depress the trigger on the handpiece.

To Remove a Wire or Pin

1. Place the nose of the collet over the free end of the wire or pin, and then advance the collet as far as possible.
2. Squeeze the clamping lever to grasp the wire or pin.
3. Depress the trigger on the handpiece and pull the handpiece and wire or pin away from the bone.

NOTE: To withdraw threaded pins, slide the handpiece function switch to the reverse mode position, squeeze the clamping lever to grasp the pin, and depress the trigger on the handpiece. (For dual trigger handpieces, depress the reverse trigger.)

Chucks with Key


Description

Chuck attachments hold wires, pins, and drill bits up to 6.4 mm [1/4 inch] in diameter. Each attachment is supplied with a chuck key.

MODEL	DIMENSIONS	REF
1/4 inch Chuck with Key	Length: 92.7 mm; Diameter: 28.6 mm; Mass: 143.6 g	6203-131-000
5/32 inch Chuck with Key	Length: 76.5 mm; Diameter: 25.0 mm; Mass: 75.0 g	6203-132-000

Features

NOTE: Chuck with Key (REF 6203-131-000) is shown for reference.

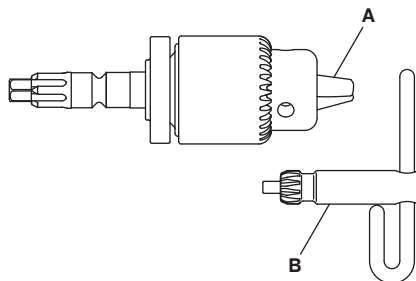

A	Jaws
B	Chuck Key

Instructions

To Install an Accessory

1. Use the chuck key to open the jaws.
2. Insert the accessory into the center of the jaws.
3. Use the chuck key to close the jaws.
4. Visually inspect the accessory to make sure it is centered in the jaws.
5. Tug the accessory to make sure it is secure before operating the handpiece.

To Remove an Accessory

1. Use the chuck key to open the jaws.
2. Remove the accessory.

Keyless Chucks


Description

Keyless chuck attachments hold wires, pins, and drill bits up to 6.4 mm [1/4 inch] in diameter.

MODEL	DIMENSIONS	REF
1/4 inch Keyless Chuck	Length: 105.4 mm; Diameter: 30.0 mm; Mass: 227.4 g	6203-133-000
1/8 inch Keyless Chuck	Length: 91.2 mm; Diameter: 23.9 mm; Mass: 122.7 g	6203-134-000

Features

NOTE: Keyless Chuck (REF 6203-133-000) is shown for reference.

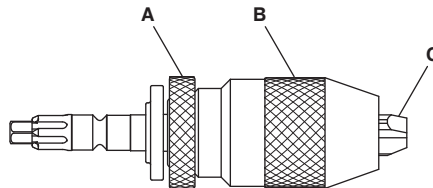

A	Knurled Ring
B	Collar
C	Jaws

Instructions




WARNING: ALWAYS manually close the jaws to secure the accessory. DO NOT operate the handpiece to automatically close the jaws on the accessory.

CAUTION: DO NOT operate the handpiece in the ream mode when using a keyless chuck.

To Install an Accessory

1. Hold the knurled ring with one hand and rotate the collar with the other hand to open the jaws.

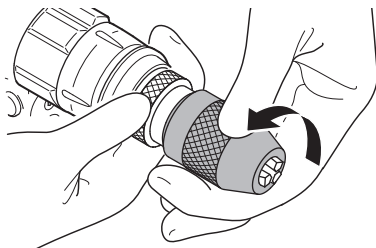

2. Insert the accessory into the center of the jaws.
3. Hold the collar and turn the knurled ring to tightly close the jaws on the accessory.
4. Visually inspect the accessory to make sure it is centered in the jaws.
5. Tug the accessory to make sure it is secure before operating the handpiece.

To Remove an Accessory

1. Hold the knurled ring with one hand and rotate the collar with the other hand to open the jaws.
2. Remove the accessory.

Locking Keyless Chucks


See the instructions for use supplied with this attachment.

Drill and Reamer Attachments

Description


AO® Small Drill

The AO Small Drill attachment (REF 6203-110-000) accepts Synthes accessories such as drill bits, taps, and automatic screwdrivers with appropriate snap-lock shanks. Length: 81.8 mm; Diameter: 22.3 mm; Mass: 105.5 g

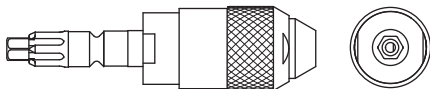


Hudson®

The Hudson attachment (REF 6203-113-000) accepts accessories such as reamers, drill bits, taps, and automatic screwdrivers with Hudson fittings. Length: 80.8 mm; Diameter: 27.9 mm; Mass: 131.3 g

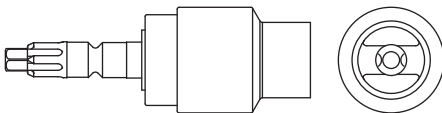


Hudson/Modified Trinkle

The Hudson/Modified Trinkle attachment (REF 6203-135-000) accepts accessories such as reamers, drill bits, and automatic screwdrivers with Hudson or modified Trinkle fittings. Length: 79.0 mm; Diameter: 27.9 mm; Mass: 117.6 g

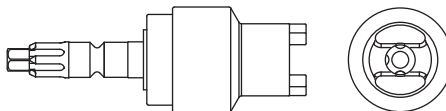


Modified Trinkle

The Modified Trinkle attachment (REF 6203-150-000) accepts accessories such as drill bits, automatic screwdrivers, and reamers with modified Trinkle fittings. Length: 79.2 mm; Diameter: 27.9 mm; Mass: 115.8 g

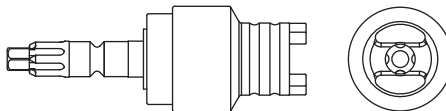


Trinkle


The Trinkle attachment (REF 6203-160-000) accepts accessories such as drill bits, automatic screwdrivers, and reamers with standard Trinkle fittings. Length: 77.7 mm; Diameter: 27.9 mm; Mass: 119.3 g

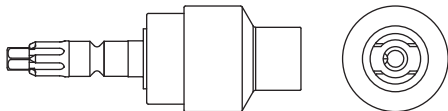**Triathlon®**


The Triathlon attachment (REF 6203-170-000) accepts Triathlon drill bits, headless pin drivers, and headless Triathlon pins. Length: 67.3 mm; Diameter: 27.7 mm; Mass: 63.6 g

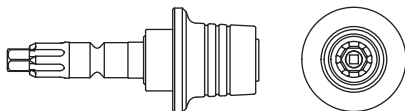**AO Large Reamer**

The AO Large Reamer attachment (REF 6203-210-000) accepts Synthes accessories such as drill bits, automatic screwdrivers, and flexible reamers with appropriate snap-lock shanks. Length: 70.8 mm; Diameter: 27.9 mm; Mass: 107.7 g

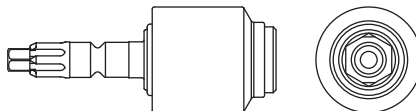
**DHS/DCS Quick Lock**

The DHS/DCS Quick Lock attachment (REF 6203-215-000) accepts Synthes DHS/DCS triple reamers, large quick-coupling drill bits, and screwdriver shafts. Length: 78.8 mm; Diameter: 27.9 mm; Mass: 132.6 g

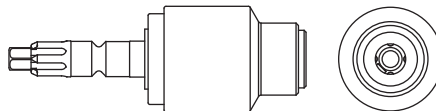

Features

NOTE: Hudson/Modified Trinkle (REF 6203-135-000) is shown for reference.

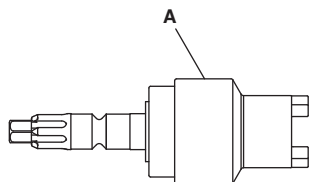

A Retaining Collar

Instructions

NOTE: For the AO Large Reamer attachment, sliding and releasing the retaining collar is not required.

To Install an Accessory

1. Slide back the retaining collar.
2. Insert the accessory into the attachment.
3. Release the retaining collar.
4. Tug the accessory to make sure it is secure before operating the handpiece.

To Insert a Pin with the Triathlon Attachment

1. Fully insert the headless Triathlon pin into the attachment. Make sure the pin is firmly engaged with the attachment.

NOTE: Alternately, insert the pin into a headless pin driver accessory. Then insert the driver accessory into the Triathlon attachment. Make sure the pin and driver accessory are firmly engaged with the attachment.

2. Rotate the shift collar on the handpiece to the drill mode position.
3. Slide the function switch on the handpiece to the forward mode position.
4. Hold the pin against the bone and depress the trigger on the handpiece to drive the pin into the bone.
5. Withdraw the attachment from the pin. The attachment will automatically disengage from the pin.

To Remove an Accessory

1. Slide back the retaining collar.
2. Remove the accessory from the attachment.
3. Release the retaining collar.

Care Instructions

For processing instructions and disposal/recycle information, see the care instructions manual supplied with the handpiece.

Troubleshooting



WARNING: DO NOT disassemble or service this equipment without the authorization of the manufacturer.

NOTE: For service, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

PROBLEM	CAUSE	ACTION
The attachment will not fit or cannot be secured in the handpiece.	The attachment and/or the distal end of the handpiece contains debris.	See the care instructions manual supplied with the handpiece.
	The attachment is damaged.	Return the attachment to Stryker for repair.
	The handpiece is damaged.	Return the handpiece to Stryker for repair.
The attachment and/or accessory wobbles in the handpiece.	The accessory is damaged.	Inspect the accessory for damage and replace the accessory as required.
	The accessory is the wrong size for the attachment.	Install a different accessory or attachment as required.
	The accessory extends too far from the distal end of the attachment, or is not properly centered in the attachment.	Remove and properly install the accessory. If wobble persists, return the handpiece and attachment to Stryker for repair.
	A reamer attachment and/or accessory is installed in the handpiece and the shift collar is in the drill mode position.	Rotate the shift collar to the ream mode position.
The accessory falls out of the attachment.	The accessory is not properly installed.	Remove and properly install the accessory.


PROBLEM	CAUSE	ACTION
The handpiece operates but the accessory does not move.	The attachment is not fully installed in the handpiece.	Remove and install the attachment. Make sure the attachment is securely locked in the handpiece.
	The shift collar on the handpiece is positioned between the drill and ream mode positions.	Fully rotate the shift collar to either the drill mode or ream mode position.
	Excessive pressure is being applied to the cutting accessory.	Release the pressure and allow the cutting accessory to do the cutting.
	The handpiece is damaged.	Return the handpiece to Stryker for repair.
The pin accessory cannot be removed from the collet attachment.	The collet mechanism is jammed.	Gently tap the side of the pin to dislodge it from the collet. CAUTION: DO NOT attempt to force the pin back through the collet.
	The collet is damaged.	Return the attachment to Stryker for repair.

Specifications

CAUTION: ALWAYS store the equipment within the specified environmental condition values throughout its useful life.

Environmental Conditions for Storage and Transportation:


Temperature Limitation:

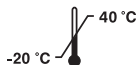

Humidity Limitation:



Atmospheric Pressure Limitation:

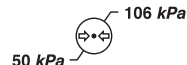

NOTE: When used with handpieces, the attachments described in this manual are applied parts. For more information, see the instructions for use supplied with the handpiece.

ES/DE/FR/IT/NL	7203-001-712
JA/ZH/KO	7203-001-722
SV/DA/FI/PT/NO	7203-001-732
PL/EL	7203-001-754
TR	7203-001-764
RU	7203-001-774



Stryker Instruments
1941 Stryker Way
Portage, Michigan
(USA) 49002
1-269-323-7700
1-800-253-3210

stryker[®]