stryker[®]

Stryker SmartLife[®] Battery Packs

REF 7212-000-000
7215-000-000

Instructions For Use

R_x ONLY

CE

ENGLISH (EN)

ESPAÑOL (ES)

DEUTSCH (DE)

FRANÇAIS (FR)

ITALIANO (IT)

NEDERLANDS (NL)

SVENSKA (SV)

DANSK (DA)

SUOMI (FI)

PORTUGUÊS (PT)

NORSK (NO)

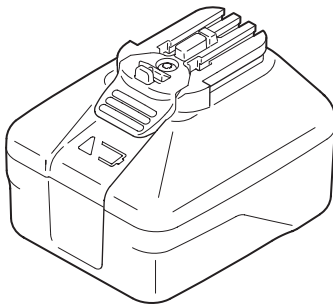
POLSKI (PL)

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Introduction

This *Instructions For Use* manual is the most comprehensive source of information for the safe and effective use of your product. This manual may be used by in-service trainers, physicians, nurses, surgical technologists, and biomedical equipment technicians. Keep and consult this reference manual during the life of the product.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

If additional information or in-service training is required, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

NOTE: The user and/or patient should report any serious product-related incident to both the manufacturer and the Competent Authority of the European Member State where the user and/or patient is established.

Indications For Use

See the instructions for use supplied with the handpiece.

Contraindications

See the instructions for use supplied with the handpiece.

For Use With

The battery pack is only intended for use with the following Stryker handpieces:

DESCRIPTION	REF
System 7 Handpieces	720X-XXX-XXX series
Cordless Driver 4 Handpiece	4405-000-000
Sabo®2 Sagittal Saw	4408-000-000

Charge the battery pack with one of the following Stryker battery chargers configured with a Stryker System 6™ Charger Module (REF 6110-625-000):

DESCRIPTION	REF
System 6 Battery Charger	6110-120-000
Universal Battery Charger	7110-120-000

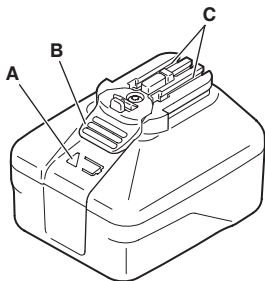
User/Patient Safety

WARNINGS:

- Before using any system component, or any component compatible with this system, read and understand the instructions. Pay particular attention to WARNING information. Become familiar with the system components prior to use.
- Only trained and experienced healthcare professionals should use this equipment.
- The healthcare professional performing any procedure is responsible for determining the appropriateness of this equipment and the specific technique used for each patient. Stryker, as a manufacturer, does not recommend surgical procedure or technique.
- Upon initial receipt and before each use, clean and sterilize the equipment as indicated. See the *Heavy Duty Care Instructions* manual supplied with the handpiece for processing instructions.
- Upon initial receipt and before each use, operate the equipment and inspect each component for damage. DO NOT use any equipment if damage is apparent. See the *Heavy Duty Care Instructions* manual supplied with the handpiece for inspection criteria.
- Before use, ALWAYS allow the equipment to reach the operation temperature range specified in the instructions for use supplied with the handpiece.
- To prevent the equipment from overheating, ALWAYS follow the recommended duty cycle specified in the instructions for use supplied with the handpiece.
- Risk of Fire, Burns and Explosion. Do not Short Circuit, Open, Crush or Incinerate. Sterilize only per instructions. Consult instructions for use.
- DO NOT allow the battery contacts to touch metal objects.

Features

B | **Battery Latch** – Retains the battery pack in the handpiece.

C | **Battery Contacts** – Interlock with the handpiece to complete the electric circuit.

A | **Indicator Light** – Provides status information for the battery pack:

- The light will blink when the battery pack is near the end of its charge cycle.
- The light will illuminate in a steady state for 10 seconds when the battery pack is expended.

Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	Battery Pack Status
	General warning sign
	Direct current

Instructions

NOTES:

- See the instructions for use supplied with the battery charger to properly charge the battery pack before each use.
- For maximum capacity, charge the battery pack before sterilization, even if the battery pack has not been used.
- A typical charging sequence will take between 5 and 30 minutes. Charging a fully expended large battery pack may take up to 75 minutes.
- For maximum capacity, store the battery pack on the battery charger to maintain a full charge. **DO NOT** store the battery pack installed in a handpiece. Discharge will occur even though the handpiece is not operating.
- Stryker recommends using battery packs that have been charged within the last 10 days.
- See the instructions for use supplied with the handpiece to properly install and remove the battery pack from the handpiece.

- Typical battery pack run time is 20 minutes or longer, depending on surgical procedure or technique.
- When the indicator light blinks, reducing the load on the handpiece may extend the usage time of the battery pack.

Care Instructions

For processing instructions, see the care instructions manual supplied with the handpiece.

Disposal/Recycle

See the care instructions manual supplied with the handpiece.

Li-ION

Per Directive 2006/66/EC, collect and recycle batteries/battery packs according to EU Member State regulations.

This rechargeable battery pack contains Lithium-Ion.

Specifications

CAUTION: ALWAYS store the equipment within the specified environmental condition values throughout its useful life.

NOTE: Specifications are approximate and may vary between devices.

Model:	Stryker SmartLife Battery Pack, Small (REF 7212-000-000)	Stryker SmartLife Battery Pack, Large (REF 7215-000-000)
Type:	Rechargeable, Lithium-Ion Battery Pack	
Dimensions:	58 mm [2.3 inch] height 71 mm [2.8 inch] width 84 mm [3.3 inch] length	76 mm [3.0 inch] height 71 mm [2.8 inch] width 84 mm [3.3 inch] length
Mass:	0.27 kg [0.6 lb]	0.41 kg [0.9 lb]
Electrical Output:	Refer to battery housing for voltage rating.	
Capacity:	Refer to battery housing for watt-hour rating.	
Mode of Operation:	See the mode of operation in the instructions for use supplied with the handpiece.	
Ingress Protection:	IPX0	
Environmental Conditions:	See the operation, storage, and transportation specifications in the instructions for use supplied with the handpiece.	

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