<sup>®</sup>  
**stryker**


# 1/4 inch (6.4 mm) Locking Keyless Chuck

**REF** 8203-133-000

**Instructions For Use**

**R<sub>x</sub> ONLY**    **CE**

ENGLISH (EN)  
ESPAÑOL (ES)  
DEUTSCH (DE)  
FRANÇAIS (FR)  
ITALIANO (IT)  
NEDERLANDS (NL)  
SVENSKA (SV)  
DANSK (DA)  
SUOMI (FI)  
PORTUGUÊS (PT)  
NORSK (NO)  
POLSKI (PL)  
ΕΛΛΗΝΙΚΑ (EL)  
TÜRKÇE (TR)  
日本語 (JA)  
中文 (ZH)  
한국어 (KO)

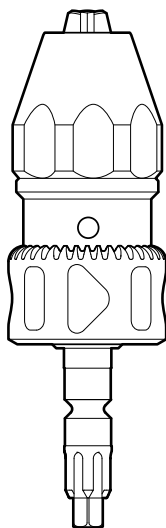## Introduction

This instructions for use manual contains information intended to ensure the safe, effective and compliant use of your product. This manual is intended for in-service trainers, physicians, nurses, surgical technologists, and biomedical equipment technicians. Keep and consult this reference manual during the life of the product.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

For additional information, including safety information, in-service training, or current literature, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

**NOTE:** The user and/or patient should report any serious product-related incident to both the manufacturer and the Competent Authority of the European Member State where the user and/or patient is established.

## Indications For Use

See the instructions for use supplied with the handpiece.

## Contraindications

See the instructions for use supplied with the handpiece.

## Description

The Locking Keyless Chuck handles wires, pins, drill bits, and reamers up to 1/4 inch (6.4 mm) diameter. The chuck can be operated in forward, reverse, or oscillation modes. Length: 108.7 mm; Diameter: 33 mm; Weight: 285 g

**NOTE:** The Locking Keyless Chuck is supplied with a key; however, the key is used **ONLY** when the accessory cannot be removed by hand. See the *To Remove an Accessory* section.

## For Use With

The System 8 1/4" (6.4 mm) Locking Keyless Chuck is intended for use with the following Stryker Heavy Duty Rotary Handpieces:

DESCRIPTION	REF
System 8 Single Trigger Rotary Handpiece	8203-000-000
System 8 Dual Trigger Rotary Handpiece	8205-000-000
System 7 Single Trigger Rotary Handpiece	7203-000-000
System 7 Dual Trigger Rotary Handpiece	7205-000-000
System 6™ Single Trigger Rotary Handpiece	6203-000-000
System 6 Dual Trigger Rotary Handpiece	6205-000-000
Electric System 6 Single Trigger Rotary Handpiece	6293-000-000
Electric System 6 Dual Trigger Rotary Handpiece	6295-000-000

## Safety Directives

### WARNINGS:

- Before using this equipment, or any component compatible with this equipment, read and understand the instructions for use. Pay particular attention to safety information. Become familiar with the equipment before use.
- Only healthcare professionals trained and experienced in the use of this medical device should operate this equipment.
- The healthcare professional performing any procedure is responsible for determining the appropriateness of this equipment and the specific technique used for each patient. Stryker, as a manufacturer, does not recommend surgical procedure or technique.
- Upon initial receipt and before each use, clean and sterilize the equipment as indicated. See the care instructions manual supplied with the handpiece.

**WARNINGS:**

- Upon initial receipt and before each use, operate the equipment and inspect each component for damage. **DO NOT** use any equipment if damage is apparent. See the care instructions manual supplied with the handpiece.
- **ALWAYS** slide the function switch to the safe mode position when the handpiece is idle, before installing an attachment or accessory, or when passing the handpiece to another person.
- **DO NOT** attempt to install or remove an attachment or accessory while the handpiece is operating.
- **DO NOT** operate the handpiece in the drill mode position when a reamer attachment and/or accessory is installed in the handpiece.
- Before operating the handpiece, **ALWAYS** gently tug the attachment and accessory to make sure the attachment is securely locked in the handpiece and the accessory is securely locked in the attachment.
- **ALWAYS** operate the equipment within the specified environmental condition values. See the instructions for use supplied with the handpiece.
- **ALWAYS** follow the recommended duty cycle to prevent the equipment from overheating. See the instructions for use supplied with the handpiece.
- **DO NOT** grasp or touch any rotating component while the handpiece is operating.
- **DO NOT** disassemble or service this equipment without the authorization of the manufacturer.

**CAUTION:** **ALWAYS** store the equipment within the specified environmental condition values throughout its useful life. See the instructions for use supplied with the handpiece.

**NOTE:** When used with handpieces, the attachment described in this manual is an applied part. For more information, see the instructions for use supplied with the handpiece.

## Accessories

### WARNINGS:

- Use only Stryker-approved equipment, unless otherwise specified.
  - DO NOT modify any equipment without the authorization of the manufacturer.
  - DO NOT reuse, reprocess, or repackage a device that is intended for single use only.
    - A single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing.
    - Design features may make cleaning difficult.
    - Reuse may create a contamination risk and compromise structural integrity resulting in operational failure.
    - Critical product information may be lost during repackaging.
- Failure to comply may lead to infection or cross infection and result in patient and/or healthcare staff injury.

**NOTE:** For a complete list of accessories, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

The following Stryker-approved accessories are sold separately:


DESCRIPTION	REF
1/4" Chuck Key	1331-001-009

## Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	General warning sign
	Lock
	Unlock

## Features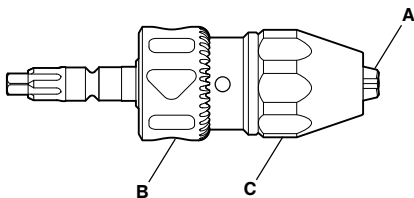

A	Jaws
B	Collar
C	Sleeve

## Instructions


**WARNING:** ALWAYS manually close the jaws to secure the accessory. DO NOT operate the handpiece to automatically close the jaws on the accessory.

**NOTE:** If the chuck is attached to the handpiece during accessory installation, rotating the chuck collar will also rotate the handpiece drive mechanism.

### To Install an Accessory


1. Hold the sleeve with one hand and rotate the collar with the other hand to open the jaws.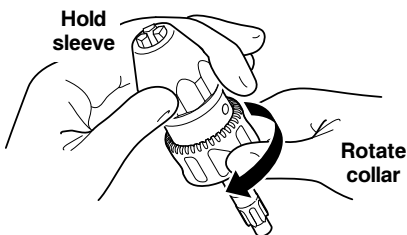

2. Insert the accessory into the center of the jaws.

**NOTE:** DO NOT use the chuck key to close the chuck jaws, as this prevents the collar from locking and may cause the jaws to loosen when the chuck is operated in reverse or oscillation modes.

3. Hold the sleeve and rotate the collar to tightly close the jaws on the accessory. The collar will move forward with one or more audible clicks. Continue rotating the collar to maximum tightness.

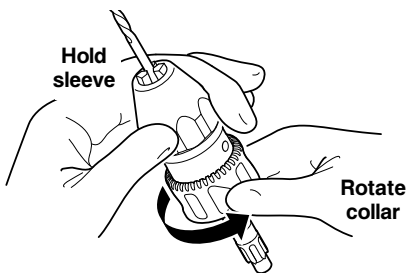4. Visually inspect the accessory to make sure it is centered in the jaws. If the accessory has three or six flats, make sure the jaws are centered directly on three of the flats.
5. Tug the accessory to make sure it is secure.
6. Install the chuck in the handpiece.

## To Remove an Accessory

1. Remove the chuck from the handpiece.
2. Hold the sleeve with one hand and rotate the collar with the other hand to open the jaws.
3. Remove the accessory.

**NOTE:** If the chuck jaws cannot be opened by hand, rotate the collar to the unlock position, and then use the key to open the jaws.

## Troubleshooting

See the instructions for use supplied with the handpiece.

## Care Instructions

For processing instructions and disposal/recycle information, see the care instructions manual supplied with the handpiece.

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