stryker®

## System 8 Battery Packs

REF 8212-000-000

REF 8215-000-000

### Instructions For Use

R<sub>x</sub> ONLY

CE

ENGLISH (EN)

ESPAÑOL (ES)

DEUTSCH (DE)

FRANÇAIS (FR)

ITALIANO (IT)

NEDERLANDS (NL)

SVENSKA (SV)

DANSK (DA)

SUOMI (FI)

PORTUGUÊS (PT)

NORSK (NO)

POLSKI (PL)

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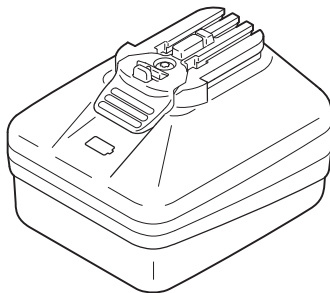
TÜRKÇE (TR)

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## Introduction

This instructions for use manual contains information intended to ensure the safe, effective, and compliant use of your product. This manual is intended for in-service trainers, physicians, nurses, surgical technologists, and biomedical equipment technicians. Keep and consult this reference manual during the life of the product.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

For additional information, including safety information, in-service training, or current literature, contact your Stryker sales representative or call Stryker customer service at 1-269-323-7700 or 1-800-253-3210. Outside the US, contact your nearest Stryker subsidiary.

## Indications For Use

See the instructions for use supplied with the handpiece.

## Contraindications

See the instructions for use supplied with the handpiece.

## For Use With

The battery pack is only intended for use with the following Stryker handpieces:

DESCRIPTION	REF
System 8 Handpieces	820X-XXX-XXX series
System 7 Handpieces	720X-XXX-XXX series
System 8 Cordless Driver	4505-000-000
Cordless Driver 4	4405-000-000
System 8 Sabo® Sagittal Saw	4508-000-000
Sabo2 Sagittal Saw	4408-000-000

Charge the battery pack with one of the following Stryker battery chargers configured with a Stryker System 6 Charger Module (REF 6110-625-000):

DESCRIPTION	REF
Universal Battery Charger	7110-120-000
System 6 Battery Charger	6110-120-000

## Safety Directives

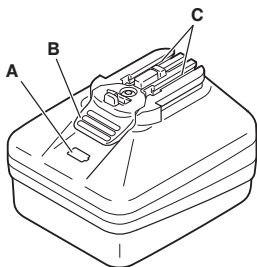
### WARNINGS:

- Before using this equipment, or any component compatible with this equipment, read and understand the instructions for use. Pay particular attention to safety information. Become familiar with the equipment before use.
- Only healthcare professionals trained and experienced in the use of this medical device should operate this equipment.
- The healthcare professional performing any procedure is responsible for determining the appropriateness of this equipment and the specific technique used for each patient. Stryker, as a manufacturer, does not recommend surgical procedure or technique.
- Upon initial receipt and before each use, clean and sterilize the equipment as indicated. See the care instructions manual supplied with the handpiece.
- Upon initial receipt and before each use, inspect the equipment for damage. DO NOT use any equipment if damage is apparent.
- Before use, ALWAYS allow the equipment to reach the operation temperature range specified in the instructions for use supplied with the handpiece.
- To prevent the equipment from overheating, ALWAYS follow the recommended duty cycle specified in the instructions for use supplied with the handpiece.
- DO NOT allow the battery contacts to touch metal objects.

**NOTE:** The user and/or patient should report any serious product-related incident to both the manufacturer and the Competent Authority of the European Member State where the user and/or patient is established.

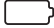
## Features

<b>A</b>	<b>Indicator Light</b> – Provides status information for the battery pack. The light will illuminate in a steady state for 10 seconds when the battery pack is expended.
<b>B</b>	<b>Battery Latch</b> – Retains the battery pack in the handpiece.
<b>C</b>	<b>Battery Contacts</b> – Interlock with the handpiece to complete the electric circuit.

## Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	General warning sign
	Battery pack status (indicator light)
<p style="text-align: center;"><b>WARNING</b> <b>ADVERTISSEMENT</b></p>	<p> <b>WARNING:</b> Battery contains Lithium-ion material. Risk of fire, burns, and explosion. DO NOT short circuit, open, crush or incinerate. Sterilize only per instructions. Consult instructions for use.</p>

## Instructions

### NOTES:

- See the instructions for use supplied with the battery charger to properly charge the battery pack before each use.
- For maximum capacity, charge the battery pack before sterilization, even if the battery pack has not been used.
- A typical charging sequence will take between 5 and 30 minutes. Charging a fully expended large battery pack may take up to 90 minutes.
- For maximum capacity, store the battery pack on the battery charger to maintain a full charge. **DO NOT** store the battery pack installed in a handpiece. Discharge will occur even though the handpiece is not operating.
- Stryker recommends using battery packs that have been charged within the last 30 days.
- See the instructions for use supplied with the handpiece to properly install and remove the battery pack from the handpiece.
- When the indicator light blinks, reducing the load on the handpiece may extend the usage time of the battery pack.

## Care Instructions

For processing instructions, see the care instructions manual supplied with the handpiece.

### Disposal/Recycle

This rechargeable battery pack contains Lithium-Ion.

### Li-ION

To comply with European Community Batteries Directive 2006/66/EC, this device has been designed for safe removal of the batteries at the end-of-life by a waste treatment facility. Infected units should be decontaminated before they are sent for recycling. If it is not possible to decontaminate the unit for recycling, the hospital should not attempt to remove the batteries from waste equipment. Continued disposal of small amounts of portable batteries to landfill and incineration is allowed under Directive 2006/66/EC and Member State regulations.

## Specifications

**CAUTION:** ALWAYS store the equipment within the specified environmental condition values throughout its useful life.

<b>Model:</b>	System 8 Battery Pack, Small (REF 8212-000-000)	System 8 Battery Pack, Large (REF 8215-000-000)
<b>Type:</b>	Rechargeable, Lithium-Ion Battery Pack	
<b>Dimensions:</b>	58 mm [2.3 inch] height 71 mm [2.8 inch] width 84 mm [3.3 inch] length	76 mm [3.0 inch] height 71 mm [2.8 inch] width 84 mm [3.3 inch] length
<b>Mass:</b>	0.27 kg [0.60 lb]	0.42 kg [0.93 lb]
<b>Voltage:</b>	9.9 V $\text{---}$ (Direct current)	9.9 V $\text{---}$
<b>Energy:</b>	10.9 Wh	21.8 Wh
<b>Capacity:</b>	1100 mAh	2200 mAh
<b>Current:</b>	30 A (Limited)	60 A (Limited)
<b>Cell Designation:</b>	3IFpR19/66	3IFpR19/66-2
<b>Mode of Operation:</b>	See the instructions for use supplied with the handpiece.	
<b>Ingress Protection:</b>	IPX0	
<b>Environmental Conditions:</b>	See the instructions for use supplied with the handpiece.	

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