Manufactured and Distributed by:
Stryker Leibinger GmbH & Co. KG
Bötzingen Straße 41
79111 Freiburg, Germany
t: +49 761 4512 0

Distributed by:
Stryker Navigation
4100 East Milham Avenue
Kalamazoo, MI 49001 USA
t: +1 269 323 7700

CE conformity:
Essential Requirements of Medical Device
Directive 93/42/EEC

Screwdriver

REF 6003-100-100

Universal Joint Screwdriver

REF 6003-100-110

Instructions For Use

Intended Use

Screwdriver and Universal Joint Screwdriver are intended for adjustment and tightening/loosening operations of Stryker Navigation surgical instruments such as, but not limited to, the Pivotal™ Resection Guide (REF 6003-100-000), OrthoLock™ (REF 6007-003-000), the NavLock (REF 6000-999-002/-003/-004/-005), the Nav MIS Jig (REF 6003-200-010/-020/-030), and any other Stryker Navigation instruments which provide a compatible interface (refer to the respective instruction for use).

Cleaning and Sterilization Recommendations

Clean and sterilize the instrument before first and every use. Refer to the Cleaning and Sterilization Guide (REF 6000-005-750).

Cleaning Group: IV

We recommend the following sterilization parameters:

Pre Vac: 3 Pre-vacuumed sterilizer cycles, 270–273 °F (132–134 °C), double wrapping, 4 min. minimum exposure, 10 min. minimum drying time

270 °F Gravity: Gravity Displacement Sterilizer, 270–273 °F (132–134 °C), double wrapping, 34 min. minimum exposure, 10 min. minimum drying time

Flash Gravity: Gravity Displacement Sterilizer, 270–273 °F (132–134 °C), without wrapping or other covers, 10 min. preheating time, 10 min. minimum exposure, 10 min. minimum drying time

The Sterilization Recommendation validation is based on ANSI/AAMI/ISO 11134 protocol. The respective individual laws, standards, directives and instructions of each country must be followed and have priority over the specified procedure (e.g. flash gravity). It is the user's responsibility to consider this and to acquire the corresponding information.

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