<p>NOTE: This portion of the document should not appear on produced Labels or IFUs.</p> <p>Stryker Instruments (269) 323-7700 (800) 253-3210</p>	<p><u>Dimensions:</u> 5 inch (width) x 5 inch Booklet</p>	<p><u>Color/Material/Finish:</u> Color Graphics on White Background 20# Bond or Equivalent</p>	<p><u>Label Stock:</u> N/A</p>	
	<p><u>Print Location:</u> Print Center</p>	<p><u>Suppliers/Services:</u> TransPerfect MDS</p>	<p><u>Description/Type:</u> Instructions For Use</p>	
			<p><u>Part Number:</u> 4126-120-700</p>	<p><u>Rev.</u> L</p>

[®]
stryker

Battery Protector Kits


REF 4126-000-000
4222-000-000
4226-000-000
6127-000-000

Instructions For Use

R_x ONLY

CE

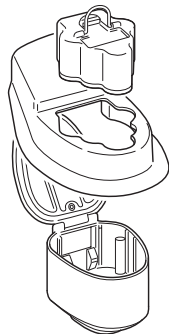ENGLISH (EN)	DANSK (DA)
ESPAÑOL (ES)	SUOMI (FI)
DEUTSCH (DE)	PORTUGUÉS (PT)
FRANÇAIS (FR)	NORSK (NO)
ITALIANO (IT)	POLSKI (PL)
NEDERLANDS (NL)	ΕΛΛΗΝΙΚΑ (EL)
SVENSKA (SV)	TÜRKÇE (TR)

2016-03

4126-120-700 Rev-L

www.stryker.com

Print Date: Jun 14, 2016 11:18:38 AM

0000003946, Rev. L Effective Date: Jun 14, 2016 10:36:00 AM

Introduction

This *Instructions For Use* manual is the most comprehensive source of information for the safe and effective use of your product. This manual may be used by in-service trainers, physicians, nurses, surgical technologists, and biomedical equipment technicians. Keep and consult this reference manual during the life of the product.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. **ALWAYS** comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. **ALWAYS** comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

If additional information or in-service training is required, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

Indications For Use

Battery Protector Kits provide a sterile enclosure for Stryker Non-sterile Batteries. After a Non-sterile Battery is charged, it is placed in a sterilized Aseptic Housing that is loaded into a Stryker handpiece. See the *For Use With* section.

Contraindications

None known.

For Use With

Stryker Battery Protector Kits contain the following components and are intended for use with the following Stryker handpieces:

KIT CONTENTS

KIT	Non-sterile Battery	Aseptic Housing	Transfer Shield	FOR USE WITH HANDPIECES
Cordless Driver Small Battery Protector Kit (REF 4222-000-000)	REF 4222-110-000	REF 4222-120-000	REF 4222-130-000	Cordless Driver (REF 4100-000-000), Cordless Driver 2 (REF 4200-000-000), Recip/Sternum Saw (REF 4107-000-000 and REF 4207-000-000)
Cordless Driver Large Battery Protector Kit (REF 4226-000-000)	REF 4126-110-000	REF 4226-120-000	REF 4126-130-000	Cordless Driver (REF 4100-000-000), Cordless Driver 2 (REF 4200-000-000), Recip/Sternum Saw (REF 4107-000-000 and REF 4207-000-000)
System 5 Battery Protector Kit (REF 4126-000-000)	REF 4126-110-000	REF 4126-120-000	REF 4126-130-000	Rotary (REF 4103-000-000 and 4203-000-000), Dual Trigger (REF 4205-000-000), Reciprocating Saw (REF 4106-000-000 and REF 4206-000-000), Sagittal Saw (REF 4108-000-000 and REF 4208-000-000)
System 6 Small Battery Protector Kit (REF 6127-000-000)	REF 4222-110-000	REF 6127-120-000	REF 4222-130-000	Cordless Driver 3 (REF 4300-000-000), Single Trigger Rotary (REF 6203-000-000), Dual Trigger Rotary (REF 6205-000-000), Sternum Saw (REF 6207-000-000), Sabo® Sagittal Saw (REF 4300-034-000), System 7 Handpieces (REF 720X-XXX-XXX series)

User/Patient Safety

WARNINGS:

- Only trained and experienced healthcare professionals should use this equipment. Before using any system component or any component compatible with this system, read and understand the instructions. Pay particular attention to WARNING information. Become familiar with the system components prior to use.
- Upon initial receipt and before each use, inspect non-sterile batteries for damage. DO NOT charge or use if there is evidence of damage. Electrolytes may leak from damaged batteries and cause serious chemical burns to the surgical site, skin and/or eyes.
- DO NOT modify any system component or accessory.
- DO NOT use this equipment in areas in which flammable anesthetics or flammable agents are mixed with air, oxygen or nitrous oxide.
- See the instructions supplied with the non-sterile battery and the System 5 or System 6 Charger used to recharge the batteries. Instructions contain important safety information and charging instructions.
- ALWAYS remove the non-sterile battery from the aseptic housing before charging the battery. Failure to comply may result in product damage.
- Sterilize the aseptic housings and transfer shields before first and every use. See the care instructions manual supplied with the handpiece.
- DO NOT sterilize the non-sterile batteries. After charging, a non-sterile battery is placed in a sterilized aseptic housing, then loaded into a handpiece.

Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Lock		Refer to instruction manual/ booklet
	Unlock		General warning sign
	Alignment marker	 Ni-Cd	This rechargeable battery pack contains Nickel-Cadmium material. To comply with European Community Batteries Directive 2006/66/EC, this device has been designed for safe removal of the batteries by the end user using the instructions provided. Infected units and batteries should be decontaminated before they are sent for recycling.

Instructions

To Install Non-sterile Battery

WARNING: DO NOT use the aseptic housing, transfer shield, or non-sterile battery unless they are dry.

CAUTION: DO NOT allow the battery contact terminals to touch metal objects.

NOTE: Ensure more than one sterilized battery protector kit is available in case a second battery installation is necessary.

Sterile Assistant Action

1. Open the lid (Figure 1).

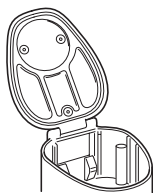
Figure 1 – Open Lid

2. With the aseptic housing chamber open, place the transfer shield onto the aseptic housing and over its chamber (Figure 2).

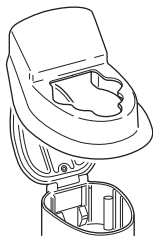
Figure 2 – Place Transfer Shield

3. Present the chamber to the circulating assistant.

Circulating Assistant Action

CAUTION: DO NOT contaminate the aseptic housing. The transfer shield guards the aseptic housing from contamination during battery installation.

4. Hold the non-sterile battery by the handle.
5. While aligning the battery's housing profile with the transfer shield and chamber opening, place the non-sterile battery into the chamber (Figure 3).

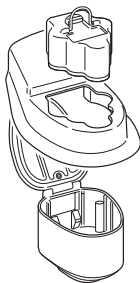
Figure 3 – Install Battery

6. Remove the contaminated transfer shield and set it aside.

WARNING: Sterilize the contaminated transfer shield before using it again. See the care instructions manual supplied with the handpiece.

Sterile Assistant Action

7. Close the aseptic housing lid and rotate its latch clockwise 90° until the detent clicks into place. Ensure the marker aligns with the lock symbol (Figure 4).

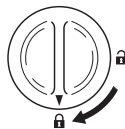
Figure 4 – Lock Lid

WARNING: Failure to turn the latch completely to the lock symbol may result in the lid opening, exposing the non-sterile battery.

8. Slide the aseptic housing containing the installed non-sterile battery firmly into the handpiece until it snaps into place.

To Replace Battery

WARNING: After installation, DO NOT remove a non-sterile battery from a sterile housing. If a new battery is required, use another sterile battery protector kit and perform the installation procedure.

NOTE: It is important to keep non-sterile batteries as clean as possible because they cannot be sterilized.

1. Wipe debris from the aseptic housing.
2. Rotate the latch counterclockwise 90° to the unlocked position. Open the lid.
3. Using the battery handle, lift the non-sterile battery out of the aseptic housing chamber.

Care Instructions

For processing instructions and disposal/recycle information, see the care instructions manual supplied with the handpiece.

Specifications

NOTE: Specifications are approximate and may vary between devices.

Non-sterile Batteries

Model:	Non-sterile Large Battery (REF 4126-110-000)	Non-sterile Small Battery (REF 4222-110-000)
Dimensions:	47 mm [1.9 inch] height 73 mm [2.9 inch] width 91 mm [3.6 inch] length	40 mm [1.6 inch] height 57 mm [2.2 inch] width 71 mm [2.8 inch] length
Mass:	0.36 kg [0.79 lb]	0.16 kg [0.35 lb]
Electrical:	1.75 A charge 9.6 V --- (Direct Current) output	1.0 A charge 9.6 V --- output

Aseptic Housings

Model:	System 5 Aseptic Housing (REF 4126-120-000)	Cordless Driver Small Aseptic Housing (REF 4222-120-000)	Cordless Driver Large Aseptic Housing (REF 4226-120-000)	System 6 Small Aseptic Housing (REF 6127-120-000)
Dimensions:	76 mm [3.0 inch] height, 83 mm [3.3 inch] width, 108 mm [4.3 inch] length	147 mm [5.8 inch] height, 67 mm [2.6 inch] width, 90 mm [3.5 inch] length	156 mm [6.1 inch] height, 83 mm [3.3 inch] width, 108 mm [4.3 inch] length	69 mm [2.7 inch] height, 67 mm [2.6 inch] width, 90 mm [3.5 inch] length
Mass:	0.30 kg [0.66 lb]	0.26 kg [0.57 lb]	0.29 kg [0.64 lb]	0.25 kg [0.54 lb]

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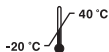
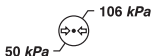
Steril Olmayan Bataryalar (Non-sterile Batteries)

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Aseptik Muhafazalar (Aseptic Housings)

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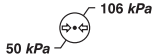
stryker[®]

Signature History Report

Version	Signature	Role	Event Date	Vote
L	Hermiz, Raquel (Contractor)	CA_Owner	2016-03-17 09:18:54 EDT	Continue
L	Pearson, Dawn	Editor	2016-03-17 14:43:58 EDT	Continue
L	Hermiz, Raquel (Contractor)	CA_Owner	2016-03-17 14:54:29 EDT	Continue
L	Collins, Jud	CA_Marketing Manager	2016-03-17 14:56:39 EDT	Approve
L	Sidow, Brian	CA_Clinical Sciences	2016-03-18 09:46:35 EDT	Approve
L	Brown Donna	CA_Product Safety	2016-03-18 09:50:16 EDT	Approve
L	Meskinaoui Samir	CA_Quality Engineer	2016-03-21 08:17:47 EDT	Approve
L	Modi Karankumar	CA_Regulatory Affairs	2016-03-21 09:20:12 EDT	Approve
L	Rane Advait	CA_Product Design Engineer	2016-03-21 15:33:48 EDT	Approve
L	Snell, Tami	Document Control	2016-03-21 15:51:49 EDT	Complete
L	Honeck, Holly (Contractor)	Editor	2016-04-19 13:18:13 EDT	Continue
L	Hermiz, Raquel (Contractor)	CA_Owner	2016-06-13 15:35:54 EDT	Approve

NOTE: IF THIS REPORT IS BLANK, PLEASE SEE WINDCHILL ECN FOR SIGNATURES.

Related Information

CR Number	CR0195190
CR Class	DMR-Commercially Released
Originating Entity	INST: Kalamazoo
Origin	UDI
Initial CR Approval Date	03/20/2015 16:21:43
CN Number	CN105608

Related Information

CA Number	CA167849
NPD Disposition Plan	