<p><i>Note: This portion of the document should not appear on produced Labels or IFUs.</i></p> <p>Stryker Instruments (269) 323-7700 (800) 253-3210</p>	<p>Dimensions: 8.5 inch (width) x 11 inch Booklet</p>	<p>Color/Material/Finish: Black Graphics on White Background 20# Bond or Equivalent</p>	<p>Label Stock: N/A</p>	
			<p>Description/Type: Instructions For Use</p>	
			<p>Part Number: 7102-450-700</p>	<p>Rev. F</p>

stryker®


Sterilization Cases

REF 7102-450-040
7102-452-040
7102-453-040
7102-454-040
7102-455-040

Instructions For Use

R_x ONLY

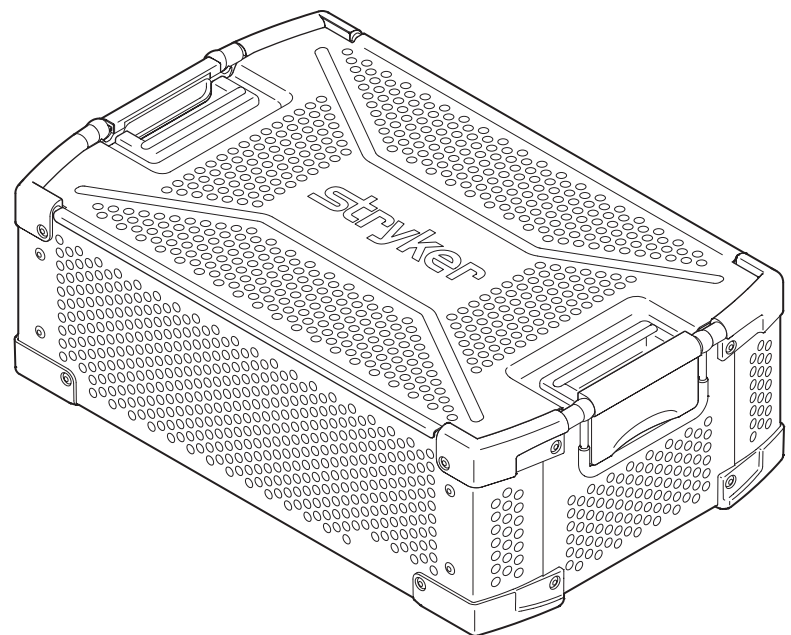CE

ENGLISH (EN)
ESPAÑOL (ES)
DEUTSCH (DE)
FRANÇAIS (FR)
ITALIANO (IT)
NEDERLANDS (NL)
SVENSKA (SV)
DANSK (DA)
SUOMI (FI)
PORTUGUÊS (PT)
NORSK (NO)
POLSKI (PL)
ΕΛΛΗΝΙΚΑ (EL)
TÜRKÇE (TR)
РУССКИЙ (RU)
日本語 (JA)
中文 (ZH)
한국어 (KO)

Introduction

This instructions for use manual contains information intended to ensure the safe, effective and compliant use of your product. Keep and consult this reference manual during the life of the product.

Audience

This manual is intended for in-service trainers, biomedical equipment technicians, and central supply/sterile processing technicians.

Conventions

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

Contact Information

For additional information, including safety information, in-service training, or current literature, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

Indications For Use

The Stryker Sterilization Case is designed to contain an insert tray and protect reusable medical devices during steam sterilization, storage and transportation. The Sterilization Case must be wrapped in a validated sterilization wrap prior to sterilization to maintain sterility. The Stryker Insert Tray is designed to provide a convenient holder for reusable medical devices during automated washing and steam sterilization. This Insert Tray may also be placed in a rigid container that does not require a validated wrap.

Contraindications

None known.

Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	General warning sign

For Use With

Each sterilization case is intended for use with a specific insert tray model. See the instructions for use manual supplied with the appropriate insert tray for additional information.

Safety Directives

WARNINGS:

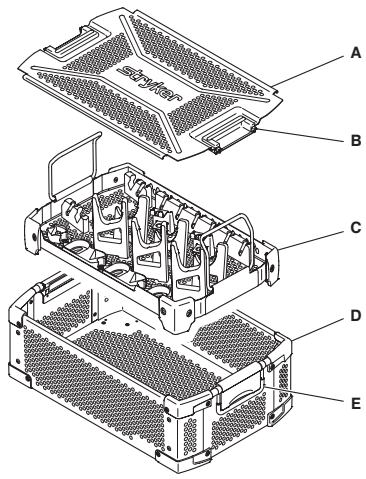
- Before using this equipment, or any component compatible with this equipment, read and understand the instructions for use. Pay particular attention to safety information. Become familiar with the equipment before use.
 - Only individuals trained and experienced in the processing of reusable medical devices should use this equipment.
 - See the instructions for use manual supplied with the appropriate insert tray for additional information on the safe and effective use of this equipment.
 - DO NOT reuse, reprocess, or repackage a device that is intended for single use only.
 - A single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing.
 - Design features may make cleaning difficult.
 - Reuse may create a contamination risk and compromise structural integrity resulting in operational failure or fragmentation during use.
 - Critical product information may be lost during repackaging.
- Failure to comply may lead to infection or cross infection and result in patient and/or healthcare staff injury.

Care Instructions

For general processing instructions, see the care instructions manual supplied with your device.

Features


NOTE: The 3/4 Size x 6" (150 mm) Case (REF 7102-454-040) and System 7 3 Handpiece Insert Tray (REF 7102-454-010) are shown for reference.

A	Lid
B	Latch
C	Insert Tray
D	Base
E	Handle


To Remove the Lid from the Base

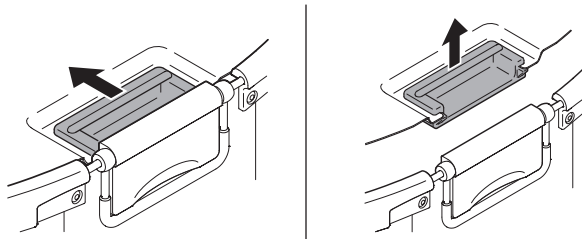Pull the two latches toward the center of the case and remove the lid from the base.

To Secure the Lid to the Base

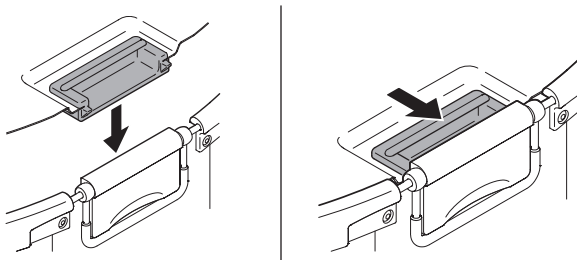Place the lid on the base and engage the two latches to secure the lid to the base.

Specifications

Model:	3/4 Size x 8" (200 mm) Case (REF 7102-450-040)
Size:	191 mm [7.5 inch] height 444 mm [17.5 inch] length 277 mm [10.9 inch] width
Weight:	2.96 kg [6.54 lb]
Material:	Stainless Steel Polyaryletheretherketone (PEEK) Thermoplastic Elastomer (TPE)

Model:	1/2 Size x 8" (200 mm) Case (REF 7102-452-040)
Size:	191 mm [7.5 inch] height 285 mm [11.2 inch] length 277 mm [10.9 inch] width
Weight:	2.43 kg [5.36 lb]
Material:	Stainless Steel Polyaryletheretherketone (PEEK) Thermoplastic Elastomer (TPE)

Model:	1/2 Size x 6" (150 mm) Case (REF 7102-453-040)
Size:	146 mm [5.7 inch] height 285 mm [11.2 inch] length 277 mm [10.9 inch] width
Weight:	2.05 kg [4.52 lb]
Material:	Stainless Steel Polyaryletheretherketone (PEEK) Thermoplastic Elastomer (TPE)

Model:	3/4 Size x 6" (150 mm) Case (REF 7102-454-040)
Size:	146 mm [5.7 inch] height 444 mm [17.5 inch] length 277 mm [10.9 inch] width
Weight:	2.65 kg [5.84 lb]
Material:	Stainless Steel Polyaryletheretherketone (PEEK) Thermoplastic Elastomer (TPE)

Model:	Full Size x 6" (150 mm) Case (REF 7102-455-040)
Size:	146 mm [5.7 inch] height 567 mm [22.3 inch] length 277 mm [10.9 inch] width
Weight:	3.46 kg [7.64 lb]
Material:	Stainless Steel Polyaryletheretherketone (PEEK) Thermoplastic Elastomer (TPE)

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stryker[®]

Signature History Report

Version	Signature	Role	Event Date	Vote
F	Hurley, Caitlyn	CA_Owner	2016-12-15 16:23:41 EST	Continue
F	Pearson, Dawn	CA_Tech Pubs-Tech Writer	2016-12-15 16:33:07 EST	Approve
F	Hurley, Caitlyn	CA_Product Design Engineer	2016-12-15 16:38:02 EST	Approve
F	Hurley, Caitlyn	CA_Owner	2016-12-16 08:52:43 EST	Continue
F	Hurley, Caitlyn	CA_Product Design Engineer	2016-12-16 08:54:17 EST	Approve
F	Pearson, Dawn	CA_Tech Pubs-Tech Writer	2016-12-16 08:54:31 EST	Approve
F	Sidow, Brian	CA_Clinical Sciences	2016-12-16 09:38:03 EST	Approve
F	Burke Adam	CA_Marketing Manager	2016-12-16 10:00:50 EST	Approve
F	Cunningham Gwynn	CA_Quality Engineer	2016-12-16 13:25:53 EST	Approve
F	Modi Karankumar	CA_Regulatory Affairs	2016-12-19 13:20:33 EST	Approve
F	Morgan, Carianne	CA_Product Safety	2016-12-23 10:30:53 EST	Approve
F	Schrauben Haley	Editor	2017-02-08 11:19:24 EST	Continue

NOTE: IF THIS REPORT IS BLANK, PLEASE SEE WINDCHILL ECN FOR SIGNATURES.

Related Information

CR Number	CR0305224
CR Class	DMR-Commercially Released
Originating Entity	INST: Kalamazoo
Origin	Product Improvement
Initial CR Approval Date	11/14/2016 15:17:25
CN Number	CN156400

Related Information

CA Number	CA189013
NPD Disposition Plan	