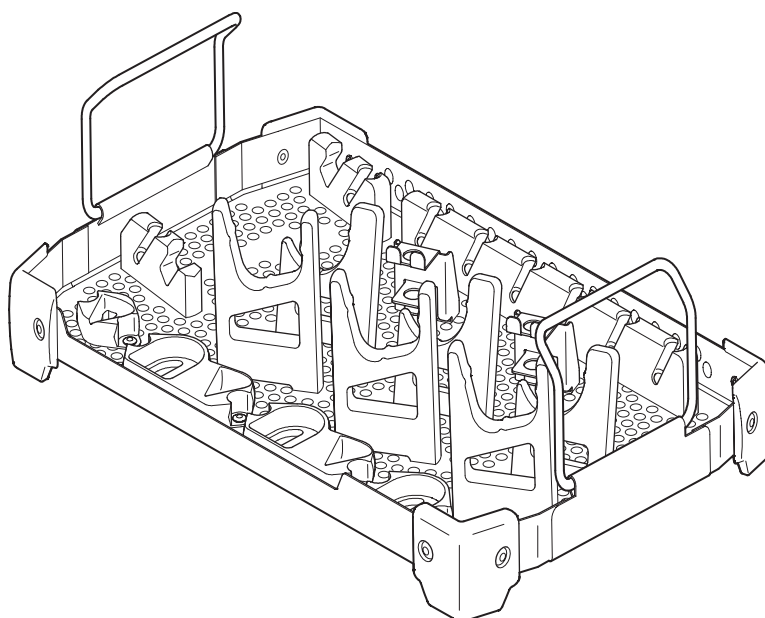
<p><i>Note: This portion of the document should not appear on produced Labels or IFUs.</i></p> <p>Stryker Instruments (269) 323-7700 (800) 253-3210</p>	<p>Dimensions: 8.5 inch (width) x 11 inch Booklet</p>	<p>Color/Material/Finish: Black Graphics on White Background 20# Bond or Equivalent</p>	<p>Label Stock: N/A</p>	
			<p>Description/Type: Instructions For Use</p>	
			<p>Part Number: 7102-450-701</p>	<p>Rev. D</p>

System 7 Insert Trays

REF	7102-450-010
	7102-452-010
	7102-453-010
	7102-454-010
	7102-455-010
	7102-458-010

Instructions For Use

R_x ONLY

ENGLISH (EN)
ESPAÑOL (ES)
DEUTSCH (DE)
FRANÇAIS (FR)
ITALIANO (IT)
NEDERLANDS (NL)
SVENSKA (SV)
DANSK (DA)
SUOMI (FI)
PORTUGUÊS (PT)
NORSK (NO)
POLSKI (PL)
ΕΛΛΗΝΙΚΑ (EL)
TÜRKÇE (TR)
РУССКИЙ (RU)
日本語 (JA)
中文 (ZH)
한국어 (KO)

Introduction

This instructions for use manual contains information intended to ensure the safe, effective and compliant use of your product. Keep and consult this reference manual during the life of the product.

This manual is intended for in-service trainers, biomedical equipment technicians, and central supply/sterile processing technicians.

The following conventions are used in this manual:

- A **WARNING** highlights a safety-related issue. ALWAYS comply with this information to prevent patient and/or healthcare staff injury.
- A **CAUTION** highlights a product reliability issue. ALWAYS comply with this information to prevent product damage.
- A **NOTE** supplements and/or clarifies procedural information.

For additional information, including safety information, in-service training, or current literature, contact your Stryker sales representative or call Stryker customer service. Outside the US, contact your nearest Stryker subsidiary.

Indications For Use

The Stryker Sterilization Case is designed to contain an insert tray and protect reusable medical devices during steam sterilization, storage and transportation. The Sterilization Case must be wrapped in a validated sterilization wrap prior to sterilization to maintain sterility. The Stryker Insert Tray is designed to provide a convenient holder for reusable medical devices during automated washing and steam sterilization. This Insert Tray may also be placed in a rigid container that does not require a validated wrap.

Contraindications

None known.

Definitions

The symbols located on the equipment and/or labeling are defined in this section or in the *Symbol Definition Chart*. See the *Symbol Definition Chart* supplied with the equipment.

SYMBOL	DEFINITION
	General warning sign

For Use With

WARNING: Use only Stryker-approved equipment, unless otherwise specified. DO NOT modify any equipment without the authorization of the manufacturer.

This section identifies components intended to be used with the equipment to obtain a safe combination.

Each insert tray is intended for use with a specific sterilization case or sterilization container model:

System 7 4 Handpiece Insert Tray (REF 7102-450-010)

DESCRIPTION		REF
3/4 Size x 8" (200 mm) Case		7102-450-040
3/4 Size x 8" (200 mm) Perforated Bottom Container	Lid	7102-550-030 (Aesculap Model JK789)
	Base	7102-550-020 (Aesculap Model JN744)
3/4 Size x 8" (200 mm) Solid Bottom Container	Lid	7102-550-030 (Aesculap Model JK789)
	Base	7102-650-020 (Aesculap Model JK744)

System 7 4 Handpiece Insert Trays are intended for use with the following Stryker reusable medical devices:

DESCRIPTION	REF
System 8 Handpieces	82XX-XXX-XXX series
System 7 Handpieces	72XX-XXX-XXX series
Heavy Duty Rotary Handpiece Attachments	8203-XXX-XXX series
	7203-XXX-XXX series
	6203-XXX-XXX series

System 7 2 Handpiece Insert Tray (REF 7102-452-010)

DESCRIPTION		REF
1/2 Size x 8" (200 mm) Case		7102-452-040
1/2 Size x 8" (200 mm) Perforated Bottom Container	Lid	7102-553-030 (Aesculap Model JK389)
	Base	7102-562-020 (Aesculap Model JN344)
1/2 Size x 8" (200 mm) Solid Bottom Container	Lid	7102-553-030 (Aesculap Model JK389)
	Base	7102-662-020 (Aesculap Model JK344)

System 7 2 Handpiece Insert Trays are intended for use with the following Stryker reusable medical devices:

DESCRIPTION	REF
System 8 Handpieces	82XX-XXX-XXX series
System 7 Handpieces	72XX-XXX-XXX series
Heavy Duty Rotary Handpiece Attachments	8203-XXX-XXX series
	7203-XXX-XXX series
	6203-XXX-XXX series

System 7 Sternum/Sabo® Insert Tray (REF 7102-453-010)

DESCRIPTION		REF
1/2 Size x 6" (150 mm) Case		7102-453-040
1/2 Size x 6" (150 mm) Perforated Bottom Container	Lid	7102-553-030 (Aesculap Model JK389)
	Base	7102-553-020 (Aesculap Model JN342)
1/2 Size x 6" (150 mm) Solid Bottom Container	Lid	7102-553-030 (Aesculap Model JK389)
	Base	7102-653-020 (Aesculap Model JK342)

System 7 Sternum/Sabo Insert Trays are intended for use with the following Stryker reusable medical devices:

DESCRIPTION	REF
System 8 Sabo Sagittal Saw	4508-000-000
Sabo2 Sagittal Saw	4408-000-000
Sabo Sagittal Saw	4300-034-000
System 8 Sternum Saw	8207-000-000
System 7 Sternum Saw	7207-000-000
Sternum Blade Guards	7207-XXX-XXX series

System 7 3 Handpiece Insert Tray (REF 7102-454-010)

DESCRIPTION		REF
3/4 Size x 6" (150 mm) Case		7102-454-040
3/4 Size x 6" (150 mm) Perforated Bottom Container	Lid	7102-550-030 (Aesculap Model JK789)
	Base	7102-552-020 (Aesculap Model JN742)
3/4 Size x 6" (150 mm) Solid Bottom Container	Lid	7102-550-030 (Aesculap Model JK789)
	Base	7102-652-020 (Aesculap Model JK742)

System 7 3 Handpiece Insert Trays are intended for use with the following Stryker reusable medical devices:

DESCRIPTION	REF
System 8 Handpieces	82XX-XXX-XXX series
System 7 Handpieces	72XX-XXX-XXX series
Heavy Duty Rotary Handpiece Attachments	8203-XXX-XXX series
	7203-XXX-XXX series
	6203-XXX-XXX series

System 7 Aseptic Housing Insert Tray (REF 7102-455-010)

DESCRIPTION		REF
Full Size x 6" (150 mm) Case		7102-455-040
Full Size x 6" (150 mm) Perforated Bottom Container	Lid	7102-555-030 (Aesculap Model JK489)
	Base	7102-555-020 (Aesculap Model JN442)
Full Size x 6" (150 mm) Solid Bottom Container	Lid	7102-555-030 (Aesculap Model JK489)
	Base	7102-655-020 (Aesculap Model JK442)

System 7 Aseptic Housing Insert Trays are intended for use with the following Stryker reusable medical devices:

DESCRIPTION	REF
Stryker SmartLife™ Aseptic Housings	7126-120-000
	7222-120-000
Stryker SmartLife Transfer Shields	7126-130-000 7222-130-000

System 7 Aseptic 2 Handpiece Insert Tray (REF 7102-458-010)

DESCRIPTION		REF
Full Size x 6" (150 mm) Case		7102-455-040
Full Size x 6" (150 mm) Perforated Bottom Container	Lid	7102-555-030 (Aesculap Model JK489)
	Base	7102-555-020 (Aesculap Model JN442)
Full Size x 6" (150 mm) Solid Bottom Container	Lid	7102-555-030 (Aesculap Model JK489)
	Base	7102-655-020 (Aesculap Model JK442)

System 7 Aseptic Housing Insert Trays are intended for use with the following Stryker reusable medical devices:

DESCRIPTION	REF
System 8 Handpieces	82XX-XXX-XXX series
System 7 Handpieces	72XX-XXX-XXX series
Heavy Duty Rotary Handpiece Attachments	8203-XXX-XXX series
	7203-XXX-XXX series
	6203-XXX-XXX series
Stryker SmartLife™ Aseptic Housing	7126-120-000
	7222-120-000
Stryker SmartLife Transfer Shield	7126-130-000
	7222-130-000

Safety Directives

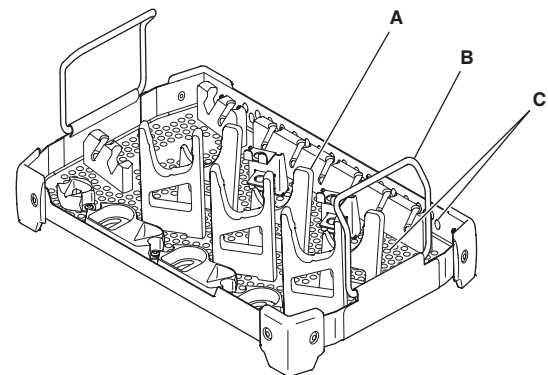
WARNINGS:

- Before using this equipment, or any component compatible with this equipment, read and understand the instructions for use. Pay particular attention to safety information. Become familiar with the equipment before use.
- Only individuals trained and experienced in the processing of reusable medical devices should process this equipment.
- DO NOT reuse, reprocess, or repackage a device that is intended for single use only.
 - A single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing.
 - Design features may make cleaning difficult.
 - Reuse may create a contamination risk and compromise structural integrity resulting in operational failure or fragmentation during use.
 - Critical product information may be lost during repackaging.
- Failure to comply may lead to infection or cross infection and result in patient and/or healthcare staff injury.
- See the instructions for use manual supplied with the appropriate sterilization case or sterilization container for additional information on the safe and effective use of this equipment.

Features


NOTE: The System 7 3 Handpiece Insert Tray (REF 7102-454-010) and 3/4 Size x 6" (150 mm) Case (REF 7102-454-040) are shown for reference.

Insert Tray

A	Bracket
B	Handle
C	Perforations (Holes)

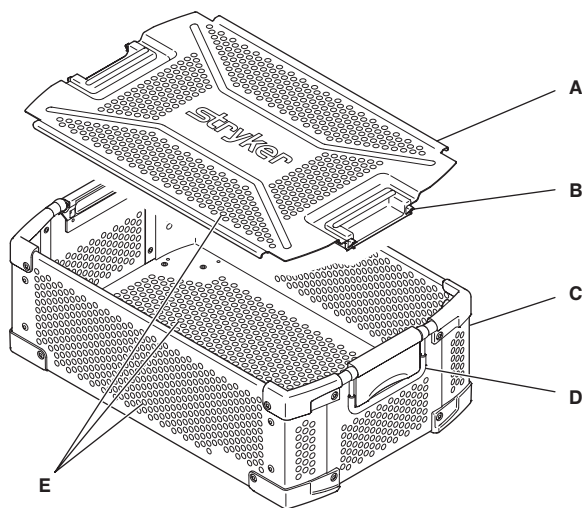
Sterilization Case

A	Lid
B	Latch
C	Base
D	Handle
E	Perforations (Holes)

Sterilization Container

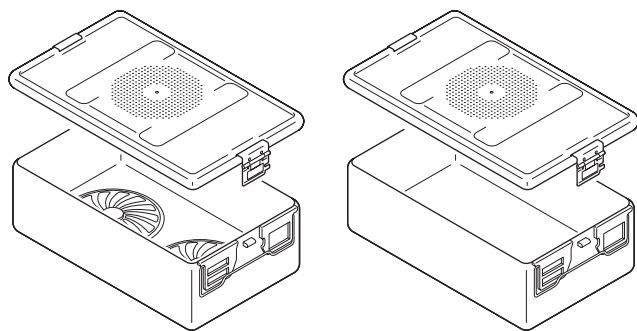NOTE: Insert trays may also be used with specific, perforated bottom or solid bottom sterilization containers. See the *For Use With* section for appropriate part numbers.

Perforated Bottom

Solid Bottom

Maximum Quantity of Devices

WARNING: DO NOT exceed the specified maximum quantity of devices in an insert tray. The maximum quantity of devices is based on Stryker validation of this equipment. Exceeding the maximum quantity of devices may prevent proper sterilization of the equipment and/or result in excessive weight. Always follow the current local recommendations and/or regulations governing weights and weight limits.

System 7 4 Handpiece Insert Tray (REF 7102-450-010)

DEVICE TYPE	MAXIMUM QUANTITY
Handpiece	4
Attachment	10
Accessory (Chuck Key)	2
Total of All Devices	16

System 7 2 Handpiece Insert Tray (REF 7102-452-010)

DEVICE TYPE	MAXIMUM QUANTITY
Handpiece	2
Attachment	8
Accessory (Chuck Key)	2
Total of All Devices	12

System 7 Sternum/Sabo Insert Tray (REF 7102-453-010)

DEVICE TYPE	MAXIMUM QUANTITY
Handpiece	2
Attachment	2
Total of All Devices	4

System 7 3 Handpiece Insert Tray (REF 7102-454-010)

DEVICE TYPE	MAXIMUM QUANTITY
Handpiece	3
Attachment	8
Accessory (Chuck Key)	2
Total of All Devices	13

System 7 Aseptic Housing Insert Tray (REF 7102-455-010)

DEVICE TYPE	MAXIMUM QUANTITY
Aseptic Housing	4
Transfer Shield	4
Total of All Devices	8

System 7 Aseptic 2 Handpiece Insert Tray (REF 7102-458-010)

DEVICE TYPE	MAXIMUM QUANTITY
Handpiece	2
Attachment	7
Accessory (Chuck Key)	2
Aseptic Housing	2
Transfer Shield	2
Total of All Devices	15

Recommended Placement of Devices

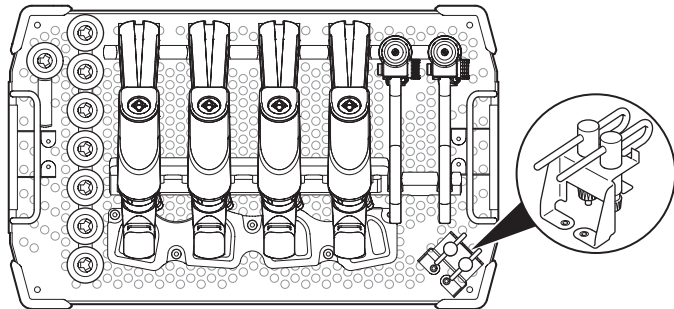
! WARNINGS:

- ALWAYS make sure the equipment is clean and dry before sterilization.
- ALWAYS make sure handpieces are oriented with the distal end down.
- ALWAYS make sure handpiece and attachment lumens remain at a vertical angle to allow water to sufficiently drain.


NOTES:

- See the *Maximum Quantity of Devices* section.
- The illustrated placement of devices is a recommendation only. Other placement combinations are acceptable.


System 7 4 Handpiece Insert Tray (REF 7102-450-010)

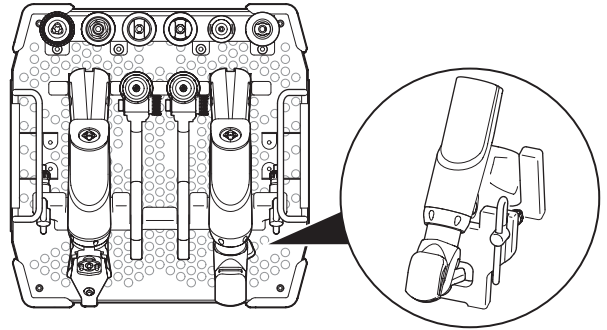
System 7 2 Handpiece Insert Tray (REF 7102-452-010)

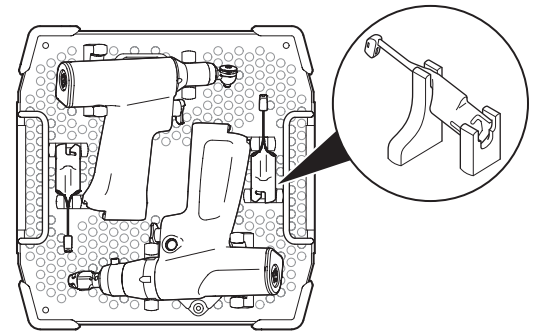
System 7 Sternum/Sabo Insert Tray (REF 7102-453-010)

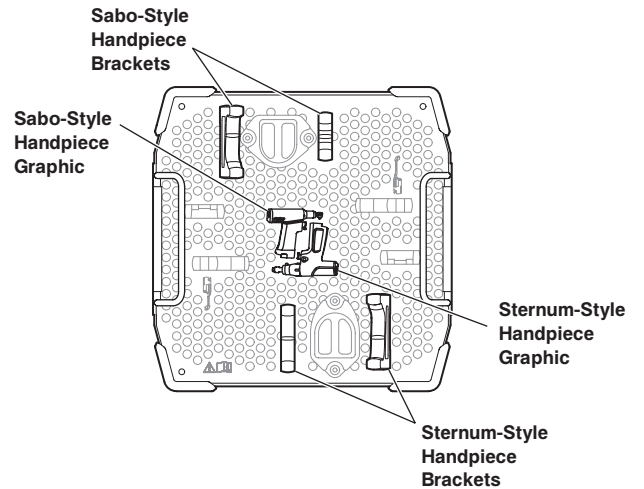
NOTE: When loading a Sternum/Sabo Insert Tray, use the tray graphics to assist in achieving the recommended placement of devices.


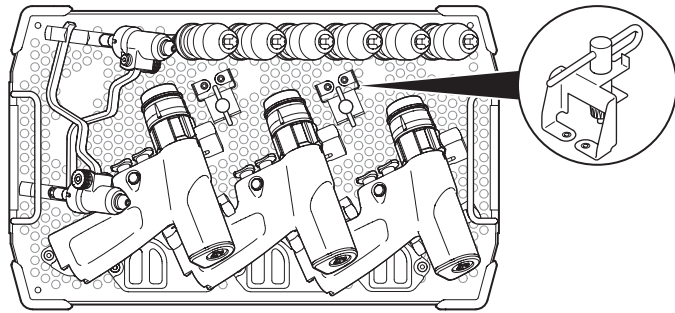
System 7 3 Handpiece Insert Tray (REF 7102-454-010)

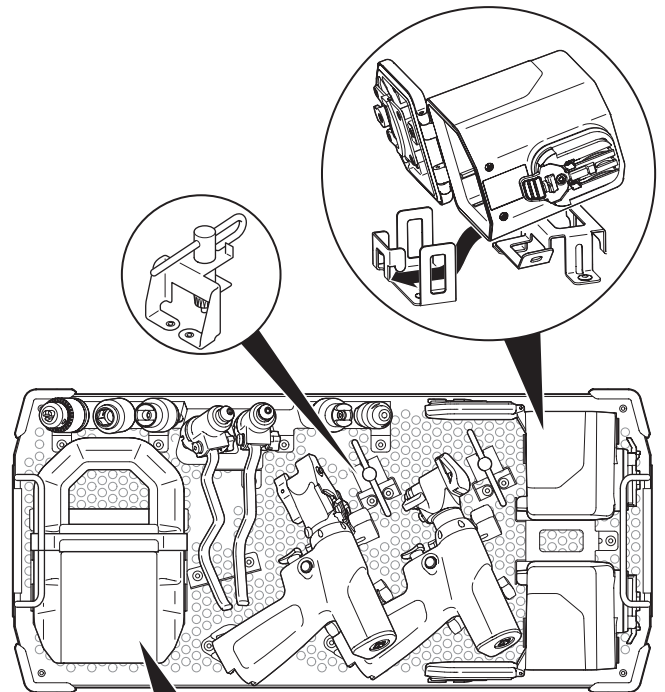System 7 Aseptic 2 Handpiece Insert Tray (REF 7102-458-010)

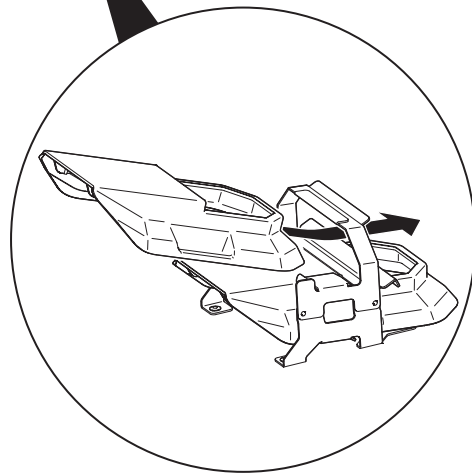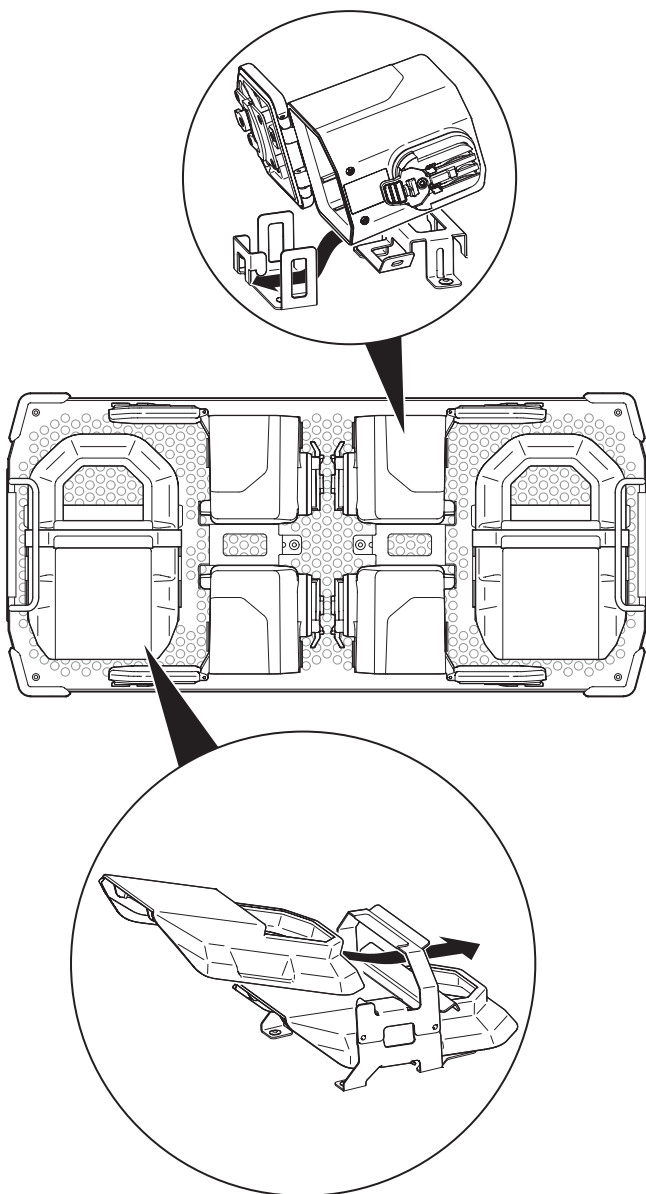System 7 Aseptic Housing Insert Tray (REF 7102-455-010)


Care Instructions

For general processing instructions, see the care instructions manual supplied with your device.

Point of Use (Pre-surgery)

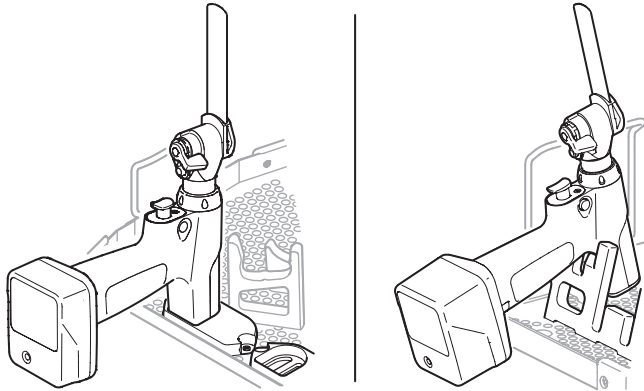Insert tray brackets may be used to hold fully assembled, sterile handpieces in the operating room prior to surgery.

Specifications

Model:	System 7 4 Handpiece Insert Tray (REF 7102-450-010)
Dimensions:	159 mm [6.3 inch] height 406 mm [16.0 inch] length 251 mm [9.9 inch] width
Mass:	1.27 kg [2.81 lb]
Material:	Stainless Steel Polypropylene Polyaryletheretherketone (PEEK)
Model:	System 7 2 Handpiece Insert Tray (REF 7102-452-010)
Dimensions:	159 mm [6.3 inch] height 247 mm [9.7 inch] length 251 mm [9.9 inch] width
Mass:	1.00 kg [2.20 lb]
Material:	Stainless Steel Polypropylene Polyaryletheretherketone (PEEK)
European Conformity:	CE

Model:	System 7 Sternum/Sabo Insert Tray (REF 7102-453-010)
Dimensions:	113 mm [4.4 inch] height 247 mm [9.7 inch] length 251 mm [9.9 inch] width
Mass:	0.80 kg [1.76 lb]
Material:	Stainless Steel Polypropylene Polyaryletheretherketone (PEEK)
European Conformity:	CE

Model:	System 7 3 Handpiece Insert Tray (REF 7102-454-010)
Dimensions:	113 mm [4.4 inch] height 406 mm [16.0 inch] length 251 mm [9.9 inch] width
Mass:	1.15 kg [2.54 lb]
Material:	Stainless Steel Polypropylene Polyaryletheretherketone (PEEK)
European Conformity:	CE

Model:	System 7 Aseptic Housing Insert Tray (REF 7102-455-010)
Dimensions:	113 mm [4.4 inch] height 529 mm [20.8 inch] length 251 mm [9.9 inch] width
Mass:	1.68 kg [3.70 lb]
Material:	Stainless Steel Polypropylene Polyaryletheretherketone (PEEK)
European Conformity:	CE

Model:	System 7 Aseptic 2 Handpiece Insert Tray (REF 7102-458-010)
Dimensions:	113 mm [4.4 inch] height 529 mm [20.8 inch] length 251 mm [9.9 inch] width
Mass:	1.72 kg [3.79 lb]
Material:	Stainless Steel Polypropylene Polyaryletheretherketone (PEEK)
European Conformity:	CE

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stryker[®]

Signature History Report

Version	Signature	Role	Event Date	Vote
D	Hurley, Caitlyn	CA_Owner	2016-12-16 09:35:57 EST	Continue
D	Sidow, Brian	CA_Clinical Sciences	2016-12-16 09:39:27 EST	Approve
D	Hurley, Caitlyn	CA_Product Design Engineer	2016-12-16 09:39:30 EST	Approve
D	Burke Adam	CA_Marketing Manager	2016-12-16 10:01:05 EST	Approve
D	Pearson, Dawn	CA_Tech Pubs-Tech Writer	2016-12-16 10:07:55 EST	Approve
D	Cunningham Gwynn	CA_Quality Engineer	2016-12-16 13:58:38 EST	Approve
D	Modi Karankumar	CA_Regulatory Affairs	2016-12-19 12:11:48 EST	Approve
D	Morgan, Carianne	CA_Product Safety	2016-12-23 10:40:40 EST	Approve
D	Schrauben Haley	Editor	2017-02-08 11:17:01 EST	Continue

NOTE: IF THIS REPORT IS BLANK, PLEASE SEE WINDCHILL ECN FOR SIGNATURES.

Related Information

CR Number	CR0305224
CR Class	DMR-Commercially Released
Originating Entity	INST: Kalamazoo
Origin	Product Improvement
Initial CR Approval Date	11/14/2016 15:17:25
CN Number	CN156400
CA Number	CA189011
NPD Disposition Plan	