Distributed by:

stryker[®]

Stryker Endoscopy
5900 Optical Court,
San Jose, CA 95138 (USA)
+1.800.624.4422
+1.408.754.2000

CHAMPION[®] PASSER Needle

REF 3910-900-091

Lincotek Medical
867 West 400 North
Logan, UT 84321 (USA)
+1.435.753.7675

EC REP
Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

1113-0046 Rev. K 2021-03

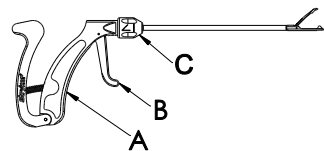
Figure 1

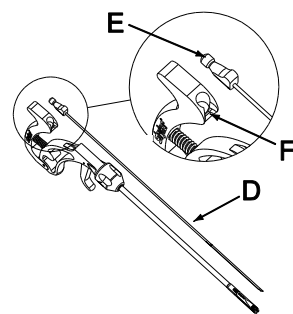
Figure 2

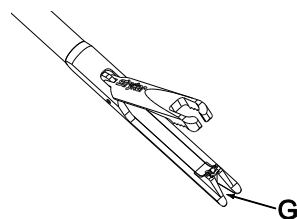
Figure 3

ENGLISH

CHAMPION[®] PASSER Needle

DESCRIPTION

The Champion[®] Passer Needle, when used with the RP 360[®]™ and Champion[®] Suture Passers is an orthopedic manual surgical instrument designed to pass USP#2 braided surgical suture through tissue in either arthroscopic or open surgery. The Champion[®] Passer Needle is only to be used with the RP 360[®]™ Suture Passer (PN 3910-900-090) and Champion[®] Suture Passer (3910-900-086, 3910-900-087, 3910-900-088, 3910-900-089).

FIGURE 1

A. Handle, B. Jaw Lever, C. Hub (PN 3910-900-090)

FIGURE 2

D. Needle, E. Knob, F. Slot

FIGURE 3

G. Slot

MATERIALS

Nitinol
Acrylonitrile butadiene styrene (ABS)
304 Stainless Steel

INDICATIONS FOR USE

RP 360[®]™ Suture Passer and Champion[®] Suture Passers used with the Champion[®] Passer Needle are indicated for passing suture through tissue in orthopedic surgery.

⚠️ CONTRAINDICATIONS

Do not use on bone or other hard tissue.

⚠️ WARNINGS

1. Federal Law restricts this device to sale, distribution, or use by or on the order of a physician.
2. The RP 360[®]™ and Champion[®] Suture Passer used with the Champion[®] Passer Needle must be used under direct visualization.
3. Use of excessive force to pass the needle through tissue, or striking bone, may cause needle breakage and patient injury.
4. In the rare event of needle breakage, fragments can be located visually or by use of radiographic imaging equipment, and removed manually through the incision site.

⚠️ PRECAUTIONS

1. Prior to use read carefully and in its entirety the instructions for use.
2. The device should be used only by physicians with a thorough understanding of orthopedic procedures.
3. Champion[®] Passer Needles are intended for **SINGLE USE ONLY. DO NOT RESTERILIZE OR REUSE.**
4. If the needle gets reused the potential risks include:
 - Needle breakage in instrument or patient.
 - Non-sterile needle resulting in an infection.
 - Unclean needle resulting in an infection.
 - Instrument damage due to compromise of coating on needle.
 - The coating is compromised causing residue to be left in patient.
 - Compromised instrument/needle function - needle won't pass or needle won't carry suture.
5. The Champion(R) Passer Needle should not be passed more than 8 cycles per surgery.

PACKAGING, LABELING AND STERILIZATION

Champion[®] Passer Needle are supplied sterile (gamma radiation) and are intended for single use only. Use only if factory supplied packaging and labeling are intact.

STORAGE: Store in a cool dry place.

INSTRUCTIONS FOR USE

Load the Champion[®] Passer Needle into the Stryker[®] Suture Passer:

1. Locate the cannulated shaft at the proximal end of the Suture Passer (Figure 2). Insert the needle into the cannulation with the fin on the handle oriented in the same direction as the open top jaw of the suture passer. Push the needle forward until the knob drops into the slot in the Suture Passer handle. (Figure 2). Ensure that the needle knob is fully seated in the slot in the Suture Passer handle.

Load Suture:

2. Grasp the distal tip of the passing suture. Slide the suture into the V-shaped slot in the lower jaw (Figure 3) with the free end between the upper and lower jaws. The suture will snap into place. Suture management may be improved if the free end is 2 to 3cm long.

CAUTION: Use only #2 braided surgical suture with the RP 360[®]™ or Champion[®] Suture Passer

and Champion[®] Passer Needle.

If using the device in open surgery omit the following references to arthroscopic cannula.

Introduce and Pass Suture:

3. With the suture loaded, the jaws of the passer closed, and the needle retracted, pass the distal tip of the suture passer through the cannula to the surgical site.
4. When ready to pass suture, open and position the jaws of the Suture Passer at the desired location. Close the jaws by GENTLY squeezing the jaw lever with the first two fingers. To deploy the needle/suture squeeze the handle with the palm while still holding the jaw lever in the closed position.
5. After passing the needle/suture release the grip on the handle to allow the needle to fully retract, while keeping the jaw closed to maintain a grip on the tissue. Grasp the loop of suture that was passed through the tissue with a hook or grasper from another portal. Open the jaws, thereby releasing the tissue, and remove the Suture Passer from the tissue. Close the jaws and remove the suture passer from the cannula. Alternately the loop of suture may be grasped using the jaws of the suture passer (3910-900-090, 3910-900-086, 3910-900-087) or captured within the upper jaw (3910-900-088, 3910-900-089).
6. After removing from the cannula, if the suture is captured within the upper jaw, close the jaw lever then partially deploy the needle, with the palm, opening the trap door and releasing the suture.
7. After completion of the surgery immediately remove and discard the disposable needle.

Rotate Jaw Relative to Handle:

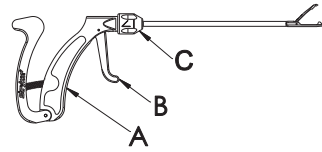
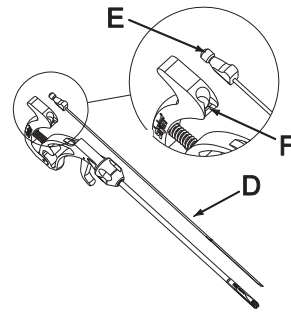
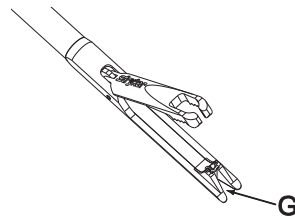
The jaw of the RP 360[®]™ Suture Passer (3910-900-090) may be rotated to four discrete positions (at 90° increments) relative to the handle by grasping the red hub and twisting to the desired location. As the hub is rotated from one position to another the user will feel the hub "snap" into place indicating it is in the correct position. The jaw of the RP 360[®]™ Suture Passer may be rotated relative to the handle at any time during the use of the device, including use after the RP 360[®]™ Suture Passer is in the joint and suture is loaded.

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