

3D SYSTEMS

CT scanning
protocol



3D SYSTEMS

CT scanning protocol

Thank you for taking a moment to read this protocol. The quality of the CT or CBCT scan is the most important aspect of creating case-specific anatomical models. Your observation of the recommendations made in this protocol will have a significant impact on the accuracy of the final model. We understand concerns about keeping the radiation dose to your patients as low as reasonably achievable, therefore, please apply these guidelines as appropriate to your patients. Please do not hesitate to contact your Sales Representative.

Please keep in mind the following key points

- Please use a 3D scanning routine that provides high resolution images as would be suitable for image guided surgery, stereotactic planning or other 3D applications. It may be useful to consult with your CT vendor's Application Specialist for advice on optimal parameters for your machine that provide the best scan with acceptable radiation dose levels.
- Acquire scans at a high spatial resolution. Series should be acquired with thin, contiguous image slices (equivalent thickness and spacing of 1.25 mm or less) and as small a field of view (FOV) as possible while still including the patient's anatomy of interest.
- Please provide images in the original scanning plane. If software post-processing is performed to reorient or reformat the scan volume, then a series of thin slice images in the original acquisition plane **MUST** be included.
- Do not use gantry tilt during image acquisition. Images acquired with gantry tilt then post-processed to reorient images (i.e. "take out" tilt) are not acceptable.
- Please ensure that scans are free from motion artifact. Patient must remain completely still through the entire scan. If patient motion occurs, the scan must be restarted. Image distortion from patient motion can severely compromise the accuracy of a model.
- Image artifact caused by metallic implants can obscure anatomy of interest. Please take steps to minimise artifact from the presence of metal. It is useful to position the patient so that the occlusal plane is parallel to the image plane (see figure). This can help to limit artifact from metallic dental restorations to the region around the teeth.
- Archive the entire study in uncompressed DICOM format on CD-R or DVD for shipping.

Recommended protocol for medical CT scanners


Scan spacing	1.25 mm (equal to slice thickness)
Pixel size	0.60 mm or less
Field of view	20.0 - 25.0 cm
Algorithm (examples)	GE: standard (not bone or detail) (Siemens: H30s, Toshiba: FC20, Philips: B)
Gantry tilt	0°
Archive media	CD or DVD
File type	DICOM (uncompressed)
Series	Original/primary/axial (no recon, reformat or post process data)

Recommended protocol for medical CBCT scanners

Scan time	Longest available
Voxel size	0.3 - 0.5mm
Field of view	Largest available
File type	CT (one file per slice)
Reconstruction	Axial
Compression	Uncompressed

Patient positioning

Occlusal plane should be parallel to the gantry.

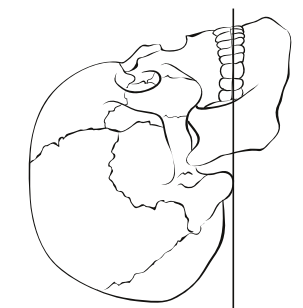

Please visit
www.3dsystems.com/medicaldata for digital transfer of DICOM images.



CMF

This document is intended solely for the use of healthcare professionals.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders.

The products depicted are CE marked in accordance with applicable EU Regulations and Directives.

This material is not intended for distribution outside the EU and EFTA.



CMF-FL-62_14141EN
 DLS 03/2018
 2017-16442

Copyright © 2018 Stryker
stryker.com

Manufactured by:


 3D Systems
 5381 South Alkire Circle
 Littleton, Colorado 80127 USA
 t: +1 844.643.1001
 (toll-free US/Canada)
 t: +1 720.643.1001
 f: +1 720.643.1009
www.3dsystems.com/healthcare
vsp@3dsystems.com
 MM-564 Rev A

Distributed by:

Stryker Leibinger GmbH & Co. KG
 Bötzingen Straße 41
 79111 Freiburg
 Tel: +49 (0) 761 4512-0