

Transmission skills checklist



with the **LIFEPAK® 35** Monitor/Defibrillator

Transmitting a 12/15-lead ECG report

Name: _____ Unit: _____


Reviewer: _____ Date: _____

This skills checklist assesses a trained user’s ability to perform data transmission with the LIFEPAK 35 monitor/defibrillator. This limited checklist does not cover all information and skills required to operate the device safely and effectively. This checklist is designed to be completed after receiving or viewing in-service training by a qualified instructor. Refer to the LIFEPAK 35 monitor/defibrillator Operating Instructions for complete directions for use, indications, contraindications, warnings, cautions, and potential adverse events. CAPITAL LETTERS indicate labels, screen messages, and voice prompts. “Push” is used when referring to keypad buttons, and “Press” is used when referring to softkeys on the touchscreen indicated by italics

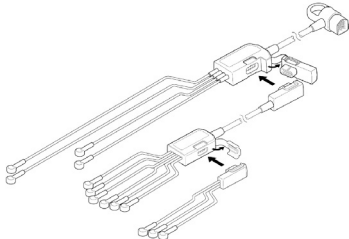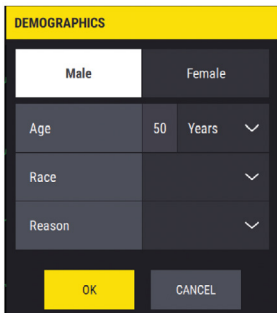Assessment criteria	Complete	Incomplete	Comments
1. Push the POWER button.			
2. Insert the lead attachments into the main cable. 			
3. Attach the ECG cable to the green connector monitor .			
4. Prepare the patient’s skin for electrode application according to local protocol or Operating Instructions			
5. Apply ECG electrodes.			
6. Press the 12-LEAD button at the bottom of the Home Screen. The LIVE view appears and shows all connected ECG Leads.			<p>Note: If a 10-wire cable is attached, the LIVE 12-LEAD view appears, showing all 12 leads.</p> <p>If a 13-wire cable is attached, the LIVE 15-LEAD view appears. Scroll up or down to view the additional leads.</p>
7. Press ACQUIRE at the top right corner of the LIVE screen.			Note: The first time you press ACQUIRE , the DEMOGRAPHICS dialog box appears
8. Enter patient’s sex, age, and race in the DEMOGRAPHICS dialog box. 			Note: If sex and age were already entered into the patient record, they will be automatically filled in and cannot be changed in dialog box.

9. When the device is finished acquiring and analyzing the ECG, an ECG report appears.

10. Press the **TRANSMIT** button at the top of the onscreen report.

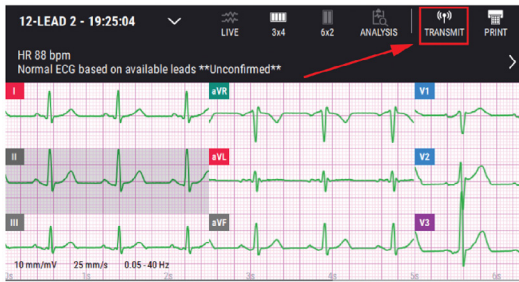


Note: Transmission sites that accept 12/15-lead reports must be set up in your LIFENET account to enable this feature.


11. Select test site to transmit to.

Note: If prompted for a transmission site selection, choose 'Test' site. It is not recommended to transmit to another organization for the purpose of testing.


12. To transmit a previously-acquired 12/15-lead report, press the down arrow shown to open the 12-lead report list.

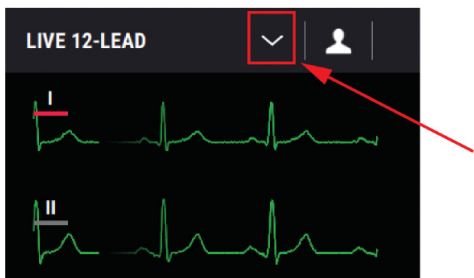

13. Press the report to view and transmit.

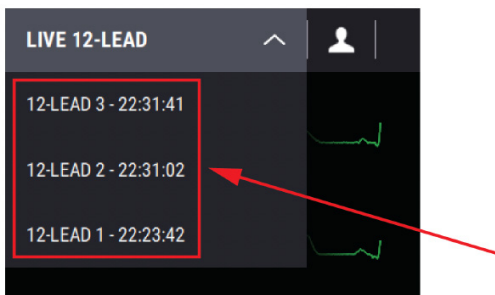

14. Press the **TRANSMIT** button at the top of the onscreen report.

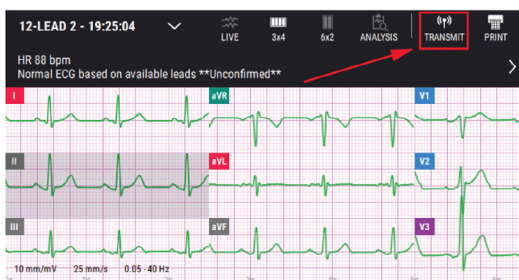

Notes: _____



Scan here for
additional resources

This document is intended solely for the use of healthcare professionals. A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate Stryker's product offerings. A healthcare professional must always refer to operating instructions for complete directions for use indications, contraindications, warnings, cautions, and potential adverse events, before using any of Stryker's products. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your representative if you have questions about the availability of Stryker's products in your area. Specifications subject to change without notice.

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: LIFENET, LIFEPAK, Stryker. All other trademarks are trademarks of their respective owners or holders.

The absence of a product, feature, or service name, or logo from this list does not constitute a waiver of Stryker's trademark or other intellectual property rights concerning that name or logo.

EC-L35-ASTL-1169371_REV-0_en_us © 2024 Stryker



Manufactured by:

Physio-Control, Inc.
11811 Willows Road NE
Redmond, WA, 98052 U.S.A.
Toll free 800 442 1142
stryker.com/emergencycare