


# Equipment guide

- 1 Console, irrigation pole, foot pedal
- 2 Handpiece and tray
- 3 Tip

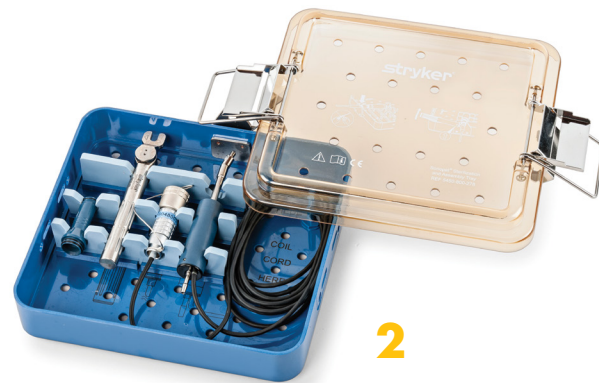

Handpiece and wrench come inside tray.

Don't remove tray or wrench from sterile field; you'll use it to set up the handpiece.



# Equipment guide

- 4 Tubing 5450-850-003
- 5 Disposable suction canister liner 5450-850-002 and reusable plastic suction canister 5450-850-001 **or** Disposable plastic suction canister 5450-851-012
- 6 Saline/irrigation bag (1000 mL)



Pass the shorter tubing with round filter off sterile field



**4**  
Pass this end off sterile field



Keep this end on sterile field, connects to handpiece



**If you have questions or need additional help, please get in touch with your sales representative or call customer service at 800 253 3210.**

## Neuro Spine ENT

This document is intended solely for the use of healthcare professionals. A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate a Stryker product. A surgeon must always refer to the package insert, product label and/or instructions for use, including the instructions for cleaning and sterilization (if applicable), before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its affiliates own, use, or have applied for the following trademarks or service marks: SONOPET, Stryker. All other trademarks are trademarks of their respective owners or holders.

Literature Number: 9100-002-174 Rev. A

Copyright © 2016 Stryker

### Manufactured by:

Stryker Neuro Spine ENT  
4100 East Milham Avenue  
Kalamazoo, MI 49001 USA  
Tel: 269 323 7700  
Fax: 269 353 3026  
Toll free: 800 253 3210

stryker.com/nse