

# Sage PrimoFit+™ External Urine Management for the Male Anatomy

**REF** 5405-X



## Packaging Information

**Contents:** (1) PrimoFit+

- Single use only
- Not made with natural rubber latex
- Non-sterile

### Instructions:

Read instructions on package prior to using product

### Indications:

For use on patients who require urine management. Use of the urine management devices helps to reduce the risk of developing CAUTI by minimizing urinary indwelling catheter use and duration. For use by healthcare professional.

### Contraindications:

Do not use on patients with urinary retention

### Cautions:

- Not intended for use on patients in the prone position
- Not intended for use on patients with female anatomy
- Not intended for use on stool collection
- Use caution if using on patients with altered mental status

### Discontinue use:

- When independent or assisted toileting is feasible
- During patient ambulation

## Specifications

Product name	Sage PrimoFit+ External Urine Management for the Male Anatomy
Product code	5405-X
Description	External Urine Management for the Male Anatomy
Manufacturer accreditation	ISO 13485:2016
Expiration date	None
Height of product	Device: 50.8 cm
Length of product	Device: 20.3 cm
Case quantity	40

# Sage PrimoFit+ External Urine Management System for the Male Anatomy

## Features and benefits

### Sage Primofit+ External Urine Management for the Male Anatomy

Stays in place:	<ul style="list-style-type: none"> <li>• Adhesive pad and base adhesive keeps device in place and allows for easy application and removal.</li> </ul>
Customizable fit:	<ul style="list-style-type: none"> <li>• Base adhesive eliminates the need to use sizing charts and promotes process efficiency.</li> </ul>
Easy to see:	<ul style="list-style-type: none"> <li>• Blue liners are easy to see after removal to reduce the risk of misplacement.</li> </ul>
Keeps skin dry:	<ul style="list-style-type: none"> <li>• Ultra soft fabric diverts urine away from the skin.</li> </ul>
Reduces leakage:	<ul style="list-style-type: none"> <li>• Semi-closed top reduces the risk of leakage while maintaining airflow.</li> </ul>
Core Structure:	<ul style="list-style-type: none"> <li>• Absorbs fluids to be suctioned to collection canister. Diverts fluids away from skin, helping to reduce risk factors associated with IAD.</li> </ul>
Wear time:	<ul style="list-style-type: none"> <li>• Can be left in place for up to 24 hours.</li> </ul>
Easy connection port:	<ul style="list-style-type: none"> <li>• Connects device to hospital suction tubing. Allows for easy and secure connection to hospital suction tubing.</li> </ul>



Sage Products LLC  
 3909 Three Oaks Road  
 Cary, IL 60013 USA  
 800 323 2220 ; +1 815 455 4700

**stryker.com**

SAGE-UM-OTHD-894155\_REV-0\_en\_us  
 Copyright © 2024 Stryker  
 11/2023

**This document is intended solely for the use of healthcare professionals.**

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: PrimoFit+, Stryker. All other trademarks are trademarks of their respective owners or holders.

The products depicted are CE marked in accordance with applicable EU Regulations and Directives.

PrimoFit+ is a Class I medical device CE.

This document is not intended for distribution outside EU, UK and EFTA.