

Sage Comfort Rinse-Free Shampoo Cap

REF 7409-X



Features and benefits

Rinse-Free Shampoo Cap

Rinse-Free	• Rinse-Free, no running water needed
All-in-one shampoo plus conditioner	• Gently cleanses and conditions in one easy step
Shampoo cap	• Can be warmed for soothing comfort

Specifications

Product name	Sage Comfort Rinse-Free Shampoo Cap
Product code	7409-X
Description	Pre-moistened fabric lined cap
Manufacturer accreditation	ISO 13485:2016
Expiration date	24 months from date of manufacture
Amount of packages per case	40

Ingredients:

Water (aqua)
 Isostearamidopropyl morpholine lactate
 Disodium cocoamphodiatetate
 DMDM hydantoin
 Phenoxyethanol
 Methylparaben
 Butylparaben
 Ethylparaben
 Propylparaben
 Fragrance (parfum)
 Simethicone

Packaging information

Contents:

(1) Rinse-Free Shampoo Cap

Indications of Use:

Pre-moistened fabric-lined cap helps eliminate mess and leaves hair fresh and clean.

Instructions for Use:

Tear open package. If desired, warm entire package in the microwave (1000W) no longer than 30 seconds. Caution: Test temperature of cap prior to use. Wait if excessively hot. Place cap on head and massage until hair is fully saturated (1-3 minutes). Remove, towel dry, comb hair.

Not made with natural rubber latex.
 Single use only.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders.

For more information please contact your local Stryker sales representative.

Stryker Australia Pty Ltd
 8 Herbert Street
 St Leonards NSW 2065
 61 2 9467 1000
 www.stryker.com.au

Stryker New Zealand Limited
 PO Box 17-136, Greenlane
 Auckland 1546, New Zealand
 64 9 573 1890
 www.stryker.com